



PROFESSIONAL MEDICAL PRODUCTS

DEAMBULATORE PRINCE - pieghevole - azzurro
PRINCE ROLLATOR - foldable - light blue
DÉAMBULATEUR PRINCE - pliable - bleu clair
ANDADOR PRINCE - plegable - azul claro

Manuale di istruzioni - Instruction Manual - Mode d'emploi - Manual de instrucciones

È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.

Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

REF HCT-9166B (Gima 43166)



Foshan HCT Medical Equipment Co.,
Ltd No.11,Dongyang 4th Road, Southern China Hardware Industry Base, Danzao Town,
Nanhai District, Foshan City, Guangdong Province, China
Made in China



SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE Amsterdam, Netherlands



Gima S.p.A
Via Marconi, 1 - 20060 Gessate (MI) - Italy
gima@gimaitaly.com - export@gimaiatly.com
www.gimaitaly.com



Thank you for choosing the Aluminium Folding Rollator.
Please read this instruction manual carefully before attempting to use it.
If you need any further assistance, contact our customer services department.
We hope you will enjoy using your new product.

GENERAL INFORMATION

The Rollator is designed for both indoor and outdoor use. It is designed to be used as a walker, not a wheelchair.

Do not sit on the device while it is moving.



DO NOT use the device if you suffer with any of the following impairments:

- Visual problems
- Inability to sit
- Severe balance issues
- Arm, shoulder or hand disabilities

PRODUCT INFORMATION

The Rollator comes carefully packed in a cardboard box which can be used for storage if required.
Please contact us immediately if the product is damaged on arrival.

PRODUCT OVERVIEW



1. Seat Back Belt
2. Mesh Bag
3. Front Wheel Locker
4. Front Wheel
5. Brake Adjustment Screw
6. Back Wheel
7. Tilt Support
8. Back Wheel Locker
9. Seat Pad with Folding Handle
10. Height Adjustment Locker
11. Brake Handle
12. Arm rest Handle

** Product images are for reference only.

PACKAGE CONTENTS

1. Rollator	x 1 body
2. Instruction Manual	1
3. Accessories	1 Cane Holder
	1 Seat Back Belt
	1 Mesh Bag

QUICK USER GUIDE INSTALLATION



Step 1: Install the front wheel


Press the red button.

Adjust the direction of the wheel, insert it and lock firmly.



Step 2: Install the back wheel Align the slot of the metal leg with the Rollator body.

You will hear a click if this has been inserted correctly. Then tighten the screw on the Handle to secure.

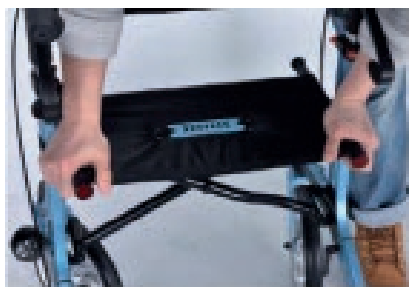

Step 3: Push on the seat

Open the Rollator and push on the seat from the edges.

You will hear a click when it is safely in place. Double check it is firmly installed before sitting down.



Step 4. Install the back belt and mesh bag

Install the belt following the pictures displayed





QUICK FIXES

Adjust the height of arm rest

Press the red button and adjust the height of armrest to a comfortable position.
Make sure both arm rests are fixed at the same height.



Operate the brakes

Pull both brake handles upwards to activate the brakes.



Park

Push the brake handles downwards until they lock into place with a click to park the device.
Pull the brake handles back up to come out of parking position.

The seat could be used ONLY in parking position.



Going over obstacles

Step on the tilting aid.

Then pull back on the handles at the same time to raise the front wheels. This will allow you to negotiate kerbs and other small obstacles with little effort.

This will allow you to negotiate kerbs and other small obstacles with little effort.



Adjust the brakes (optional)

Loosen the allen screw by pulling it out. (arrow 1)

Do not move the Rollator when adjusting the brakes.

To make small adjustments: Loosen the counter nut. (arrow 1)

Turn the setup screw (arrow 2)

clockwise to reduce the braking intensity. Turn the setup screw counter clockwise to increase the braking intensity.


Finally, retighten the counter nut.



STORAGE

To fold and store the Rollator, simply pull up the handle on the seat to collapse the device.

REPAIR PARTS

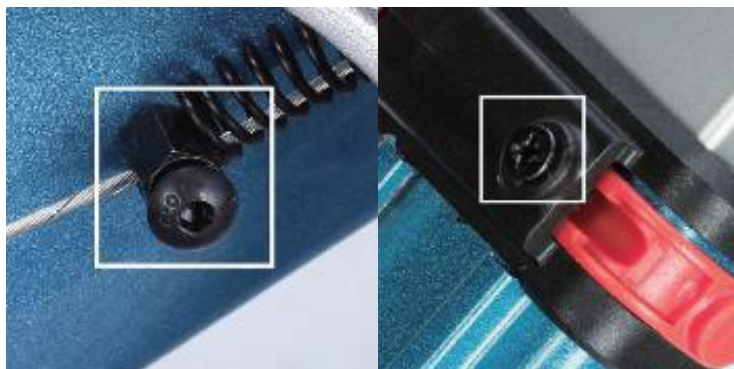

TIPS FOR FIRST USE

Check the brakes and wheels before using.

The Rollator should ideally be used on dry, level ground. Practice using it inside until you are confident to use it outdoors. When using the Rollator, rest your hands on the brake to ensure you can stop immediately. This will reduce the risk of falling.

Avoid sudden braking to avoid the Rollator slipping.

Seek assistance when using stairs or find an available lift.

The maximum weight capacity is 300-350 lbs and 11 lbs for the mesh bag.

Avoid using the Rollator in heavy storms or on very wet ground. Decelerate when travelling downhill.

CLEANING AND STORAGE

The frame can be wiped with a damp cloth or washed with a neutral cleaning agent.

The wheels should only be cleaned with a plastic-bristled brush. Do not use wire brushes.















Mild soap is suitable to clean the seat back, seat and transporting pouch. Do not use a steam cleaner.

Dry the rollator with a towel.

Applying machine oil to the brake cables and any screws will help to prevent rust.

PRODUCT SPECIFICATION

Size and weight	
Seat size	17,7 x 9,8 in
Seat height	21,2 in
Total width	25,7 in
Total length	27,8 in
Total height/Height of handle (adjustable at 6 levels)	31,89~36,85 in
Empty weight (including belt and mesh bag)	19,1 lb
Max Load	136 kg
Mesh bag weight capacity	11 lb
Folding size	27,8 x 9,8 x 31,89 in
Wheels	
Front wheels	10 x 1,37 in
Back wheels	8 x 1,37 in
Materials	
Frame	Aluminium (corrosion protection)
Seat	Nylon, black
Tire	black polyurethane

	Caution: read instructions (warnings) carefully		Medical Device compliant with Regulation (EU) 2017/745		Product code
	Manufacturer		Non-sterile		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place		Medical Device
	Serial number		Authorized representative in the European community		Date of manufacture
	Lot number		Importer		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.