

ENGLISH
19/04/1/S SHOULDER HARNESS BELT WITH SAFETY FASTENER
20/04/1/S WAIST BELT WITH SAFETY FASTENER
21/04/1/S LAP BELT WITH SAFETY FASTENER
WARNINGS

Before carrying out any operation, visually check the integrity of the device. If the product shows any abnormalities, immediately contact the point of sale.

Erregi S.a.s. shall not be held responsible in case of improper use of the device. In the event of a serious accident involving the use of the device, please report the incident to the manufacturer and to the competent authority of the member state where the user and/or patient lives.

DESCRIPTION

Ref. 19/04/1/S, 20/04/1/S, 21/04/1/S: Postural belts made of class 1 fireproofed water-repellent polyamide fabric, with class 1 IM fireproof of polyurethane foam padding and polypropylene straps with safety fastener.

INDICATION

Comfortable postural belts for patients in wheelchairs, also helpful to prevent falls.

PRECAUTIONS FOR USE

The device must be suitably tightened for it to work properly. When adjusting the straps, make sure that a hand can be inserted between the patient and the device. We recommend regularly monitoring the patient and the area covered by the belt.

Always wear personal protective clothing when using the device.

CONTRAINDICATIONS

No contraindications were found except in cases of ascertained sensitivity to the component materials.

MATERIAL

External fabric: 92% polyamide 8% polyurethane, class 1 fireproofed. Padding: class 1 IM fireproof polyurethane foam.

WASHING INSTRUCTIONS


The manufacturer is not liable for any changes in the technical features of the aid in the event of failure to observe the washing methods.

DISINFECTION







Use medical surgical products.

STORAGE

Store away from sources of light, heat and humidity.

DISPOSAL

Dispose of the device in compliance with the current regulations on environmental protection and recycling.

REF	Product code	MD	Medical Device
LOT	Lot number		Manufacturer
CE	Medical Device compliant with Regulation (EU) 2017/745		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place
	Date of manufacture		Not made with natural rubber latex
UDI	Unique identifier		

DO NOT remove the label from the Medical Device, as it allows tracing the supplied product (Manufacturer, UDI, batch number, product code).

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF 21/04/1/S (GIMA 43175)
20/04/1/S (GIMA 43176)
19/04/1/S (GIMA 43177)

 **Erregi S.a.s.**
Via Adua 40
21040 Vedano Olona (Va)
commerciale@erregiausilimedici.it
www.erregiausilimedici.it
MADE IN ITALY

