

## ENGLISH

### RESTRAINT BED BELT

#### WARNINGS

Before carrying out any operation, visually check the integrity of the device. If the product shows any abnormalities, immediately contact the point of sale. Eregi S.a.s. shall not be held responsible in case of improper use of the device. In the event of a serious accident involving the use of the device, please report the incident to the manufacturer and to the competent authority of the member state where the user and/or patient lives.

#### DESCRIPTION

**Art 24/07:** 100% cotton double restraint belt for beds, composed of two strips, the innermost one must be positioned around the patient's waist and fixed with a fastening button, the outermost one must be fixed to the bed side rails with the fastening straps and buttons. The closing system is with fastening buttons, the magnetic key is used to open the fastening buttons. 1 fastening button is used to block the upper strip around the patient's waist, the other two fastening buttons are used to anchor the straps to the bed side rails. The device is used to restrain an uncooperative patient in need of therapy.

**The patient can be placed both in the supine and lateral position with this belt.**

**Art 24/07/01:** 100% cotton single restraint belt for beds, composed of a strip to be positioned around the patient's waist and secured with a fastening button, plus two straps to be secured to the bed side rails with a hook closure. The closing system is with a fastening button, the magnetic key is used to open the fastening button.

The fastening button is used to lock the upper strip to the patient's waist, the two straps with hook are used to anchor the belt to the bed side rails.

The device is used to restrain an uncooperative patient in need of therapy.

**The belt only allows the patient to be in a supine position.**

#### INDICATION

Devices used for restraining bedridden patients.

#### CONTRAINDICATIONS

No contraindications were found except in cases of ascertained sensitivity to the component materials.

#### PRECAUTIONS FOR USE

The device can only be applied by doctors and paramedics. The belt must not be used on the pelvis or ribcage. **Devices with magnetic closures; do not use on patients who have a pacemaker, as it may cause tachycardia.** Fit the belt on the patient properly; it must be possible to place a hand between the patient and the belt. Always check that the fasteners are securely fastened, otherwise the patient's safety may be compromised. If the fasteners do not open, the belts can be cut using a cutter or shears. Always wear personal pro-

TECTIVE clothing when using the device. Device not fireproof.

#### MATERIAL

100% COTTON.

#### WASHING INSTRUCTIONS



**Attention:** remove keys and fastening buttons before washing, contact with liquids could compromise their functionality.

The manufacturer is not responsible for any changes in the technical characteristics of the device in the event of failure to comply with the washing instructions.

#### DISINFECTION







Use medical surgical products.

#### STORAGE

Store away from sources of light, heat and humidity.

#### DISPOSAL

Dispose of the device in compliance with the current regulations on environmental protection and recycling.

<b>REF</b>	Product code	<b>MD</b>	Medical Device
<b>LOT</b>	Lot number		Manufacturer
<b>CE</b>	Medical Device compliant with Regulation (EU) 2017/745		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place
	Date of manufacture		Not made with natural rubber latex
<b>UDI</b>	Unique identifier		

DO NOT remove the label from the Medical Device, as it allows tracing the supplied product (Manufacturer, UDI, batch number, product code).

#### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

---

**REF** 24/07/01 L-XL (Gima 43185)  
24/07 S-M (Gima 43186)  
24/07 L-XL (Gima 43187)

 **Erregi S.a.s.**  
Via Adua 40  
21040 Vedano Olona (Va)  
commerciale@erregiausilimedici.it  
www.erregiausilimedici.it  
MADE IN ITALY

