


Marks	Designations	A	B	C	D	E	F	G	H	Weight	
3005**	Open Standard seatplate	86 cm	51.5 cm	60.5 cm	72 cm	52 cm	38 cm	39 cm	58 cm	6.6 kg	130 kg
3003**	Open Full seatplate										
3015**	Open Adjustable with standard seatplate	86/97 cm	51.5 cm	60.5 cm	72/83 cm	52/63 cm	38 cm	39 cm	58 cm	6.8 kg	130 kg
3014**	Open Adjustable with full seatplate										
3300**	Club with standard seatplate	91 cm	42 cm	54 cm	67 cm	45 cm	50 cm	44 cm	53 cm	8 kg	130 kg
3000**	Club with full seatplate										
3400**	Confort with standard seatplate	91 cm	42 cm	54 cm	67 cm	45 cm	50 cm	44 cm	53 cm	9 kg	130 kg
3100**	Confort with full seatplate										
3030**	Even with standard seatplate	87.5 cm	51.5 cm	64.5 cm	75 cm	51 cm	38.5 cm	40 cm	42 cm	7.5 kg	130 kg
3031**	Even with full seatplate										
3200**	New club with standard seatplate	84 cm	53 cm	61 cm	73 cm	52 cm	35 cm	40 cm	49 cm	7 kg	130 kg
3201**	New club with full seatplate										
3001**	Club XXL with standard seatplate	92 cm	65 cm	71 cm	67 cm	46.5 cm	46 cm	44 cm	53 cm	10 kg	160 kg
3101**	Confort XXL with standard seatplate									10.5 kg	
3062**	Club XXL with full seatplate	92 cm	55 cm	71 cm	67 cm	46.5 cm	46 cm	44 cm	53 cm	10 kg	160 kg
3162**	Confort XXL with full seatplate									10.5 kg	
3046**	Best Up	94.5 cm	44 cm	47 cm	54 cm	48.5 cm	50 cm	39.5 cm	45 cm	9.4kg	130kg
3646**	Best Up XXL	88 cm	75 cm	87 cm	75 cm	56 cm	59 cm	44 cm	59 cm	9.5kg	160 kg
Dimension of the hole: Width = 21 cm Depth = 25.5 cm											

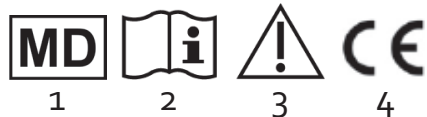

	FR	EN	DE	ES	
	Signification des symboles	Explanation of symbols	Zeichenerklärung	Explicación de los símbolos	
1	Dispositif Medical	Medical Device	Medizinprodukt	Dispositivo médico	
2	Lire le mode d'emploi avant l'utilisation.	Read the instruction manual before use	Vor Gebrauch Gebrauchsanweisung lesen!	Lea el manual de instrucciones antes de utilizarlo	
3	Respecter les instructions de sécurité!	Observe the safety instructions!	Sicherheitshinweise beachten!	Siga las instrucciones de seguridad	
4	Conformité CE	CE conformity	CE-Konformität	Conformidad con la normativa CE	
	PT	PL	HU	TR	
	Significado dos símbolos	Objaśnienia symboli	Szimbólumok jelentése	Sembollerin açıklamaları	
1	Dispositivo médico	Wyrób medyczny	Orvosi eszköz	Tıbbi cihaz	
2	Antes de utilizar, leia, por favor, o manual.	Przed użyciem zapoznaj się z instrukcją obsługi	Használat előtt olvassa el a használati útmutatót	Kullanmadan önce kullanma kılavuzunu okuyunuz	
3	Siga as instruções de segurança	Stosuj się do instrukcji bezpieczeństwa.	Tartsa be a biztonsági utasításokat!	Güvenlik uyarılarını dikkate alınız	
4	Certificado CE	Deklaracja CE	CE megfeleléség	CE uygunluğu	
	IT	NL	SI	HR	GR
	Significato dei simboli	Verklaring van de symbolen	Pomen simbolov	Objašnjenje simbola	Επεξήγηση Συμβόλων
1	Dispositivo Medico	Medisch hulpmiddel	Medicinski pripomoček	Medicinski uređaj	Ιατρική Συσκευή
2	Prima dell'uso leggere le relative istruzioni!	Voor gebruik de handleiding lezen.	Pred uporabo preberite navodila	Pročitati korisničke upute prije uporabe	Διαβάστε τις οδηγίες πριν από τη χρήση
3	Osservare le istruzioni per la sicurezza	Veiligheidsinstructies respecteren	Upoštevajte varnostna navodila!	Pažljivo pogledajte sigurnosne upute	Τηρείτε τις οδηγίες ασφαλείας!
4	CE conformità	CE conformiteit	CE Certifikat	CE-Deklaracija	CE Συμμόρφωση

EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Before seating make sure that you have correctly placed the seatplate on the chair.
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips)..
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a commode chair and in proper way : don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- ♥ DO NOT hang anything on the Chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
- ♥ Be careful when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before using, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to seat on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device

shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, Indication and Contraindication

Commode chairs can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom. Standard, foldable or on castors, they shall be use only indoor.

Indication

Used in the home when the patient is too debilitated to reach the bathroom.

Contraindication

Don't use the commode chair in case of:

- Cognitive disorder - Loss of balance - Siting impairment

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Mounting the bucket

There are 2 different ways to install/remove the bucket :

- 1- from the left or right side under the seat, insert the pan into the metallic support (single possibility for XXL models).
- 2- lift up the seatplate and remove the bucket from the top, see photo [B]. Close the bucket with the lid before transporting it.

Mounting the seatplate

The seatplate can be easily removed for cleaning or for replacement as it is only connected to the frame with 2 clips see photo [A].

To remove the seatplate, take out the clips from the frame by lifting them up, see photo [B].

To mount the seatplate, clip the two clips on the back pipe of the metal frame, see photo [C].

By adding the cover on the hole, the commode chair can be used as a regular chair.

Rental: This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

- 1- Visual inspection: ensure that the product has not been damaged and check for any defect.
- 2- Functional inspection: ensure that the product functions properly and repair it if needed.
- 3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories).

Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.

Use only appropriate tools to perform the maintenance, cleaning and disinfection. The user's manual is an integral part of the product and must be handed over to every new user.

4. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

5. Materials

The product is made up of steel tubing covered with epoxy painting. Upholsteries are in PVC. The chair is delivered with a pan. Some models are equipped with swinging armrests.

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

See label page 1

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Duration of use: 2 years.