**SEDIA COMODA E DOCCIA  
COMMUNE AND SHOWER CHAIR  
CHAISE D'AISANCE ET DE DOUCHE  
SILLA WC Y DUCHA  
CÓMODA E CADEIRA DE DUCHE  
KOMMODE UND DUSCHSTUHL  
KOMMOD OCH DUSCHSTOL  
ΚΟΜΟΔΑ ΚΑΙ ΚΑΡΕΚΛΑ ΝΤΟΥΣ**

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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**REF** HE10635081 (Gima 43193)

**HEBEI HEALTHPLUS MEDICAL DEVICE CO.,LTD**  
NO.1,Chuangye Street,Matou Industry Park,HanDan City,HeBei  
Province,China  
Made in China

**Y.Sung Handelsvertretung**  
Dusselthaler Str. 24, 40211  
Dusseldorf Germany

**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

HE10678103

# ALUMINUM LIFTING COMMODE CHAIR

Parts of the product

- 1. Backrest
- 2. Armrest
- 3. PU seat
- 4. Leg tube
- 5. Tip

## Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up - risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate

## Assembly

### Step 1

**Attach backrest tube to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located under the seat.**

**Step 2**

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs.

Make sure that push button “snap” correctly through holes and are stable and adjusted the same way.

**Technical data**

<b>Product</b>	ALUMINUM LIFTING COMMODOE CHAIR
<b>Product number</b>	HE10635081
<b>Weight capacity (lbs/kg)</b>	300 lbs/ 136kg
<b>Product dimension (mm) L x W x H</b>	(521-558)×(692-730)×(850-955)mm
<b>Product weight (lbs/kg)</b>	7.1kg
<b>Backrest material</b>	PU
<b>Seat material</b>	PU
<b>Arm pad material</b>	EVA Foam
<b>Frame material</b>	Aluminum
<b>Frame color</b>	Sliver
<b>Quantity in box (piece)</b>	1
<b>Product weight including box (lbs/kg)</b>	8.0kg
<b>Carton dimension L x W x H</b>	645*585*395mm

**Symbols**

	Caution: read instructions (warnings) carefully		Keep in a cool, dry place
	Manufacturer		Date of manufacture
	Lot number		Product code
	Medical Device compliant with Regulation (EU) 2017/745		Authorized representative in the European community
	Medical Device		Consult instructions for use
	Serial number		Imported by

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.