

**GIMA 43196 - 43198**



Zhongshan Kangdebao(KDB) Rehabilitation Equipment Co.,Ltd.  
Card 1, No.5, 24# Longcheng Road, Dongsheng Town, Zhongshan City, Guangdong Province, China  
Made in China



**KDB-697**  
**KDB-698**



SUNGO Europe B.V.  
Fascinatio Boulevard 522, Unit 1.7,  
2909VA Capelle aan den IJssel, The Netherlands



**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)



**ENGLISH**

**NAME:**

**Commode Chair**

**All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.**

**INTENDED USE:**

The Commode Chair is used for excreting of people who are experiencing illness, injury or disability, it can be used at home or hospital.

**OPERATOR**

The device can be operated by the user self, the nursing staff or the caregiver.

**PATIENT POPULATIONS**

People with disabilities, patients and the elderly.

**CONTRAINDICATIONS**

Supervise children, when they use the Commode chair.

**INSTRUCTION FOR USE**

1. Lift the seat plate, place the bucket properly on the bucket frame, then put the seat plate down for use.
2. After use, close the bucket lid, remove it from the bucket frame.
3. Maximum supported weight of user is 125 kg.

**MAINTENANCE**

1. The surface of the product should not be scratched or scratched by hard objects to prevent material damage.
2. For daily cleaning and maintenance, apply neutral synthetic detergent, wipe with a damp cloth, and then wipe with a dry cloth.
3. Routinely clean and disinfect once a day by medical staff or users, and thoroughly disinfect the commode when changing patients.
4. Do not contact with strong acids and alkalis to prevent corrosion.

**PRECAUTIONS AND WARNINGS:**

1. Please place the commode chair in a dry and ventilated place away from direct sunlight, and avoid corrosive chemicals such as oil, acid, and alkali.
2. The commode chair should avoid exposure to the sun and rain, and the commode chair should be wiped dry immediately after each use.
3. Some high-quality lubricants can be added to the movable parts for lubrication, which is better for use.
4. Regularly check the connection of the commode chair to prevent accidents caused by loosening and falling off.
5. Frequently check whether there are cracks everywhere in the commode chair and other defects that will affect the use of the chair.
6. Frequently check whether the seat frame and the seat plate are firm.
7. The product should be placed steadily and not shaken strongly to prevent structural damage.
8. Do not stand on the chair with dismantled feet, otherwise the product may fall over.

**STORAGE**

The packed wheelchair should be stored in a ventilated and dry place, no corrosive gas and well ventilated and clean room. To avoid damp, strong light and keep away from heat sources during transportation.

**WARRANTY**





The warranty period of this product will start from your purchase day. Accessories 90 days. Main frame 365 days.

**DISPOSAL**

Disposal restrictions depend upon the contamination incurred during use and are subject to national or local legislation.

**LABELS, PACKING LOGO DESIGN:**

| Symbol | Introductions  | Symbol | Introductions                                       |
|--------|--|--------|---|
|        | Medical Device compliant with Regulation (EU) 2017/745 |        | Medical Device                                      |
|        | Manufacturer   |        | Authorized representative in the European community |
|        | Date of manufacture                                    |        | Keep away from sunlight                             |
|        | Lot number   |        | Non sterile   |
|        | Caution: read instructions (warnings) carefully        |        | Product code  |

|  |                              |   |                           |
|--|------------------------------|---|---------------------------|
|   | Consult instructions for use |   | Keep in a cool, dry place |
|  | Imported by                  |  | Unique identifier         |

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies