



REF KDB-699 (GIMA 43202) Zhongshan Kangdebao(KDB) Rehabilitation Equipment Co.,Ltd.
Card 1, No.5, 24# Longcheng Road, Dongsheng Town, Zhongshan City, Guangdong Province, China
Made in China SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA
Capelle aan den IJssel, The Netherlands **Gima S.p.A.**
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

ENGLISH

NAME**Commode Chair****Model:** KDB-699**Weight capacity:** 125 Kg.**INTENDED USE:**

The Commode Chair is used for excreting of people who are experiencing illness, injury or disability, it can be used at home or hospital.

OPERATOR

The device can be operated by the user self, the nursing staff or the caregiver.

PATIENT POPULATIONS

People with disabilities, patients and the elderly.

CONTRAINDICATIONS

Supervise children, when they use the Commode chair.

INSTRUCTION FOR USE

1. This product is foldable structure, you just need to unfold the frame and install the necessary parts such as wheels, fix the bucket and seat panel before use.
2. Lift the seat plate, place the bucket properly on the bucket frame, then put the seat plate down for use.
3. After use, close the bucket lid, remove it from the bucket frame.
4. The commode chair can be folded up when not in use, which is convenient for storage and takes up less space.

MAINTENANCE

1. The surface of the product should not be scratched or scratched by hard objects to prevent material damage.
2. For daily cleaning and maintenance, apply neutral synthetic detergent, wipe with a damp cloth, and then wipe with a dry cloth.
3. Routinely clean and disinfect once a day by medical staff or users, and thoroughly disinfect the commode when changing patients.
4. Do not contact with strong acids and alkalis to prevent corrosion.

PRECAUTIONS AND WARNINGS

1. Please place the commode chair in a dry and ventilated place away from direct sunlight, and avoid corrosive chemicals such as oil, acid, and alkali.
2. The commode chair should avoid exposure to the sun and rain, and the commode chair should be wiped dry immediately after each use.
3. Some high-quality lubricants can be added to the movable parts for lubrication, which is better for use.
4. Regularly check the connection of the commode chair to prevent accidents caused by loosening and falling off.
5. Frequently check whether there are cracks everywhere in the commode chair and other defects that will affect the use of the chair.
6. Frequently check whether the seat frame and the seat plate are firm.
7. The product should be placed steadily and not shaken strongly to prevent structural damage. Commode chair with wheels should NOT be used as a manual wheelchair, only for short-distance mobility.
8. Do not stand on the chair with dismantled feet, otherwise the product may fall over.

STORAGE














The packed commode chair should be stored in a ventilated and dry place, no corrosive gas and well ventilated and clean room. To avoid damp, strong light and keep away from heat sources during transportation.

DISPOSAL

Disposal restrictions depend upon the contamination incurred during use and are subject to national or local legislation.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

LABELS, PACKAGING LOGO DESIGN:

Symbol	Meaning	Symbol	Meaning
	Medical device compliant with Regulation (EU) 2017/745		Medical device
	Manufacturer		Authorized representative in the European Community
	Manufacturing date		Store away from sunlight
	Batch number		Not sterile
	Warning: Read and follow the instructions (warnings) for use carefully		Product code
	Read the instructions for use		Store in a cool, dry place
	Imported from		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.