


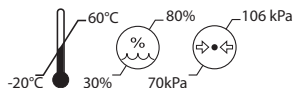
GIMA 43430

 Zhongshan Kangdebao(KDB) Rehabilitation Equipment Co.,Ltd.
Card 1, No.5, 24# Longcheng Road, Dongsheng Town, Zhongshan City, Guangdong Province, China
Made in China

REF KDB-506

EC REP SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA
Capelle aan den IJssel, The Netherlands

 **Gima S.p.A.**
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



ENGLISH

PRODUCT NAME

Lift and Transfer Device

Model Number: KDB-506

Weight capacity: 125 Kg.

INTENDED USE:

The lift and transfer device is for transferring the lower body paralysis, leg and foot inconvenience patients or the elderly from bed, wheelchair, seat, or toilet seat. This device greatly relieved caregivers' work. It improves the efficiency of nursing work, and reduces the nursing risk.

INDICATIONS

Only used for human.

CLEANING METHODS/MAINTENANCE

When necessary, clean the device with warm water or rubbing alcohol and ensure that the castors are free of dirt and hair. Do not use cleaning agents containing phenol or chlorine, as this may damage aluminum and polyamide materials.

In order to avoid possible formation of rusting, pay attention of following points for better maintenance and to improve its service life:

*Removing corrosive substances such as the shower gel, shampoo, etc every time after use.

*Then push the product to a ventilated dry place, this is also good habit every time after use.

PRECAUTION AND WARNING

Please read and follow the safety precautions listed below, which could make the operation simple and trouble-free.

Read and understand this instruction before using.

WARNING - Important safety information for hazards that might cause serious injury.

CAUTION - Information for preventing damage to the product.

NOTE - Information to which you should pay special attention.

WARNING

*ALWAYS plan your lifting operations before commencing.

*ALWAYS carry out the DAILY CHECK LIST before using the device.

*ALWAYS familiarize yourself with the controls and safety features of the device before lifting the patient.

*DO NOT use a damaged device.

*ALWAYS install the device according to the instruction.

- *ALWAYS check that the device's safe workload is appropriate for the patient's weight
- *ALWAYS follow the instructions of the user's manual to operate the device.
- *ALWAYS lift the patient with the casters locked. Be sure to align the shifter with the correct center of gravity.
- *DO NOT try to move the device by pushing against the atuator, hanger or patient.
- *ALWAYS move the device with the push handle.
- *ALWAYS keep the patient in a minimum comfortable position before transfer.
- *DO NOT move the loaded device over the speed of 3km/h 0.8m/s.
- *DO NOT move on uneven or rough ground. Especially if it was loaded with a patient.
- *DO NOT impose safety controls. All the controls are easy to use and do not require excessive force.
- *DO NOT park the loaded device on any slopes.
- *DO NOT lift the patient unless you are trained and able to do so.

TRANSPORTATION AND STORAGE

The transport and storage environment of the device shall be as follows

- a) Temperature range : -20°C~+60°C.
- b) The relative humidity ranges from : 30% ~ 80%.
- c) Atmospheric pressure ranges from 70kPa to 106kPa.

PACKAGE:


















The package is one set in 1 carton.

DISPOSAL

Disposal restrictions depend upon the contamination incurred during use and are subject to national or local legislation.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

LABELS, PACKAGING LOGO DESIGN:

Symbol	Meaning	Symbol	Significato
	Medical device compliant with Regulation (EU) 2017/745		Medical device
	Manufacturer		Authorized representative in the European Community
	Manufacturing date		Store away from sunlight
	Batch number		Not sterile
	Warning: Read and follow the instructions (warnings) for use carefully		Product code
	Read the instructions for use		Store in a cool, dry place
	Imported from		Temperature limit
	Humidity limit		Atmospheric pressure limit
	Unique identifier		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.