




# GIMA

PROFESSIONAL MEDICAL PRODUCTS

## LETTINO PER BAMBINI CHILDREN BED LIT D'ENFANT CAMA INFANTIL

MANUALE D'USO – ISTRUZIONI E MONTAGGIO  
USER MANUAL – ASSEMBLY INSTRUCTIONS  
MODE D'EMPLOI – INSTRUCTIONS ET MONTAGE  
MANUAL DE USO – INSTRUCCIONES Y MONTAJE

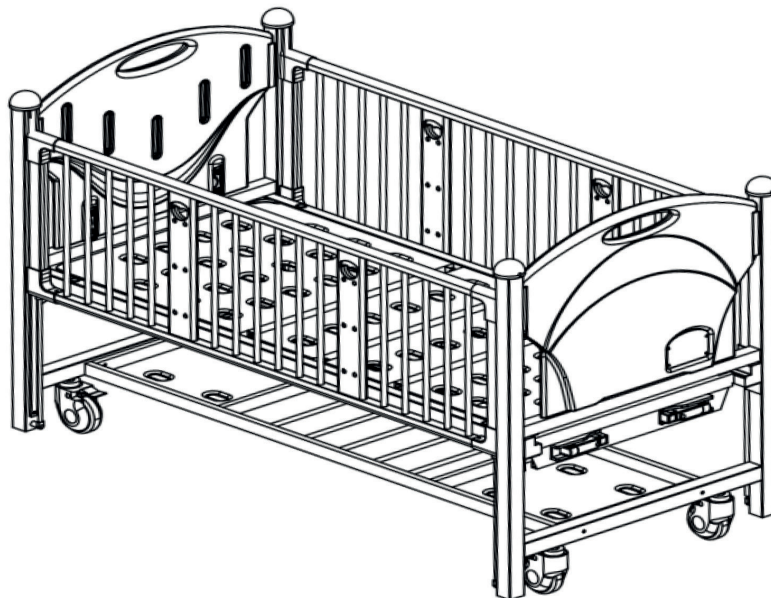

- È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.
- All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.
- Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.
- Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

### Gima 43506 - 43507



**Jiangsu Saikang Medical Equipment Co.,Ltd**  
No. 35 Lehong Road, Modern Agriculture Demonstration  
Park, Zhangjiagang City, Jiangsu Province, China  
Made in China




**CX2X**



Sungo Europe B.V.  
Fascinatio Boulevard 522,  
Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands



**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
www.gimaitaly.com

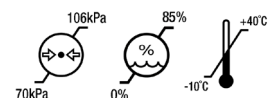

---

## ***MANUAL INTRODUCTION***

The manual children bed is a bed specially designed for hospitalized children patients or others in need of some form of health care.

**Note:**


**Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.**

- This manual is applicable to CX2x Children bed. It contains the installation method, operating instructions and maintenance check points.
- The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- This user manual includes the description, instruction, maintenance and technical parameters of the product.
- Any questions, please contact the after-sale service center or distributor. Anyone using or operating this product should read and comply with it.
- Except for some unexpected situations, users are required to be familiar with the usage and precautions.

**TECHNICAL SPECIFICATION**

- Model: CX2X
- Product Weight:90KG
- Material:Powder coated steel structure with PP head and foot board
- Intended user:Children patients
- Frequently used functions:  
back-rest lefting and Knee-rest lefting
- Clinical benefits:  
comfortable feeling for children patients, Simple operation functions and quality assurance.

**INSTALLATION INSTRUCTION**

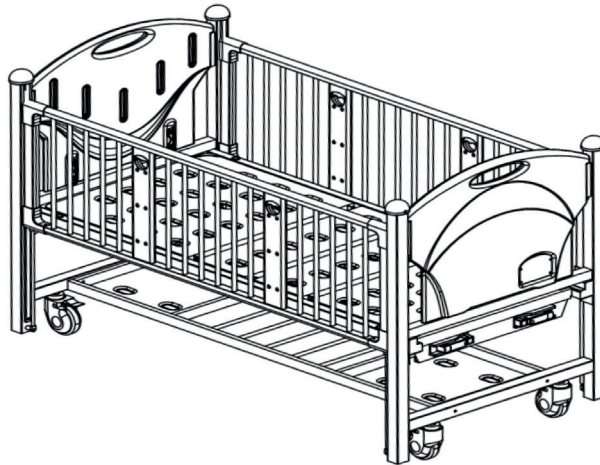

|              |             |    |           |
|--------------|-------------|----|-----------|
| Children Bed | Length      | mm | 1880±10mm |
|              | Width       |    | 900±10mm  |
|              | Height      |    | 1260±10mm |
| Tilt range   | Back        | /  | 0-75±5    |
| Tilt range   | Leg         | /  | 0-35±5°   |
| Weight       | Safety load | Kg | 170       |

This bed is not suitable for patients weighing less than 10 kg. The maximum recommended patient weight is 145 kg.

The safe working load is calculated as follows :



Maximum patient weight: 145kg

Mattress: 5kg

Accessories (including attached loads): 20kg

TOTAL: 170kg

|   |  |
|---|--|
| <p><b>1</b></p>   | <p><b>2</b></p>  |
| <p><b>Bed frame installation:</b> Put the bed surface on the support frame of the headboard and the footboard, and fix it with M8*20 outer hexagon bolts (8 in total)</p> | <p><b>Chassis installation:</b> The hole pointed out by arrow is fixedly connected with M8*85 outer hexagon bolts and M8 locking nuts (4 in total)</p> |
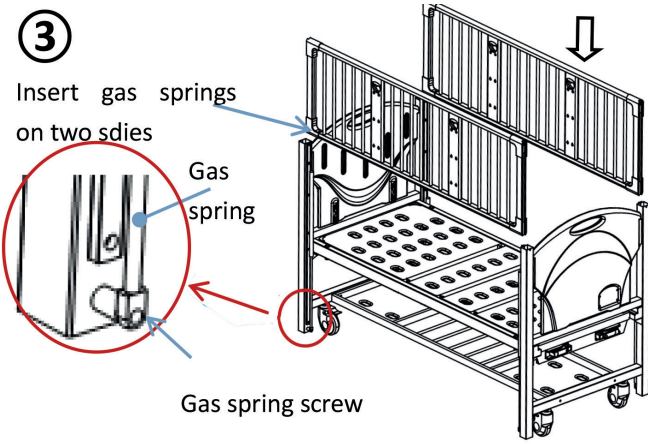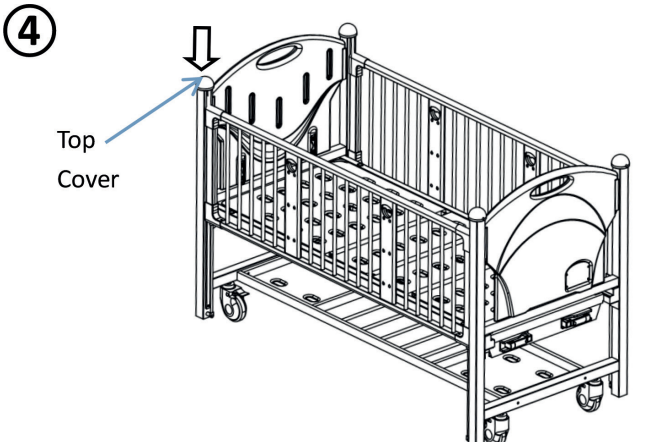
|  |  |
|--|--|
| <p><b>③</b></p> <p>Insert gas springs on two sides</p>  <p>Gas spring</p> <p>Gas spring screw</p>   | <p><b>④</b></p>  <p>Top Cover</p>              |
| <p><b>Gas spring and guardrail installation:</b></p> <ol style="list-style-type: none"> <li>1. Fix the gas spring to the bed frame as shown in the figure, and fix it with 4 M10*40 outer hexagon bolts</li> <li>2. Insert both ends of the guardrail into the gas spring, and clip it to the bed frame from top to bottom.</li> </ol> | <p><b>Top cover installation:</b></p> <ol style="list-style-type: none"> <li>1. Install the top cover on four corners</li> </ol> |
| <p><b>Note:</b> When assembling step 3, the gas spring screw cannot be locked, and a gap of <math>5\pm 1</math>MM should be reserved so that the gas spring can adjust the position by itself, see Figure 2</p>  |  |

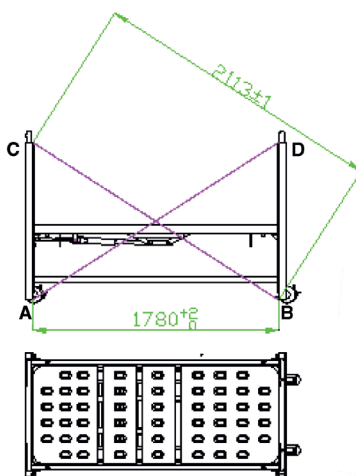


Figure 1

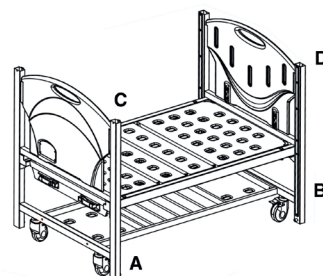


Figure 2

## FAULTS EXCLUSION

| Common failures                       | Probable reasons  | Excluding methods   |
|---------------------------------------|---|---|
| Difficult to move the whole baby crib | <ol style="list-style-type: none"> <li>1. The caster is locked without releasing the brake</li> <li>2. The caster is damaged</li> </ol> | <ol style="list-style-type: none"> <li>1. release the brake</li> <li>2. replace the caster</li> </ol> |

## STAINLESS STEEL BABY CRIB-MAINTENANCE

1. In order to use the baby crib safely, it is necessary to perform regular safety inspections on the baby crib. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
2. When the baby crib frame ages and it reaches a certain servicing time, which the metal part of the baby crib and plastic parts of the baby crib frame can be recycle.
3. Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
4. If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the baby crib, which will cause the stainless steel surface to rust.
5. Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
6. It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
7. The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
8. Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels



and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.

9. For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a baby crib equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the baby crib also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
10. If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

**PACKAGING, TRANSPORT, STORAGE**

1. The packaging of baby crib is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of the baby crib.
3. The stainless steel baby crib should be stored:
  - a) Ambient temperature: -10°C ~ +40°C.
  - b) Relative humidity: ≤ 85%.
  - c) Atmospheric pressure: 70 kPa ~ 106 kPa.



Explanation of symbols:



**Note! Check the file that comes with the bed**





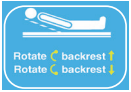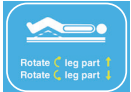








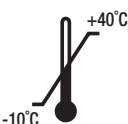





**Don't use the device in direct contact with the skin**

| Symbol picture   | Symbol description                             | Symbol picture   | Symbol description                             |
|--|--|--|--|
|  | Rotate this handle to control back plate angle |  | Rotate this handle to control knee plate angle |

**AFTER-SALES SERVICE**

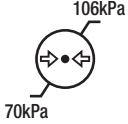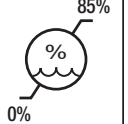1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these materials when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. Do not disassemble the internal device of this product by yourself to avoid unnecessary damage. If you find that the use is affected due to quality problems, please contact our after-sales service department or our authorized dealer.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

|   |   |   |  |   |                           |
|---|---|---|--|---|---------------------------|
|  | Keep away from sunlight                             |  | Medical Device compliant with Regulation (EU) 2017/745 |  | Keep in a cool, dry place |
|  | Product code  |  | Consult instructions for use                           |  | Manufacturer              |
|  | Data di fabbricazione                               |  | Lot number   |  | Medical Device            |
|  | Caution: read instructions (warnings) carefully     |  | Atmospheric pressure limit                             |  | Humidity limit            |
|   | Authorized representative in the European community |  | Temperature limit                                      |  | Imported by               |
| <b>GIMA WARRANTY TERMS</b><br>The Gima 12-month standard B2B warranty applies.      |   |  | Unique identifier                                      |  | Serial number             |