



GIMA

PROFESSIONAL MEDICAL PRODUCTS

SEDIA PER MASSAGGI PIEGHEVOLE
FOLDABLE MASSAGE CHAIR
CHAISE DE MASSAGE PLIANTE
FALTBARER MASSAGESTUHL
SILLA DE MASAJE PLEGABLE
CADEIRA DE MASSAGEM DOBRÁVEL

GIMA 44050

Fabbricante/Manufacturer/Fabricant/Fabricante:

Anji Sukar Furniture Co., Ltd.

Tangpu Industrial Zone, Dipu Town, Anji, Zhejiang

Province, China. P.C.: 313300

peter@zjruixiang.com

REF MC006

Importato da/imported by/importé par/importado por

GIMA S.p.A

Via Marconi, 1 – 20060

Gessate (MI) – Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Made in China



PRECAUTIONARY INSTRUCTIONS

1. Carry out periodic maintenance on the chair. Make sure all screws are secure and that the cable is not deteriorated or frayed. Make sure there are no cracks in the metal structure.
2. Do not overload the chair beyond the recommended weight limit. Doing so could result in serious injury to the user. This product is tested to hold a maximum working load of 120 kgs.
3. Do not use alcohol-based cleaners when cleaning the vinyl. They could cause damage to the vinyl material. Use warm soapy water to clean, then wipe dry.
4. When using the chair, make sure that all four legs are at the same level. Using the legs at different levels can cause the chair to be unstable and could lead to injury.
5. Do not allow children to assemble or disassemble the massage chair, unless under your supervision.
6. Be careful when using sharp objects, as they can pierce the covering. Keep the chair's covering away from sharp objects.



ASSEMBLY INSTRUCTION

Steps are as below:



a



b



c




d



e




f

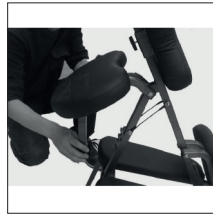


g




h

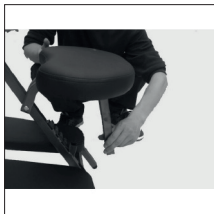

i

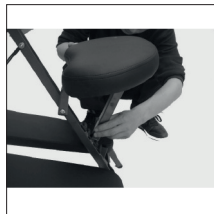

j

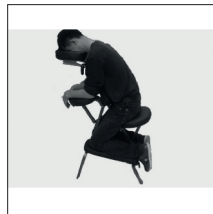

k

1. Remove the massage chair from the cardboard box as shown in picture a.
2. Open the four legs of the chair and make sure the legs stand on the ground at the same level, as shown in pictures b and c.
3. Adjust the headrest cradle to the direction which will make the user feel comfortable, as shown in picture d.
4. Adjust the position of the chest rest and then secure the position using the black knob behind the chest rest, steps as shown in pictures e and f.
5. Adjust the seat pad to a comfortable position using the seat support as shown in pictures g and h.
6. Adjust the hand rest to a comfortable position using the hand rest support as shown in pictures i and j.
7. Now the portable massage chair is ready for use.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.