



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.


Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

ECONOMIC EXAMINATION BEDS

User manual

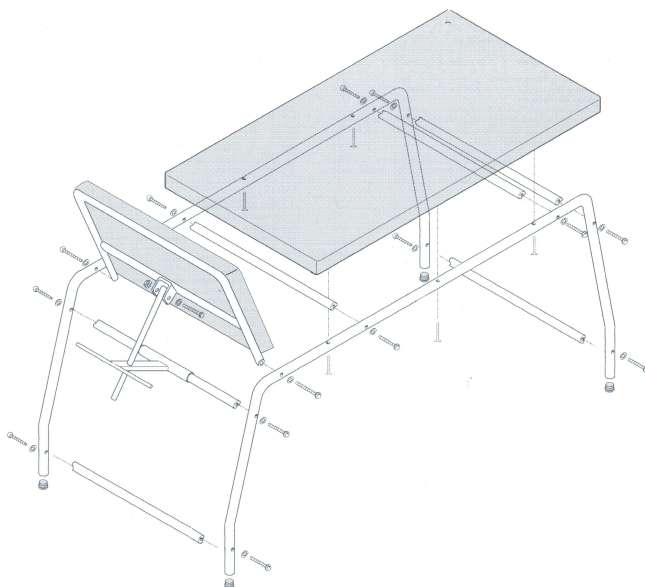

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

REF 44450 - 44451 - 44455 - 44456



Gima S.p.A.
Gessate (MI) - Italy
Made in Italy



CE

- 44450** - Economic black painted examination bed
- 44451** - Economic white painted examination bed
- 44455** - Economic blue chrome examination bed
- 44456** - Economic beige chrome examination bed

INTRODUCTION

The economic examination bed was designed for use in hospitals, community facilities and nursing homes; it is particularly suitable for medical centre/clinic use.

Pursuant to Directive 2007/47/EC, the bed falls within the CLASS I MEDICAL DEVICES.

DESCRIPTION OF THE PRODUCT

The bed has small dimensions; the mattress, washable and flame-retardant, consists of two parts, a first part that is fixed and a second part that can be reclined into a variety of positions.


Components

- No. 2 legs
- No. 1 cross-piece with raise backrest
- No. 5 cross-pieces
- Backrest padding
- Seat padding

INSTRUCTIONS

The bed is supplied completely disassembled

1. Take the 2 legs and start by securing the cross-pieces, including the cross-piece with with raise backrest, using bolts.
2. Lastly attach the padding to the frame using screws.

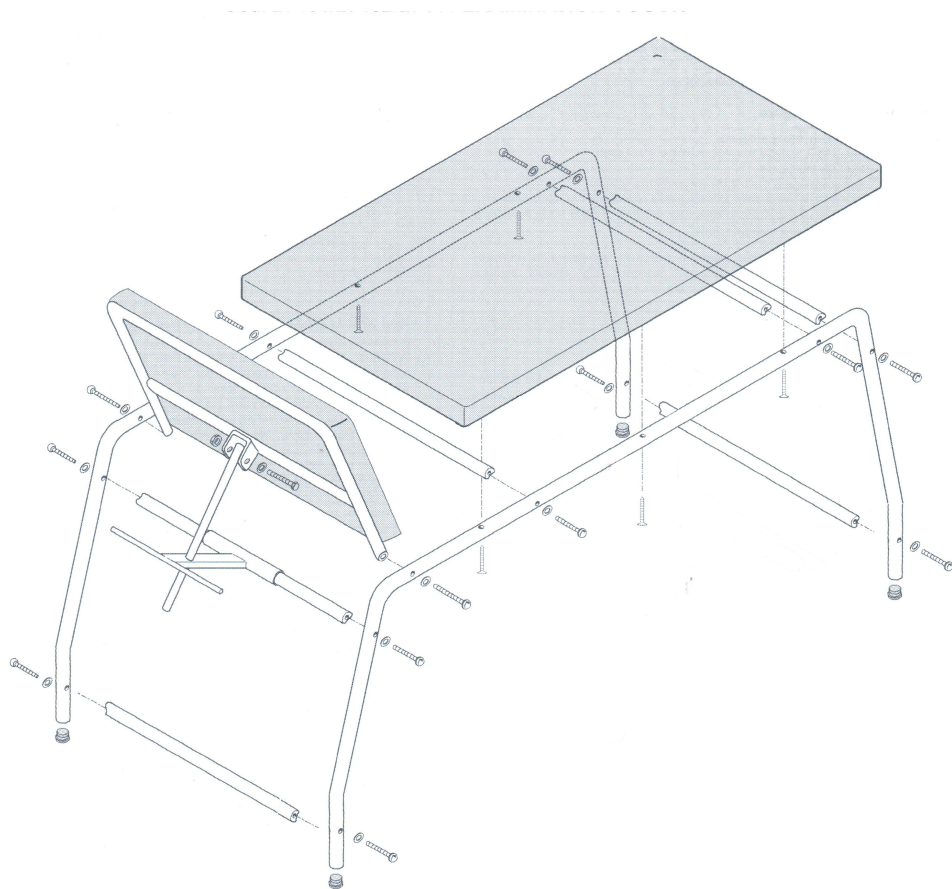

Technical datasheet

2016	44450 – ECONOMIC EXAMINATION COUCH BLACK COLOR 44451 – ECONOMIC EXAMINATION COUCH WHITE COLOR 44455 – ECONOMIC EXAMINATION COUCH BLUE COLOR 44456 – ECONOMIC EXAMINATION COUCH BEIGE COLOR	
GENERAL DESCRIPTION OF PRODUCT	Examination couch designed for medical office / clinic. The mattress is composed of two parts, a first fixed part, a second part of reclining positions.	
DESCRIPTION OF INDIVIDUAL PARTS	Framework	- Framework steel tubular got from cold band dimensions 30x1.5. - Frame chromium-plated or white painted ral 9010
	Upholstery	- Dimension 61x50, 61x130. - Upholstery made with wood panel mm. h14 and foam rubber mm. h40. - Simulated leather LEDERFLEX GAZEBO/AE (composition 87% PVC, 13% CO), washable, CLASS 1, - Available colors: Black 991, White 9010, Blue 4915, Beige 151
	Measurement	cm.180 x 61 x h75
	Weight	29 Kg
	Max Capacity	90/100 Kg
	Cleaning Rules (Coating)	Clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches, synthetic detergents, polishes or aerosol spray.
	Note	Colours and characteristics can be changed for improvements and/or alterations

WARNING ON USE

- Comply with the maximum weights indicated in the technical datasheets.
- When adjusting the height of the backrest ensure that no-one is able to introduce their hands, legs or anything else into the movement area: risk of crushing.
- Avoid direct contact with open wounds.
- Lay the patient down only when the bed frame is completely horizontal on all sides.
- Move the bed without lifting it off the ground.
- If the bed has wheels, make sure the brakes are released before moving it.
- Do not use outdoors.

CLEANING AND MAINTENANCE

- To ensure its efficient, long-term use, it is advisable to avoid prolonged exposure to the sun rays, contact with saline environments and storage in very humid environments.

- Check that the nuts locking the legs are tightened properly.
- Periodically check all fixing points.
- Clean the mattress with a damp and soapy cloth and rinse well with clean water.
- Do not use solvents, bleaches and chemical cleaners or sprays for polishing.
- Clean the bed without using abrasive products. Dry with care.
- If disinfection is required, only use suitable non-corrosive products.
- Only use original spare parts.

WARRANTY

- The bed comes with a two-year warranty for material and manufacturing defects, but does not include consumables.
- Tampering or damage caused by incorrect use of the bed will invalidate the warranty immediately.
- This medical device has been created exclusively for the above mentioned uses. Any other use is deemed improper.
- The manufacturer declines all liability for injury to persons and property due to improper use of the device.