




GIMA

PROFESSIONAL MEDICAL PRODUCTS

EXAMINATION TABLE
LETTINO DA VISITA
DIVAN D'EXAMEN
CAMILLA DE EXPLORACIÓN

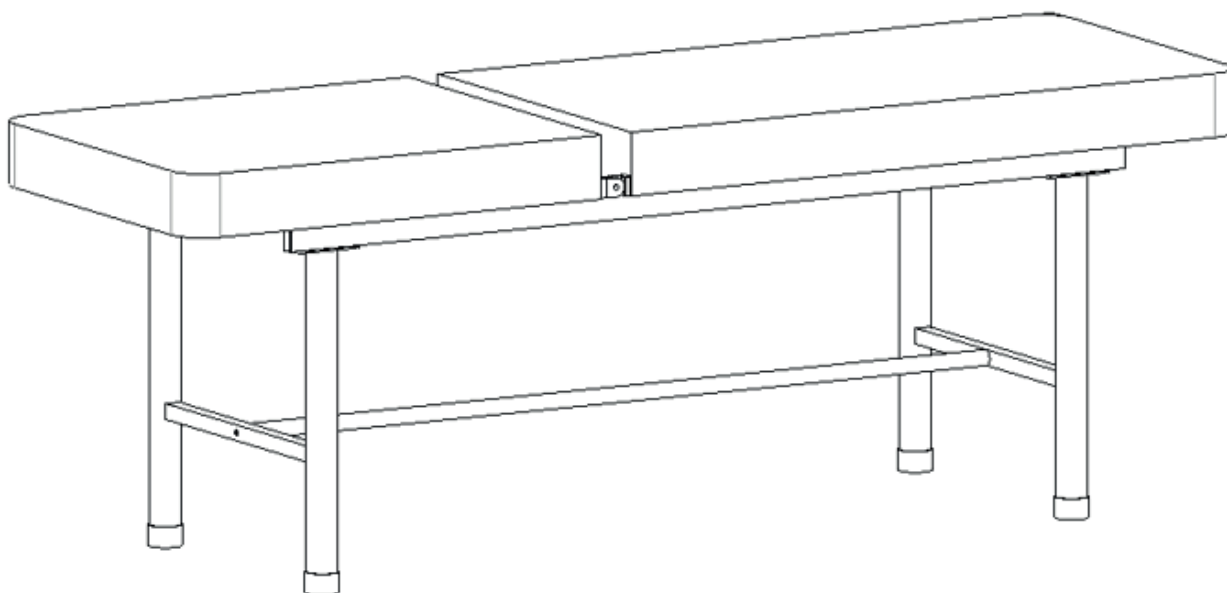

Manuale d'uso - User manual -
Manuel d'utilisation - Introducción del manual

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

ATTENTION : Les opérateurs doivent lire attentivement et comprendre complètement le présent manuel avant d'utiliser le produit.

ATENCIÓN: Los operadores deben leer atentamente y comprender completamente el presente manual antes de utilizar el producto.

Gima 44460



Jiangsu Saikang Medical Equipment Co.,Ltd
No. 35 Lehong Road, Modern Agriculture Demonstration
Park, Zhangjiagang City, Jiangsu Province, China
Made in China

REF X09



SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelleaan den IJssel, The Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



Manual introduction

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.

- This manual is applicable to X09 stainless steel examination table. It contains the installation method, operating instructions and maintenance check points.
- The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- This user manual includes the description, instruction, maintenance and technical parameters of the product.
- Any questions, please contact the after-sale service center or distributor.
- Anyone using or operating this product should read and comply with it.
- Except for some unexpected situations, users are required to be familiar with the usage and precautions.

INTENDED USE

A medical examinations table is integral to patient comfort while suffering from injury, trauma or shock

1. Installing guide

1.1 List of standard accessories

After opening the packing case, please check whether the following parts are complete in the packing case.

Table a:




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

No.	Name	Picture	Specification	QTY	Unit
1	Couch plate		/	1	pc
2	Foot structure		539*543mm	2	pcs
3	Connecting pipe		Φ25*1.2*1323-M8mm	1	pc
4	Screw	/	Refer to Table a.1	1	set

Table a. 1:


No.	Name	Picture	Specification	QTY	Unit
1	Outer hexagon screw		M10*25mm	8	pcs
2	Flat washer		Φ10*Φ21*H1.8mm	8	pcs
3	Spring washer		Φ10*Φ17*H3mm	8	pcs
4	Lock nut		M10mm	8	pcs
5	Outer hexagon screw		M8*40mm	2	pcs
6	Flat washer		Φ8*Φ16*H1.2mm	2	pcs
7	Spring washer		Φ8*Φ14*H2.5mm	2	pcs

1.2 Foot structure installation

Lay a protective pad and put the main structure upside down;

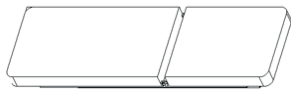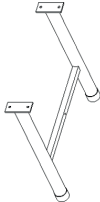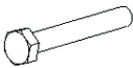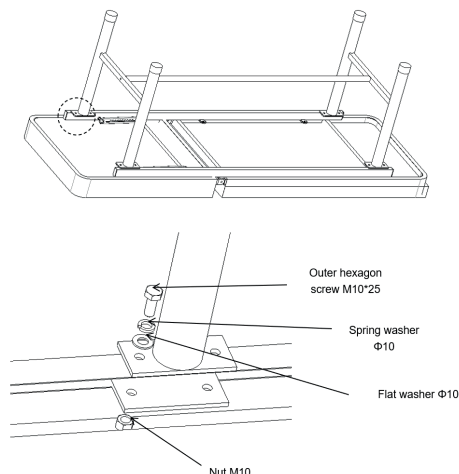

2. Use method

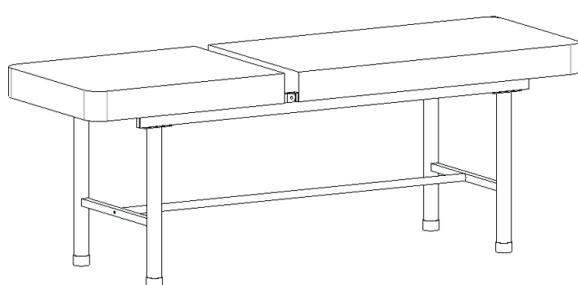

Dimensions	Length	mm	1900±10
	Width		600±10
	Height		680±10
Weight	Safe working load	kg	170

3. Stainless steel examination table-maintenance

1. In order to use the examination table safely, it is necessary to perform regular safety inspections on the examination table. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
2. When the examination table frame ages and it reaches a certain servicing time, which the metal parts and plastic parts of the examination table can be recycle.
3. Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
4. If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the examination table, which will cause the stainless steel surface to rust.

4. Packaging, transport, storage



1. The packaging of examination table is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of the examination table.
3. The examination table should be stored:
 - a) Ambient temperature: -40°C ~ +70°C.
 - b) Relative humidity: ≤ 95%.
 - c) Atmospheric pressure: 500hPa ~ 1060hPa.

5. After-sales service


1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification, the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

Notice!: Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.


1.3 Connection pipe installation

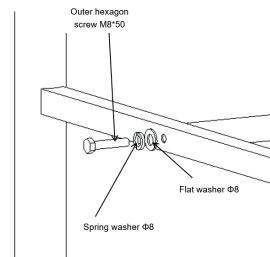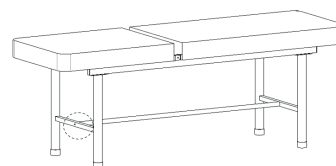

2.1 Backrest lifting function

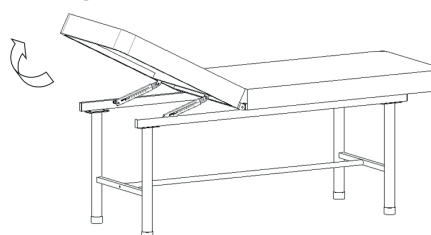

Lift the back plate to achieve the backrest lifting function, And lift the plate to the highest position, then can put down the back plate.

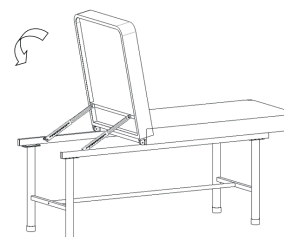

ENGLISH

Symbols	
Symbols	Meaning
	Manufacturer
	Authorized Representative in the European community
	Disposal of Electrical & Electronic Equipment (WEEE): Do not treat this product as household waste
	Medical Device compliant with Regulation (EU) 2017/745
	Observe the instructions for use
	Medical device
	Product code
	Unique device identifier
	Imported by
	Temperature limit
	Humidity limit
	Atmospheric pressure limit
	Keep away from sunlight
	Keep in a cool, dry place
	Caution: read instructions (warnings) carefully
	Consult instructions for use
	Date of manufacture
	Lot number