



GIMA

PROFESSIONAL MEDICAL PRODUCTS

BARELLA PROFESSIONALE DA CORZIA
PROFESSIONAL STRETCHER TROLLEY
CHARIOT BRANCARD PROFESSIONNEL D'URGENCE
CARRO CAMILLA PROFESIONAL DE URGENCIAS




- È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.
- All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.
- Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.
- Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

REF

44760



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in Italy

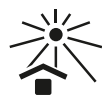


INTRODUCTION

In accord with Regulation (EU) 2017/745, this product is part of the CLASS I MEDICAL DEVICES. You should carefully read this manual before using this medical device.

DESCRIPTION

A professional stretcher trolley is designed for use in hospitals to move patients in A&E and help transfer them from any diagnostic equipment.

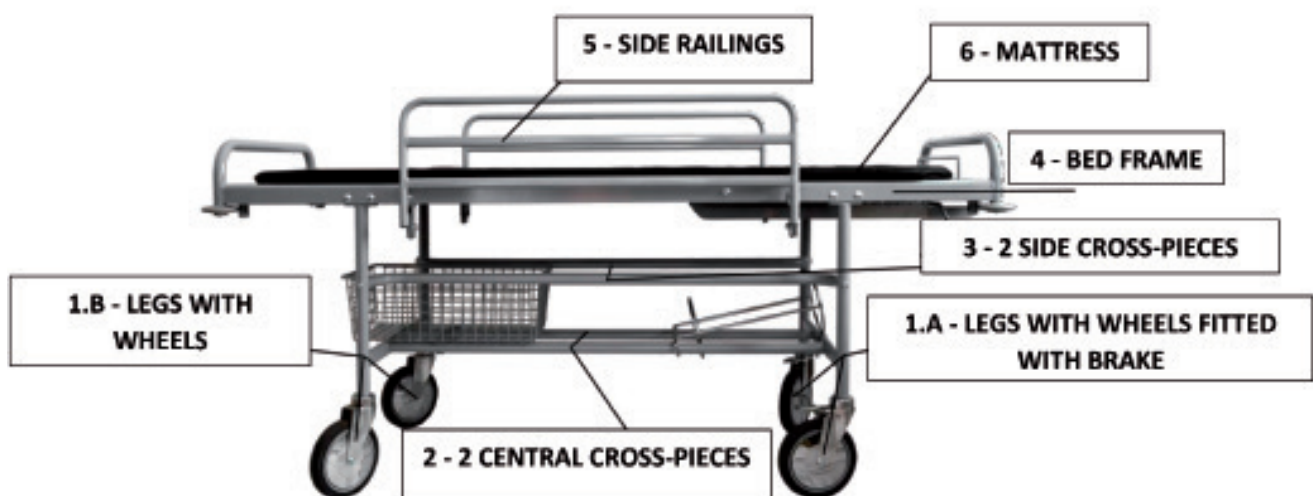

The professional stretcher trolley has a back rest which can be adjusted in five positions by means of an automatic crank, in addition to drop-side railings with automatic lock. The frame has synthetic rubber shockproof bumpers and is fitted with four castor wheels, two of which have independent brakes.

COMPONENTS:

The stretcher trolley is supplied completely dismantled, including accessories for assembly. It consists of the following parts:

- 1.A** • Legs with front wheels fitted with brake
- 1.B** • Legs with rear wheels
- 2** • 2 central cross-pieces
- 3** • 2 side cross-pieces
- 4** • Bed frame with automatic crank and bumpers
- 5** • Two drop-side railings with automatic lock (On request)
- 6** • Black mattress (On request)


On request: Mattress (cod. 44765); Two drop-side railings (cod 44761); Basket (cod. 44763); Oxygen bottle holder (cod. 44764); I.V. Stand with two hooks to attach the pole directly to the bed frame.

ASSEMBLY INSTRUCTIONS:

- N.12 SCREW TTQST 8x40 + bolts and washers
- N.4 SCREW TTDE 8x50 + washers
- N.4 SCREW TTDE 8x40 + washers

ASSEMBLY INSTRUCTIONS:


Begin the assembly phase by setting up a stable horizontal work plan. Fix the two pairs of legs with wheels (1.A and 1.B) through the two central cross-pieces (2) using four TTDE 8x40 SCREWS + washers.

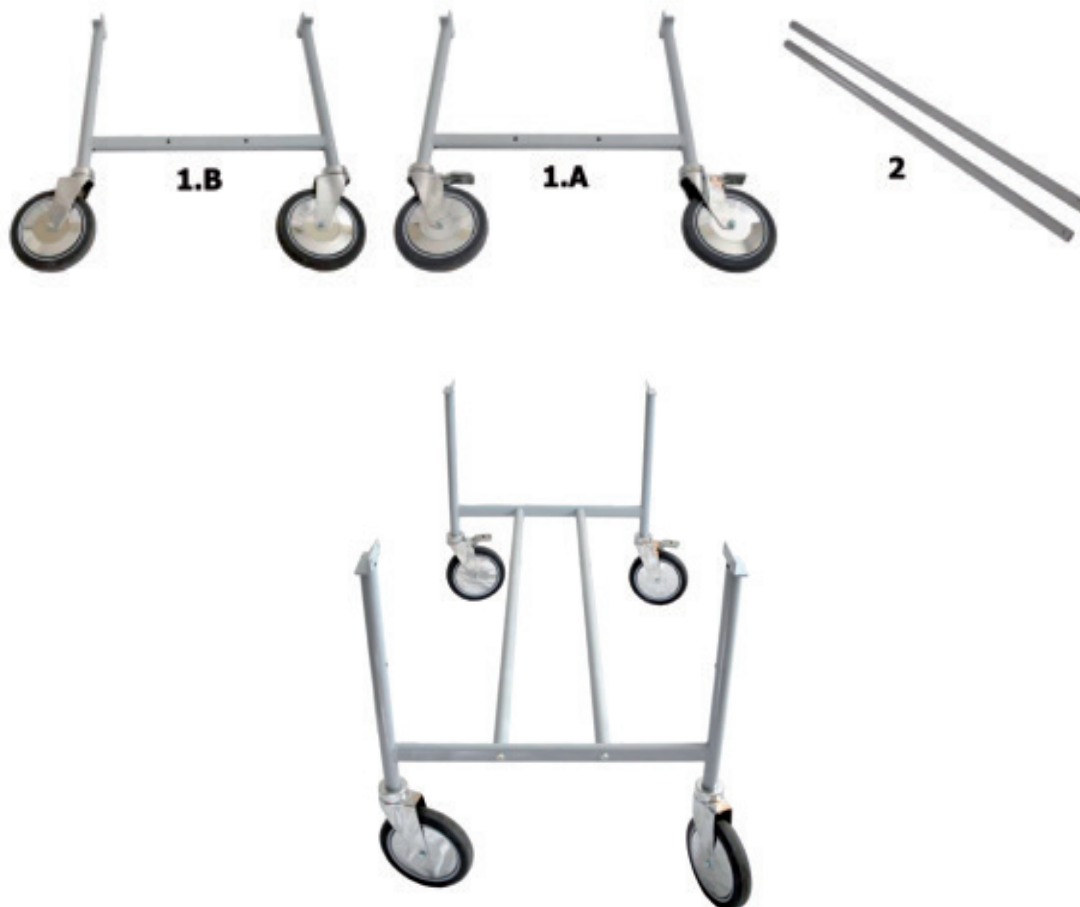

Now fix the side cross-pieces (3) to the frame, again using four TTDE 8x50 SCREWS + washers.



Take the base frame of the stretcher (4) and fix it to the lower structure using eight TTQST 8x40 SCREWS + nuts and washers.

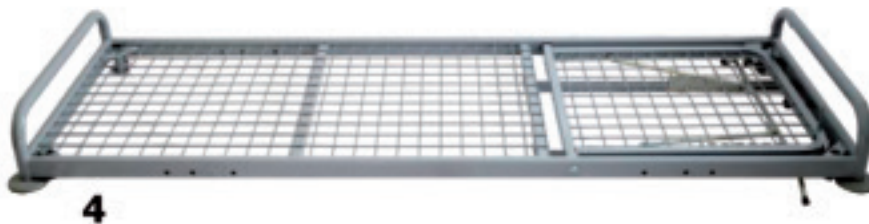

If provided, take the side railings and fix them to the structure of the frame (4) using four TTQST 8x40 SCREWS + nuts and washer for each railing (see enlarged image).



TECHNICAL DATA

Measurement	Stretcher only incl. bumpers: 202 x 71 x h 90 cm (bed platform h 78 cm) Stretcher with side rails: 202 x 75 x h 90 cm (bed platform height 78 cm) Box: cm. 205 x 24 x h76
Weight	40 kg 48 kg with box
Max capacity	135 kg

WARNINGS!












- Do not exceed the maximum capacity specified in the data sheets.
- Check the locking nuts are tightened.
- Avoid direct contact with open wounds.
- Do not use outdoors!
- When adjusting the height of the couch and the back, make sure that no one inserts hands, legs or anything into the movement area: CRUSHING RISK!
- Move the bed without lifting it from the floor.
- Before moving the bed, unlock the wheels.

MAINTENANCE AND CLEANING

- Use only original spare parts.
- Periodically check all fixing points.
- To preserve at best, it is recommended to avoid prolonged exposure to sunlight, contact with saline environments and storage in very humid environments.
- Wash the stainless steel surfaces with soapy water and use appropriate disinfectants.
- Rinse with a damp cloth and dry thoroughly.
- Do not spill on the surface cleaners containing chlorine or ammonia, fatty and oily substances.
- Do not use cleaning tools containing iron. If disinfection is necessary, use only suitable non-corrosive products.

CLEANING RULES FOR COATING:

Clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches, synthetic detergents, polishes or aerosol spray. Attention: Normally synthetic leather light colours should not come into contact with clothing coloured with unfixed dyes (such as jeans and the like) not to cause stains and marks that cannot be cleaned up. e cleaning tools containing iron. If disinfection is necessary, use only suitable non-corrosive products.

	Keep away from sunlight		Medical Device compliant with Regulation (EU) 2017/745		Keep in a cool, dry place
	Product code		Consult instructions for use		Manufacturer
	Caution: read instructions (warnings) carefully		Lot number		Medical Device
	Manufacturing date		Unique device identifier		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.