



GIMA

PROFESSIONAL MEDICAL PRODUCTS

**MEDICAL TROLLEY
CARRELLO CLINICO
CHARIOT CLINIQUE
CARRO DE ANESTESIA
CARRINHO CLÍNICO
KLINIKWAGEN
ΤΡΟΧΗΛΑΤΟ ΝΟΣΗΛΕΙΑΣ
КЛИНИЧНА КОЛИЧКА
ZDRAVOTNICKÝ VOZÍK
KLINIKKAVAUNU
KLINIČKA KOLICA
KLINIKAI KEREEKES KOCSI
MEDICININIS VEŽIMĖLIS
MEDISCHE KAR
WÓZEK MEDYCZNY
CĂRUCIOR PENTRU
INSTRUMENTAR MEDICAL
CLINICAL TROLLEY**



MANUALE D'USO – USER MANUAL – MODE D'EMPLOI – MANUAL DE USO - MANUAL DO USUÁRIO - BENUTZERHANDBUCH - ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ - INSTRUKCJA OBSŁUGI - MANUAL DE UTILIZARE - UŽIVATELSKÝ MANUÁL - ANVÄNDARMANUAL - HANDLEIDING - KORISNIČKI PRIRUČNIK - HASZNÁLATI UTASÍTÁS - РЪКОВОДСТВО ЗА УПОТРЕБА - ΟΗΓΕΚΙΡΙΑ - VARTOTOJO VADOVAS

Gima 45715

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante / Hersteller / Κατασκευαστής / Producent / Producent / Výrobce / Tillverkare / Fabrikant / Proizvođači / Gyártó / производител / Valmistaja / Gamintojas:
Jiangsu Saikang Medical Equipment Co.,Ltd
No. 35 Lehong Road, Modern Agriculture Demonstration Park, Zhangjiagang City, Jiangsu Province, China
export@saikangmedical.com
Made in China

REF SKR-CT625-1


Importato da / Imported by / Importé par / Importado por / Importado por / Importiert von / Εισάγεται από / Importowane przez / Importat de / Importováno / Importerad av / Geïmporteerd door / Uvezao / Importálja / Внесен от / Maahantuoja / Importavo / :

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

1. Casters installation

Align the wheel bolt with the mounting hole on the bottom of the cart and insert the wheel. Insert the screw into the mounting hole and tighten the screw clockwise.

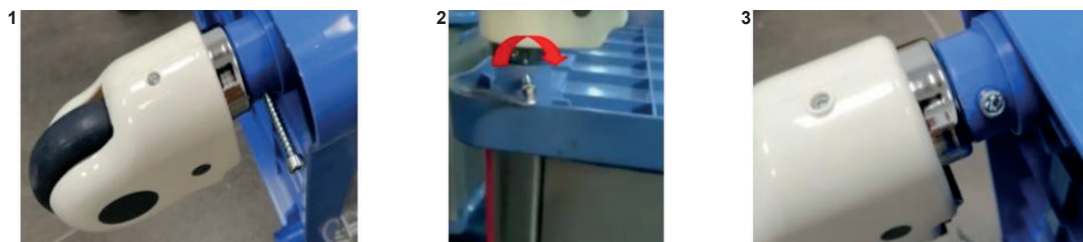

2. Use method of casters

Lock the caster: After pushing the anesthesia trolley to the desired position, press the ON button with your foot and lock the caster when you hear a "click", as shown in the figure below.

Unlock the caster: Press the OFF button with your foot and unlock the castor when you hear a "click".



3. Faults exclusion

| Common failures | Probable reasons | Excluding methods |
|-------------------------------------|---|--|
| Difficult to move the whole trolley | 1. The caster is locked without releasing the brake 2. The caster is damaged | 1. Release the brake,check 2.1 2. Replace the caster,check1.2 |

4. Maintenance

- In order to use the anesthesia trolley safely, it is necessary to perform regular safety inspections on the anesthesia trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
- When the anesthesia trolley frame ages and it reaches a certain servicing time, which the metal part of the anesthesia trolley and plastic parts of the anesthesia trolley frame can be recycle.
- Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
- If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the anesthesia trolley, which will cause the stainless steel surface to rust.
- Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
- It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
- The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
- Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
- For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a anesthesia trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the anesthesia trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
- If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

5. Packaging, Transport, Storage

- The packaging of trolley is carried out according to the contract or product standard.
- Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
- The trolley should be stored:
 - Ambient temperature: +5°C ~ +40°C.
 - Relative humidity: ≤ 85%.
 - Atmospheric pressure: 70 kPa ~ 106 kPa.

6. After-sales service

- Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
- If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
- From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification, the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
- From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.

Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD