



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**CARRELLO D'EMERGENZA NEO**  
**NEO EMERGENCY TROLLEY**  
**CHARIOT D'URGENCE NEO**  
**CARRO DE EMERGENCIAS NEO**  
**CARRINHO DE EMERGÊNCIA NEO**  
**NEO NOTFALLWAGEN**  
**NEO ΤΡΟΧΗΛΑΤΟ ΕΠΕΙΓΟΝΤΩΝ**  
**ΠΕΡΙΣΤΑΤΙΚΩΝ**  
**WÓZEK AWARYJNY NEO**  
**CĂRUCIOR MEDICAL DE URGENȚĂ NEO**

**NEO NOUZOVÝ VOZÍK**  
**NEO NÖDLÄGESVAGN**  
**NEO TROLLEY - ANESTHESIEWAGEN**  
**NEO KOLICA ZA HITNE SLUČAJEVE**  
**NEO ÉRZÉSTELENÍTŐ KOCSI**  
**NEO КОЛИЧКА ЗА СПЕШНО**  
**ОТДЕЛЕНИЕ**  
**NEO-ELVYTYSKÄRRY**  
**NEO SKUBIOS PAGALBOS VEŽIMĖLIS**

MANUALE D'USO – USER MANUAL – MODE D'EMPLOI – MANUAL DE USO - MANUAL DO USUÁRIO -  
BENUTZERHANDBUCH - ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ - INSTRUKCJA OBSŁUGI - MANUAL DE UTILIZARE - UŽIVATELSKÝ  
MANUÁL - ANVÄNDARMANUAL - HANDLEIDING - KORISNIČKI PRIRUČNIK - HASZNÁLATI UTASÍTÁS -  
РЪКОВОДСТВО ЗА УПОТРЕБА - ΟΗΓΕΚΙΡΙΑ - VARTOTOJO VADOVAS



Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante  
/ Hersteller / Κατασκευαστής / Producent / Producent / Výrobce /  
Tillverkare / Fabrikant / Proizvođači / Gyártó / производител / Valmistaja  
/ Gamintojas: **Jiangsu Saikang Medical Equipment Co.,Ltd**  
No. 35 Lehong Road, Modern Agriculture Demonstration Park,  
Zhangjiagang City, Jiangsu Province, China  
export@saikangmedical.com  
Made in China

Importato da / Imported by / Importé par / Importado por / Importado  
por / Importiert von / Εισάγεται από / Importowane przez / Importat de /  
Importováno / Importerad av / Geïmporteerd door / Uvezao / Importálja  
/ Внесен от / Maahantuoja / Importavo / :

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
www.gimaitaly.com

**REF** SKR054-ET

**Gima 45722**


### 1. CASTERS INSTALLATION

Align the wheel bolt with the mounting hole on the bottom of the cart and insert the wheel. Insert the screw into the mounting hole and tighten the screw clockwise.


**1****2****3**

### 2. USE METHOD OF CASTERS

Lock the caster: After pushing the emergency trolley to the desired position, press the ON button with your foot and lock the caster when you hear a "click", as shown in the figure below.

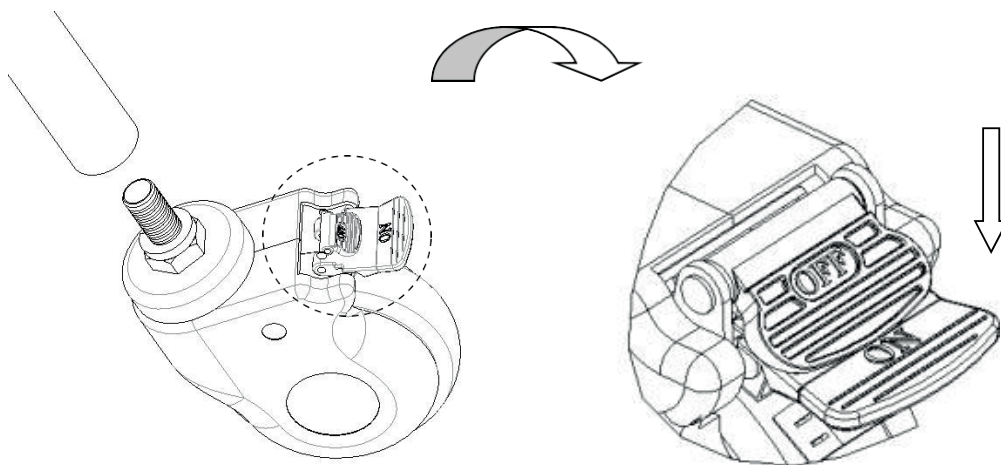

Unlock the caster: Press the OFF button with your foot and unlock the castor when you hear a "click".

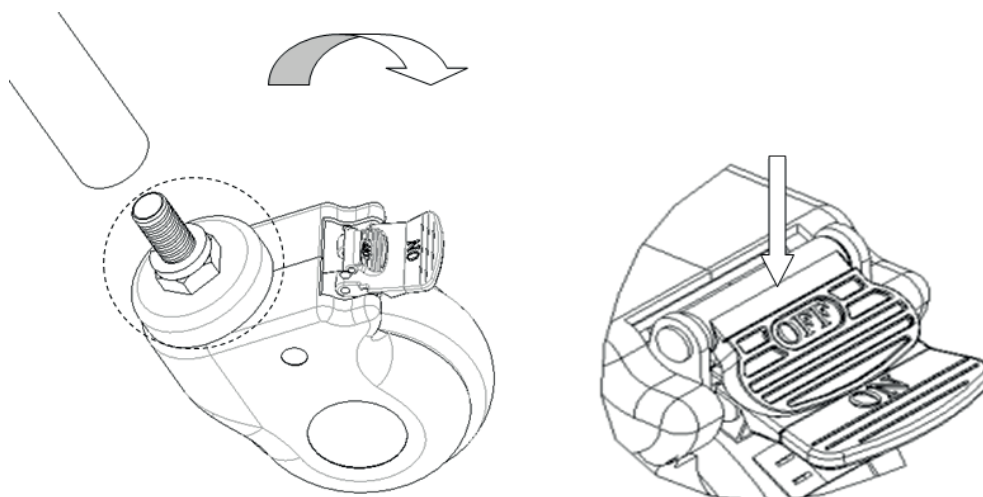

### 3. FAULTS EXCLUSION

Common failures	Probable reasons	Excluding methods
Difficult to move the whole trolley	1. The caster is locked without releasing the brake 2. The caster is damaged	1. Release the brake 2. Replace the caster

### 4. MAINTENANCE

- In order to use the emergency trolley safely, it is necessary to perform regular safety inspections on the emergency trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
- When the emergency trolley frame ages and it reaches a certain servicing time, which the metal part of the emergency trolley and plastic parts of the emergency trolley frame can be recycle.
- Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
- If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the emergency trolley, which will cause the stainless steel surface to rust.
- Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
- It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
- The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
- Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
- For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a emergency trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the emergency trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
- If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

### 5. PACKAGING, TRANSPORT, STORAGE

- The packaging of trolley is carried out according to the contract or product standard.
- Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
- The trolley should be stored:
  - Ambient temperature: +5°C ~ +40°C.
  - Relative humidity: ≤ 85%.
  - Atmospheric pressure: 70 kPa ~ 106 kPa
  - The max load is 50 Kg

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies