

**CARRELLO PORTAMEDICINALI  
MEDICINE TROLLEY  
CHARIOT MÉDICAL  
CARRO DE MEDICAMENTO  
CARRINHO DE MEDICAMENTOS  
MEDIZINWAGEN  
ΤΡΟΧΗΛΑΤΟ ΝΟΣΗΛΕΙΑΣ  
WÓZEK MEDYCZNY  
ČĀRUCIOR MEDICAL**

**VOZÍK NA LÉKY  
MEDICINTEKNISK VAGN  
MEDICIJNWAGEN  
MEDICINSKA KOLICA  
GYÓGYSZERES KOCSI  
МЕДИЦИНСКА КОЛИЧКА  
LÄÄKEVAUNU  
MEDICININIS VEŽIMĒLIS**

MANUALE D'USO – USER MANUAL – MODE D'EMPLOI – MANUAL DE USO - MANUAL DO USUÁRIO -  
BENUTZERHANDBUCH - ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ - INSTRUKCJA OBSŁUGI - MANUAL DE UTILIZARE - UŽIVATELSKÝ  
MANUÁL - ANVÄNDARMANUAL - HANDLEIDING - KORISNIČKI PRIRUČNIK - HASZNÁLATI UTASÍTÁS -  
РЪКОВОДСТВО ЗА УПОТРЕБА - ΟΗΓΕΚΙΡΙΑ - VARTOTOJO VADOVAS



## Gima 45725

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante  
/ Hersteller / Κατασκευαστής / Producent / Producent / Výrobce /  
Tillverkare / Fabrikant / Proizvođač / Gyártó / производител / Valmistaja  
/ Gamintojas:

**Jiangsu Saikang Medical Equipment Co.,Ltd**

No. 35 Lehong Road, Modern Agriculture Demonstration Park, Zhangjia-  
gang City, Jiangsu Province, China  
export@saikangmedical.com  
Made in China

Importato da / Imported by / Importé par / Importado por / Importado  
por / Importiert von / Εισάγεται από / Importowane przez / Importat de /  
Importovano / Importerad av / Geïmporteerd door / Uvezao / Importálja  
/ Внесен от / Maahantuoja / Importavo / :

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
www.gimaitaly.com

**REF SKR054-MT**

### 1. CASTERS INSTALLATION

Align the wheel bolt with the mounting hole on the bottom of the cart and insert the wheel. Insert the screw into the mounting hole and tighten the screw clockwise.



1




2




3

### 2. USE METHOD OF CASTERS

Lock the caster: After pushing the medicine trolley to the desired position, press the ON button with your foot and lock the caster when you hear a “click”, as shown in the figure below.

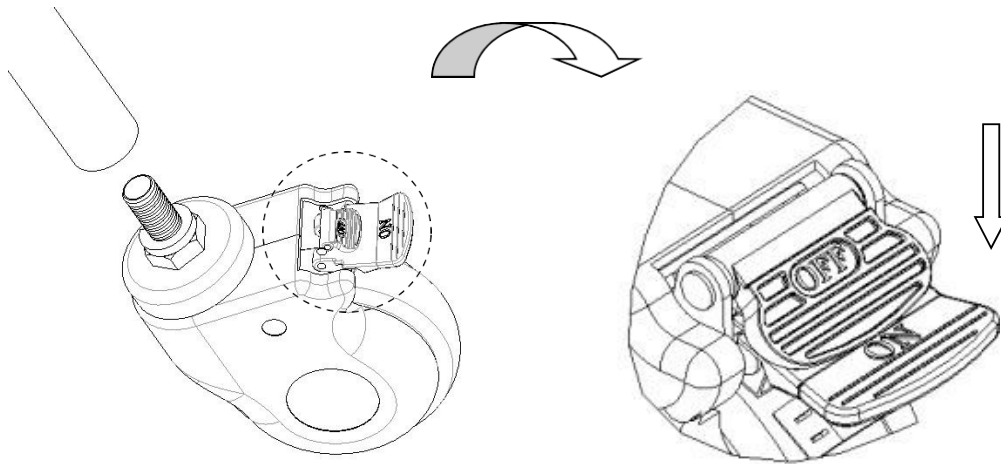

Unlock the caster: Press the OFF button with your foot and unlock the castor when you hear a “click”.

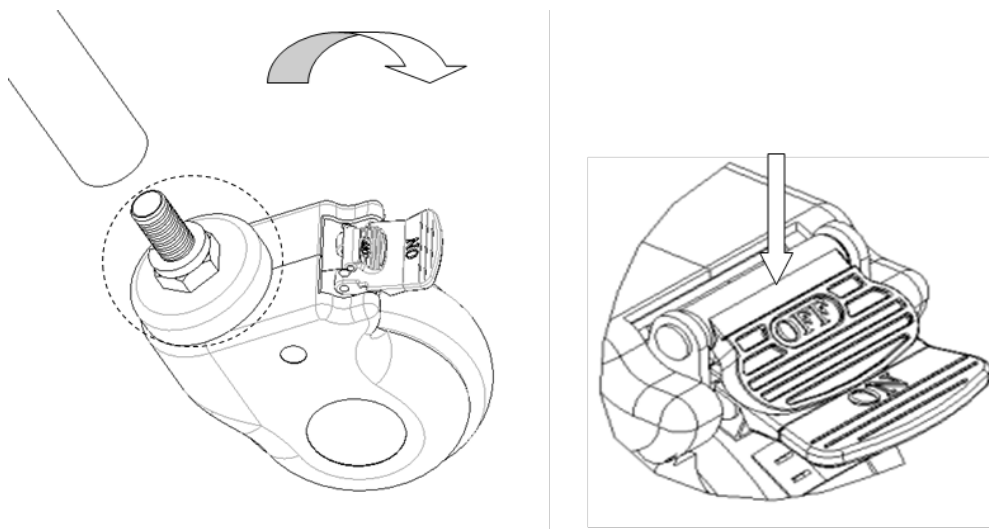

### 3. FAULTS EXCLUSION

Common failures	Probable reasons	Excluding methods
Difficult to move the whole trolley	1. The caster is locked without releasing the brake 2. The caster is damaged	1. Release the brake 2. Replace the caster

### 4. MAINTENANCE

- In order to use the medicine trolley safely, it is necessary to perform regular safety inspections on the medicine trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
- When the medicine trolley frame ages and it reaches a certain servicing time, which the metal part of the medicine trolley and plastic parts of the medicine trolley frame can be recycle.
- Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
- If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the medicine trolley, which will cause the stainless steel surface to rust.
- Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
- It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
- The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
- Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
- For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a medicine trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the medicine trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
- If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

### 5. PACKAGING, TRANSPORT, STORAGE

- The packaging of trolley is carried out according to the contract or product standard.
- Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
- The trolley should be stored:
  - Ambient temperature: +5°C ~ +40°C.
  - Relative humidity: ≤ 85%.
  - Atmospheric pressure: 70 kPa ~ 106 kPa
  - The max load is 50 Kg

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies