

BARELLA-SEDIA A ROTELLE PIEGHEVOLE
WHEELCHAIR FOLDING STRETCHER
CHARIOT-BRANCARD PLIABLE
ROLLSTUHL UND KLAPPBARE TRAGE
CAMILLA PLEGABLE PARA SILLAS DE RUEDAS
ΠΤΥΣΣΟΜΕΝΟ ΦΟΡΕΙΟ-ΑΝΑΠΗΡΙΚΗ ΚΑΡΕΚΛΑ
НОСИЛКА-ИНВАЛИДНА КОЛИЧКА С КОЛЕЛА, СГЪВАЩА СЕ
SKLÁDACÍ NOSÍTKA PRO INVALIDNÍ VOZÍK
SAMMENKLAPPELIG KØRESTOLSBÅRE
KOKKUPANDAV RATASTEGA KANDERAAM-TOOL
TAITETTAVA PAARI-PYÖRÄTUOLI
SKLOPIVA INVALIDSKA KOLICA NOSILA
ÖSSZECSUKHATÓ KEREKESSZÉK-ÖSSZECSUKHATÓ
SULANKSTOMAS NEJGALIOJO VEŽIMĖLIS – NEŠTUVAI
RATIŅKRĒSLA SALIEKAMĀS NESTUVES
INKLAPBARE ROLSTOELBRANCARD
NOSZE SKŁADANE W WÓZEK INWALIDZKI
TARGĀ PLIABILĀ TIP SCAUN CU ROTILE
ZLOŽLJIVA NOSILA ZA INVALIDSKI VOZIČEK
HOPFÄLLBAR SJUKBÅR FÖR RULLSTOL

REF TD010113A (GIMA 56020)



Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd
No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made in China



SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Technical Parameter:

1. Size(L*W*H): Unfolded size: 196*50*18 (cm) Folded size: 106*50*15 (cm)
2. Folded into wheelchair: Back-height: 136cm Seat-height: 52cm
3. Net weight:9kg Gross weight:11kg
4. The bearing pressure: 159kg



Operation method

1. As shown in Figure one: Due to expansion aluminum alloy wheelchair stretcher, is good to use safety belt after the patient is on the stretcher.
2. Folded the stretcher to wheelchair type, As shown in Figure two, fold the first, lift up to ward the direction as shown by the arrow, which would come to the position shown in Figure three. Fasten the safe buckle (Ensures the buckle 2 to buckle on the shear bolt 1).
3. Show the chart 3 for support feet, also has another kind is feet wheel type. Product performance, Usage, Applicable scope and so on are all same.
4. As shown in Figure four, after folding stretcher, is advantageous for the carry home (When the stretcher is stay in Figure three, Loosens the safe buckle, Steps on a cross rod beside the feet wheel, upward raises the seat back cushion pole, causing a stretcher return to in Figure one, and can be folded).

Applicable scope:

This product can be used in Narrow entrance hall Corridor Winding corridor or multi-platform stair hall and so on enters with difficulty enclosed area.

Maintenance:

1. Frequently Maintains neatly (Including disinfection)
2. Frequently Inspection whether the parts become less crowded.

Reminds specially









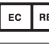



Must guarantee the safety belt in good condition when used, and certainly to ensure the buckle to buckle on the shear bolt when wheelchair stretcher type is used.

Storage and transportation::

1. This product should store within moisture-proof non-corrosion environment.
2. This product available general conveyance for transportation.
3. According to the graphical representation in packing case Carries on the storage, transportation.

Packing and accessory::

1. This product is packed in carton
2. Accessory: Certificate of competency, user manual.

	Manufacturer		Keep in a cool, dry place		Imported by
	Product code		Non-sterile		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		
	Medical Device compliant with Regulation (EU) 2017/745				

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.