



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

TRUSTEEL™

**FILO DA SUTURA CHIRURGICA NON RIASSORBIBILE USP
FILO DA SUTURA CHIRURGICA STERILE, MONTATO SU AGO
MONOFILAMENTO IN ACCIAIO INOX LVM 316**

**NONABSORBABLE SURGICAL SUTURE, U.S.P.
STERILIZED SURGICAL NEEDLED SUTURE MONOFILAMENT
STAINLESS STEEL LVM 316 GRADE WIRE**

**SUTURE CHIRURGICALE NON RÉSORBABLE, USP SUTURE
CHIRURGICALE STÉRILISÉE, AIGUILLETÉE FIL
MONOFILAMENT EN ACIER INOXYDABLE LVM 316**

**NICHT-RESORBIERBARES CHIRURGISCHES
NAHTMATERIAL, U.S.P STERILISIERTES CHIRURGISCHES
NADELNAHTMATERIAL MONOFILER EDELSTAHL LVM 316
GRADE DRAHT**

**SUTURA QUIRÚRGICA NO ABSORBIBLE, U.S.P. SUTURA
QUIRÚRGICA ESTERILIZADA CON AGUJA
MONOFILAMENTO DE ACERO INOXIDABLE, GRADO DEL
FILAMENTO LVM 316**

**SUTURA CIRÚRGICA NÃO ABSORVÍVEL, U.S.P.
SUTURA AGULHADA CIRÚRGICA ESTERILIZADA FIO
MONOFILAMENTO DE AÇO INOXIDÁVEL CLASSE LVM 316**

**ΜΗ ΑΠΟΡΡΟΦΗΣΙΜΟ ΧΕΙΡΟΥΡΓΙΚΟ ΡΑΜΜΑ U.S.P
ΑΠΟΣΤΕΙΡΩΜΕΝΟ ΧΕΙΡΟΥΡΓΙΚΟ ΡΑΜΜΑ ΜΕ ΒΕΛΟΝΑ
ΜΟΝΟΚΛΩΝΟ ΣΥΡΜΑ ΑΠΟ ΑΝΟΞΕΙΔΩΤΟ ΧΑΛΥΒΑ
ΚΑΤΗΓΟΡΙΑΣ LVM 316**

**Manuale d'uso - User manual - Manuel de l'utilisateur
Guía de uso - Gebrauchs- und instandhaltungsanleitung
Guia para utilização - Οδηγίες χρήσης**

	IT - Identificatore univoco del dispositivo GB - Unique device identifier FR - Identifiant unique de l'appareil ES - Identificador de dispositivo único DE - Eindeutige Gerätekennung PT - Identificador exclusivo do dispositivo GR - Μοναδικό αναγνωριστικό συσκευής
	IT - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB - Caution: read instructions (warnings) carefully FR - Attention: lisez attentivement les instructions (avertissements) DE - Achtung: Anweisungen (Warnings) sorgfältig lesen ES - Precaución: lea las instrucciones (advertencias) cuidadosamente PT - Cuidado: leia as instruções (avisos) cuidadosamente GR - Προσοχή: διαβάστε προσεκτικά τις οδηγίες (επιστάσεις)

	IT - Codice prodotto GB - Product code FR - Code produit DE - Erzeugniscode ES - Código producto PT - Código produto GR - Κωδικός προϊόντος
	IT - Dispositivo monouso, non riutilizzare GB - Disposable device, do not re-use FR - Ne pas réutiliser DE - Für einmaligen Gebrauch, nicht wiederverwenden ES - Dispositivo monouso, no reutilizable PT - Dispositivo descartável, não reutilizar GR - Προϊόν μιας χρήσεως. Μην το χρησιμοποιείται εκ νέου
	IT - Numero di lotto GB - Lot Number FR - Numéro de lot DE - Chargennummer ES - Número de lote PT - Número de lote GR - Αριθμός παρτίδας
	IT - Data di fabbricazione GB - Date of Manufacturing FR - Date de fabrication DE - Herstellungsdatum ES - Fecha de fabricación PT - Data de fabrico GR - Ημερομηνία παραγωγής
	IT - Data di scadenza GB - Expiration date FR - Date d'échéance DE - Ablaufdatum ES - Fecha de Caducidad PT - Data de validade GR - Ημερομηνία λήξεως
	IT - Sterilizzato con ossido di etilene GB - Sterilized using ethylene oxide FR - Stérilisé à l'oxyde d'éthylène DE - Sterilisiert mit Ethylenoxid ES - Esterilizado con óxido de etileno PT - Esterilizado com óxido de etileno GR - Αποστειρωμένο με αιθυλενοξείδιο
	IT - Rappresentante autorizzato nella Comunità europea GB - Authorized representative in the European community FR - Représentant autorisé dans la Communauté européenne DE - Autorisierter Vertreter in der EG ES - Representante autorizado en la Comunidad Europea PT - Representante autorizado na União Europeia GR - Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση
	IT - Fabbricante GB - Manufacturer FR - Fabricant DE - Hersteller ES - Fabricante PT - Fabricante GR - Παραγωγός
	IT - Non ri-sterilizzare GB - Do not resterilize FR - Ne pas restériliser DE - Nicht erneut sterilisieren ES - No reesterilizar PT - Não reesterilize GR - Μην αποστειρώνετε
	IT - Conservare in luogo fresco ed asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco DE - An einem kühlen und trockenen Ort lagern PT - Armazenar em local fresco e seco GR - Διατηρείται σε δροσερό και στεγνό περιβάλλον
	IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil DE - Vor Sonneneinstrahlung geschützt lagern ES - Conservar al amparo de la luz solar PT - Guardar ao abrigo da luz solar GR - Κρατήστε το μακριά από ηλιακή ακτινοβολία
	IT - Limite superiore di temperatura GB - Upper limit of temperature FR - Limites supérieure de température DE - Obergrenze der Temperatur ES - Limitaciones superiorde temperatura PT - Limitação superior de temperatura GR - Ανώτερο όριο θερμοκρασίας
	IT - Non utilizzare se l'imballaggio è danneggiato GB - Don't use if package is damaged FR - Ne pas utiliser si le colis est endommagé DE - Nicht verwenden, wenn das Paket beschädigt ist ES - No usar si el paquete está dañado PT - Não use se o pacote estiver danificado GR - Μην το χρησιμοποιείτε αν η συσκευασία είναι κατεστραμμένη
	IT - Leggere le istruzioni per l'uso GB - Consult instructions for use FR - Consulter les instructions d'utilisation DE - Gebrauchsanweisung beachten ES - Consultar las instrucciones de uso PT - Consulte as instruções de uso GR - Διαβάστε προσεκτικά τις οδηγίες χρήσης
	IT - Dispositivo medico conforme alla Direttiva 93/42/CEE GB - Medical Device complies with Directive 93/42/EEC FR - Dispositif médical conforme à la directive 93/42 / CEE DE - Medizinprodukt gemäß Richtlinie 93/42/CEE ES - Dispositivo médico según a la Directiva 93/42 / CEE PT - Dispositivo médico em conformidade com a Diretiva 93/42/CEE GR - Ιατρική συσκευή σύμφωνα με την οδηγία 93/42 / CEE

- REF** TS661P4 (GIMA 22790)
 TS650P4 (GIMA 22791)
 TS652P4 (GIMA 22792)
 TS659P4 (GIMA 22793)
 TS649P4 (GIMA 22794)
 TS644P2 (GIMA 22795)
 TS657P4 (GIMA 22796)

EC REP MED DEVICES LIFESCIENCES B.V.
 Kraijenhoffstraat 137 A, 1018RG Amsterdam, Netherlands
 Email: info@meddevices.net Phone: +31-202254558

 Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

 **Healthium Medtech Private Limited**
 Plot No.1600, R-6 West, Sri City (SEZ),
 Chervi Village, Irrugulam Post, Satyavedu Mandal, Chittoor District,
 Andhra Pradesh- 517588, India
 Email : sales@healthiummedtech.com
 Mfg. Lic. No.: 42/CT/AP/2012/S/R

     **2265**

     