EN

INTENDED USE: Fertility Gel, supports the motility of the sperm as it reaches the egg, thus increasing the probability of fertilization. In addition, its content is effective in maintaining its viability as it simulates the environment in which the sperm can survive.

APPLICATION: Gynoseed Fertility Gel is applied inside the vagina before the sexual intercourse. Dispose of the packaging adequately after use.

MECHANISM OF ACTION: Fertility gel is formulated to provide a balance of pH to increase the survival of the sperm and to increase the survival. It contains magnesium and calcium ions to support conception.

INDICATIONS: Fertility Gel, supports the motility of the sperm as it reaches the egg, thus increasing the probability of fertilization. In addition, its content is effective in maintaining its viability as it simulates the environment in which the sperm can survive.

CONTRAINDICATIONS:

- Infection in the genital area
- Irritated genital area
- Known allergic condition to any of its ingredients
- Breastfeeding status
- Excessive vaginal fluid secretion

WARNINGS AND USAGE PRECAUTIONS: Keep between 5 °C and 30 °C. Keep away from direct sunlight. Fertility is not a contraceptive gel product. Do not use as a vaginal lubricant. Avoid contact with eyes. Do not swallow. Use the entire tube, do not store it for second use. Do your general personal cleaning after the application and sexual intercourse. Keep away from children. Store in dry and cool places. Do not be used in those who are allergic to the ingredients. Do not be used after curettage.

SIDE EFFECTS: There is no known side effect when used in accordance with the instructions for use and warnings.

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