IT

TAMPONE COTONE - 20 cm

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GB

COTTON SWABS - 20 cm

Cotton applicator swab - big headed - indicated for cleaning, drying, wiping, application of solutions
Only for specialists

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Disposable device: any reuse could cause the risk of cross-contaminations and/or failed sterility or cleaning.

- Do not expose to heat • Promptly use after wrapping • Check device's conditions before use.

Intended use: Stick cotton BH is used for cleaning, drying, wiping, application of solutions.

Indications: it is particularly indicated for gynecologists and obstetrics. It can be used to remove the cervical mucus, pad bleeding during day surgery operations, for the disinfection of the cervix, vagina and vulva and during colposcopy for the application of 5% acetic-acid solution or Lugol's solution to the cervix.

Contraindications: none

Directions for use: hold the stem when wrapping

- To apply solutions dip the stick cotton in the solution and wait for a while to help absorption • Apply to the

mucosae with moderate pressures. • After use, dispose of as normal hospital waste.

GIMA WARRANTY TERMS
The Gima 12-month standard B2B warranty applies.

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GIMA S.p.A.