120.00 mm

FFP2 NR

Value&Safe

Mascherina filtrante piccola - arcobaleno
 Filtering mask small - rainbow
 Masque filtrant petite - arc-en-ciel
 Mascarilla filtrante pequeña - arco iris
 Filtrierende Maske Kleine - Regenbogen

1 Pezzo
 Pièce
 Pieza
 Stück

REF AD-T001 (20738)

LOT

Modello / Model / Modèle / Modelo / Modell: AD-T001

AnDum Protective Equipment Technology (Changzhou) Co., Ltd.
 No. 216 Qianjie, Hengshanqiao Town, Changzhou Economic Zone,
 Changzhou City, Jiangsu Province, China
 Made in China

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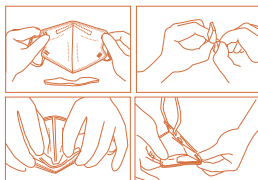
Warning:

1. Please check the visible damage, dirt and the expiry date before use. 2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death. 3. A properly selected respirator is essential to protect your health. Before using this respirator, consult an industrial hygienist or occupational safety professional to determine the suitability for your intended use. Always check the respirator for fit prior to use. 4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%. 5. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this respirator in explosive environment. 6. Leave work area immediately and return to fresh air if, a) breathing becomes difficult, or b) dizziness or other distress occurs. 7. Never alter or modify this respirator in any way. 8. Do not use for more than one shift (8 hours). No maintenance necessary. Discard the used respirator after single shift use. 9. Keep respirator in the packaging box away from direct sunlight until use. 10. When using the mask, objects not related to the wearing of the mask, such as mask packaging materials, should be cleaned away in time. 11. Minors should wear and use the mask under the supervision of an adult. The caregiver should observe and educate the minor on the correct way to wear the mask. Minors should not play or engage in moderate or higher intensity exercise while wearing the mask and should not disassemble the breathing valve or its

internal parts. If symptoms such as breathing discomfort or skin irritation occur during wear, the mask should be removed promptly and medical attention should be sought if necessary. 12. Do not swallow the snaps and sealing sponges in the bag.

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SGS Fimko Oy,