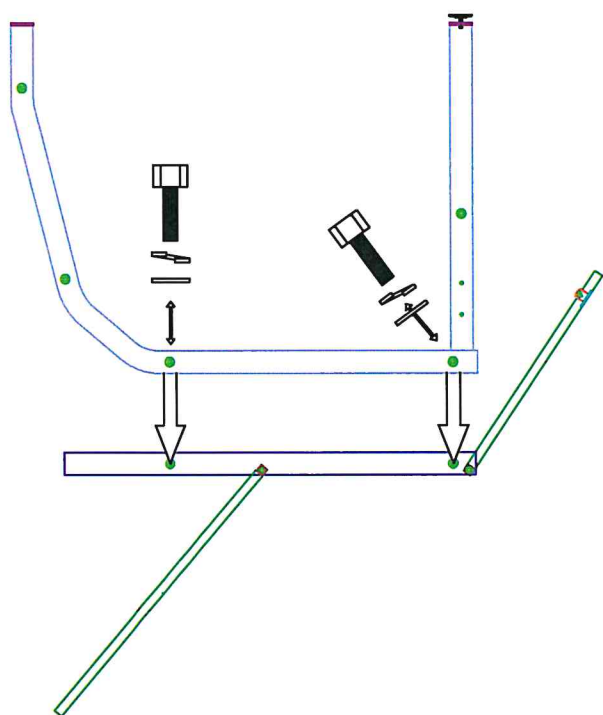
ATTENZIONE:

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
ATTENTION:

the operators must carefully read and completely understand the present manual before using the product.

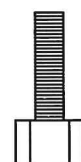FLAT WHEEL RONDELLA PIANA ← → 
 OPEN WHEEL RONDELLA SPACCATA ← → 

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1. Place the legs facing the threaded sleeve into the frame of the padded platform by keeping hole with bigger diameter on the external side of the bed.
 2. Insert the screws (make sure that their wheels are correctly placed into allen screw).
- Screws must be firmly fixed at the end of assembly.

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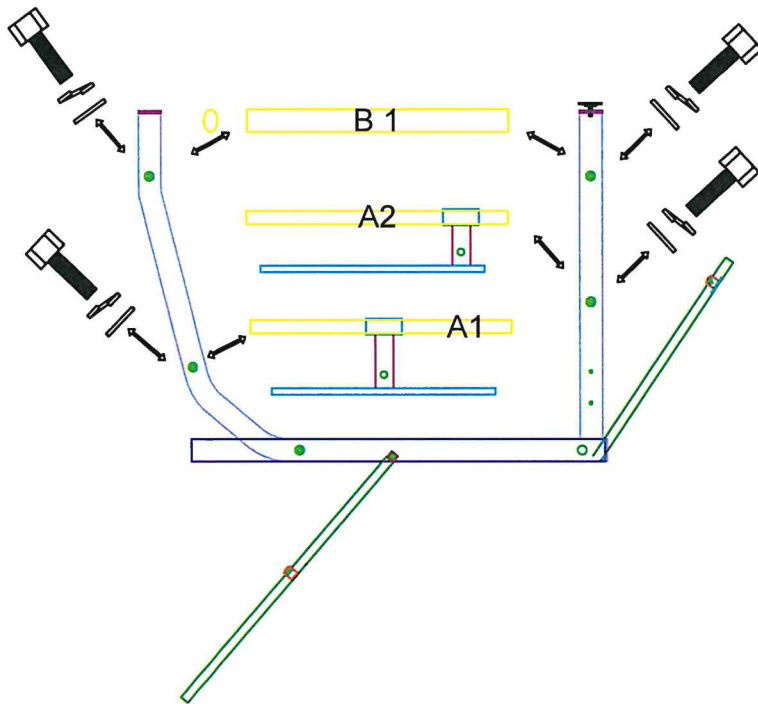
3. Insert the accessory for blocking the cushions into the chromed crossbar A1,A2.

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

4. Place the crossbars as shown in the drawing:
 A1 blocking with long crossbars
 A2 blocking with short crossbars
 B1 oval tube.

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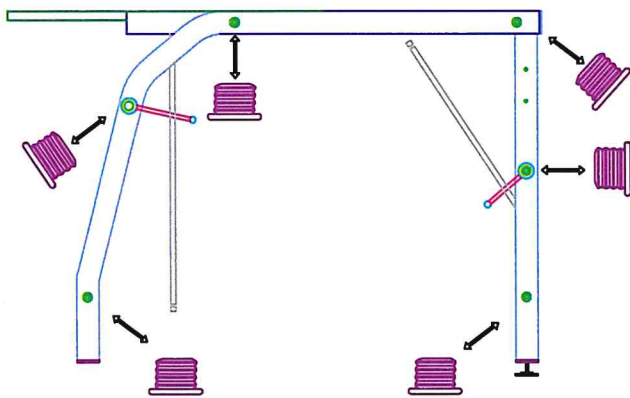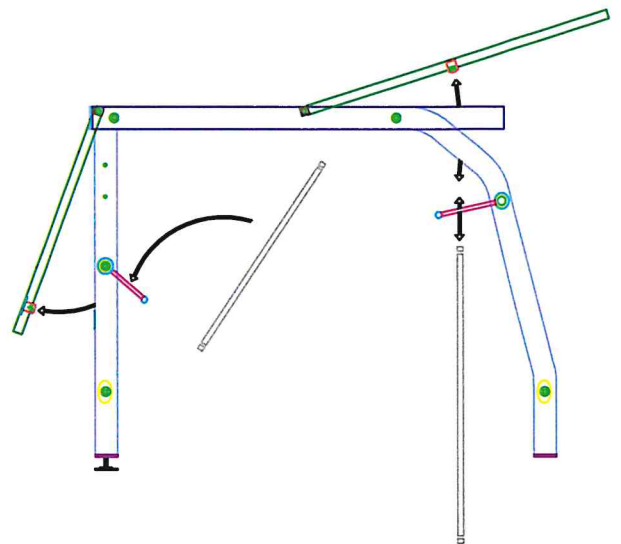
5. Insert the long chromed rod in A1 and the short rod in A2
 fasten the rods to the cushions with proper screws after checking that the block
 accessory faces the center of bed.

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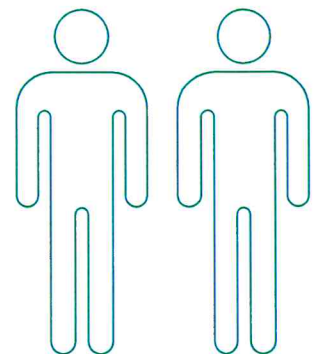6. Fasten the screws firmly and insert 12 plastic head covers.

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WARRANTY CERTIFICATE

This certificate must be kept until the expiry date. It must be presented every time a technical intervention is necessary, accompanied by the invoice, tax receipt, delivery note or any supporting document showing the name of the retailer and the date of purchase. The user who is unable to produce these documents will lose all the warranty rights. The warranty starts from the date of purchase shown on the invoice, tax receipt or delivery note and it is valid for 24 months for the moving and electric parts, 36 months for product structure. The guarantee refers to the replacement or free repair, within the established terms, of the components of the appliance which, at the discretion of the manufacturer, were found to be defective at the origin due to manufacturing defects. This warranty does not cover:

- transport faults (dents, scratches and similars);
- damage due to falls;
- damage caused by carelessness, negligence, inability to use the product, tampering and repairs carried out by unauthorized personnel;
- damages due to incorrect installation; if this is carried out by the user or by unauthorized personnel;
- damages caused by insufficiency or inadequacy of the electrical system or alteration deriving from environmental, climatic or other conditions, as well as damages caused by phenomena that do not depend on the normal operation of the appliance;
- technical interventions concerning the installation of the appliance and the connection to the power supply systems;
- home repairs for convenience checks or alleged defects. If requested, the user will have to pay to the person in charge a contribution previously estimated in advance with the intervention;
- ordinary maintenance;
- whatever may be normal decay for use.

However, product replacement is excluded. The warranty intervention can be refused when the product has been modified or transformed in any way. The restoration of the appliance will be carried out within the time limits compatible with the organizational requirements, by authorized technicians, upon request by the user to the retailer or directly to the manufacturer.

In the event of home intervention, the user is required to pay the fixed fee for transfer fees. If the appliance is repaired at the retailer or at the manufacturer, the transport costs and the relative risks will be charged to the user. Warranty repairs do not result in an extension of the warranty itself. Parts replaced under warranty remain property of the manufacturer.

No one is authorized to change the terms and conditions of the warranty or to issue other verbal or written ones. The manufacturer declines all responsibility for any damage, of any nature, that may directly or indirectly derive from person, animals or things resulting from failure to comply with all the instructions contained in this manual concerning, in particular, the installation warnings, use and maintenance of the appliance or in any case improper use. The manufacturer is also not responsible for any damage caused to people, animals or things due to breakdown or forced suspension of use of the equipment. The Court of Como has jurisdiction for any dispute.

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SPECIAL WARNINGS AND LIMITATIONS OF USE:

the structure must NOT be used:

-used in environment with risk of fire, explosive or corrosive area

-exposed to severe weather conditions

-tampered with and/or modifications must be made to original product

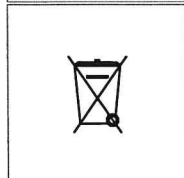
-do not put parts of the body in areas subject to movement and/or adjustments

the structure must be used in conditions of stability and on flat surface

-keep bed still while moving patient on and of bed to avoid any risk

-the lamps used for therapeutic irradiation can damage the upholstery imitation leather

-corrosive and/or abrasive detergents can damage upholstery imitation leather

MADE IN ITALY