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*Swivel Chair for Bathtub Painted Steel*

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*Before making any operation, visually inspect the integrity of the device.*

*If the product is abnormal, immediately contact the store from which you purchased the product. Vaimas srl does not assume any responsibility for the improper use of the device.*

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*Swivel chair for bathtub with seat and back in polypropylene with holes for water drainage. Painted steel structure and rotation components and assembly in stainless steel.*

*Supports for bathtub covered in pvc.*

*The device can be rotated 360 ° in four positions from 90 ° to facilitate access to the pool for people with reduced mobility.*

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*Frame and armrests in Painted steel.  
Seat and back in polypropylene.  
Supports for bath in PVC.  
Flyers and knobs in polyamide and stainless steel.*

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*Dispose of the device comply with regulations relating to environmental protection and recycling.*

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*Before use, always check the correct positioning of the device on the edges of the bathtub and tighten the side screws. Check the condition of plastic covers of the supports for the bathtub. Ensure the stability of the chair verifying that the locking lever under the seat is properly secured in the hole in the desired position. Always assist users with limited walking ability.*

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*Place the chair on the bathtub so that the lateral supports of the device are properly supported on the edges of the tub. Adjust the two knobs located laterally*

*on the frame of the chair until the two plastic protections become adherent to the inner surface of the bathtub.*

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*Clean after each use with a soft cloth and neutral detergent. Do not use any acidic or abrasive sponges or alcohol base.*

*Periodically verify the operation of the locking lever under the seat. Check the welding paying attention to cracks or lifting of paint. Check the proper tightening of screws and bolts and, in particular:*

- the fixing screws of the seat, paying attention to the state of the plastic substrates on which are screwed (There must be no cracks or breaks);*
- the fastening nut of the central pin located under the seat.*

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*Keep away from sources of light, heat and moisture. Avoid direct exposure to sunlight.*

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*The device is covered by a 24 month warranty for manufacturing defects.*

*The warranty does not apply to products modified or repaired without the prior consent of Vaimas srl.*