it

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en

INSTALLATION INSTRUCTIONS,
USER MANUAL AND
PRE-SALE INFORMATION

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GENERAL WARNINGS		page 5
WARNINGS AND CONTRAINDICATIONS		page 5
INTENDED USE		page 5
INSTALLATION		page 5
USE OF THE PRODUCTS		page 5
CLEANING AND MAINTENANCE		page 5
DISPOSAL		page 5
ICONS LEGEND		pages 6–7
PRODUCTS INSTALLATIONS:		pages 8–17
cod. EH-12-RX		pages 8–9
cod. EH-16-RX		pages 8–9
cod. EH-18-RX		pages 8–9
cod. EH-24-RX		pages 8–9
cod. EH-CPS-RX		pages 8–11
cod. EH-WF-RX		pages 8–11
cod. EH-12F-RX		pages 8–11
cod. EH-16F-RX		pages 8–11
cod. EH-18F-RX		pages 8–11
cod. EH-24F-RX		pages 8–11
cod. EH-40F-RX		pages 8–11
cod. EH-12-PCX	EH-12-IXX	pages 12–13
cod. EH-18-PCX	EH-18-IXX	pages 12–13
cod. EH-24-PCX	EH-24-IXX	pages 12–13
cod. EH-28-PCX	EH-28-IXX	pages 12–13
cod. EH-32-PCX	EH-32-IXX	pages 12–13
cod. EH-36-PCX	EH-36-IXX	pages 12–13
cod. EH-1212-PCX	EH-1212-IXX	pages 12–13
cod. EH-1224D-PCX	EH-1224D-IXX	pages 12–13
cod. EH-1224S-PCX	EH-1224S-IXX	pages 12–13
cod. EH-WA-60-PCX	EH-WA-60-IXX	pages 14–15
cod. EH-WA-70-PCX	EH-WA-70-IXX	pages 14–15
cod. EH-WA-80-PCX	EH-WA-80-IXX	pages 14–15
cod. EH-WA-PH-PCX	EH-WA-PH-IXX	pages 16–17
WARRANTY CERTIFICATE		page 18

GENERAL WARNINGS

Read these instructions carefully before installation and/or use the product; keep the instructions as long as you have this product. Please find more information at the following website www.kdesign-group.com.

1. WARNINGS AND CONTRAINDICATIONS

- Before installation or use, always check that the product is intact and in perfect condition for its usage.
- People with disabilities, temporary or permanent mobility limitations, should not use this product alone as there is the possibility to squeeze parts of the body, cause injuries or entrap the head.
- Use the product only when the surface temperature is suitable for use: hot water, direct sunlight, temperature of the sauna, extreme cold, etc. may have changed the surface temperatures of the products that might cause harm to the skin or health.
- The total product weight is less than 10 kg
- All the dimensions are given in cm; the weight are in kg and lbs .

2. INTENDED USE

- The grab bars are products supplied as support; and serve as tools to compensate reduced capacity and/or reduced physical performance. Never the less, the grab bars can also be used as prevention, security and assistance.

3. INSTALLATION

- The products and accessories must be mounted and installed only by a professional installer.
- Before installing the product, make sure that there are no damaged parts.
- Where the contractor mounts the product, he should ensure that the entire area of the wall to which the mounting plate is to be fitted, is flat (no projecting joints or protruding tiles), that the structure of the wall is suitable for the installation of the product and has no weak points.
- Use the correct wall fitting screw kit, accordingly to the wall specification were intended to attach the product.

4. USE OF THE PRODUCT

- The use of the wall mounted grab bars and wall mounted lift up grab bars which are permanently attached to a wall is intended to support people when standing or changing position.
- Do not use the product if it is damaged, worn, crack or broken, if so waste dispose it.
- Before use, always make sure that the product is firmly fixed.
- Do not exceed the maximum user weight.

5. CLEANING AND MAINTENANCE

- The product does not need regular maintenance but periodically checks of its integrity and that the wall fitting are firmly attached.
- The product can be washed with warm water and mild soap.
- Do not use professional products for cleaning, disinfecting, or abrasives.
- The fold down grab bar friction regulator should be checked regularly and ensured that is tighten correctly as per instructions (v. page 14).

6. DISPOSAL

- Consult your ecological oasis in order to correctly waste-dispose in accordance with your country's legislation.

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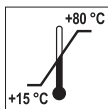
This product complies with the provisions of Council Directive 93/42/EEC.

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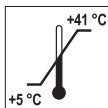
Do not use the product if it is exposed to a temperature higher than +80 °C and lower than +5 °C; it may cause damage to your skin or health.

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Do not use the product if it is exposed to a temperature higher than +41 °C and lower than +15 °C; it may cause damage to your skin or health.

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Be careful while using in order to prevent your hands or other parts of the body being crushed.

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Be careful: danger of head entrapment.

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Warning: screws and bolts for installation are not included.

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Warning: the product needs to be assembled.

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Warning: the product must be installed by a professional installer.

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Extra information downloadable at www.kdesign-group.com.

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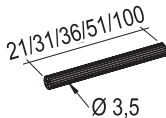
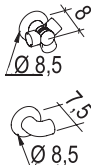
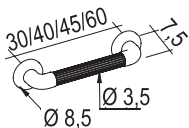
en Hazards and warnings
(*) Do not cut and use pieces
of a length more than 51 cm.

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
en It is very important to insert the
tube all the way inside the wall
flange till it reaches completely
the end of the internal slot, with-
out leaving any gap.

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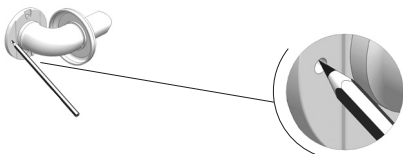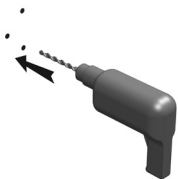
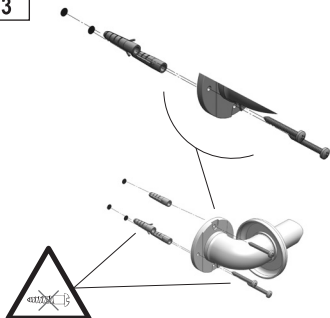
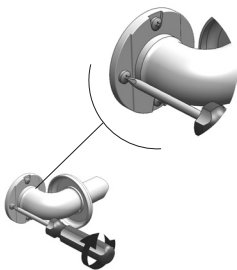
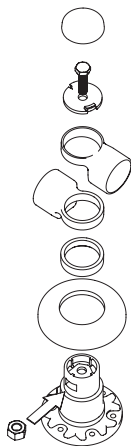
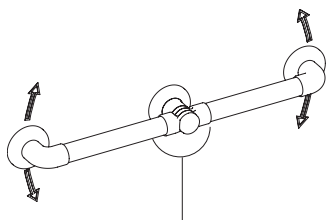
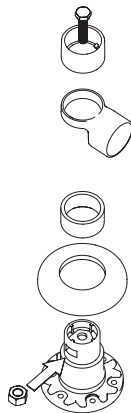
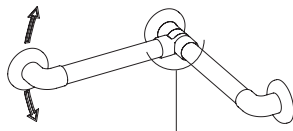
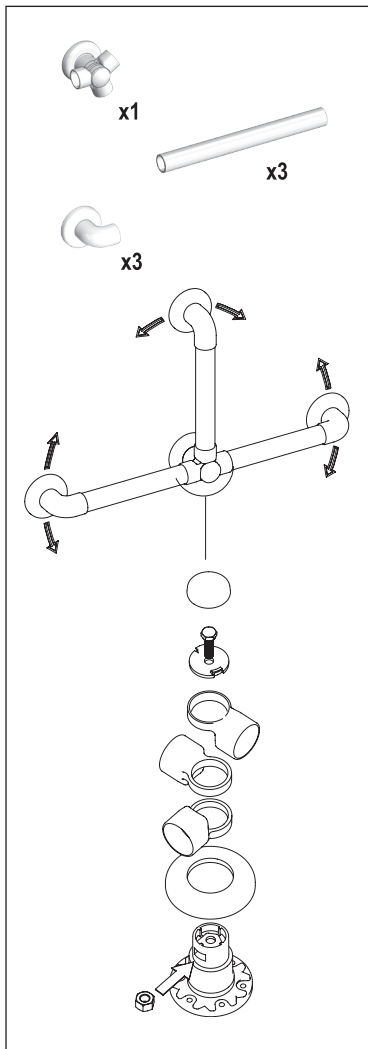
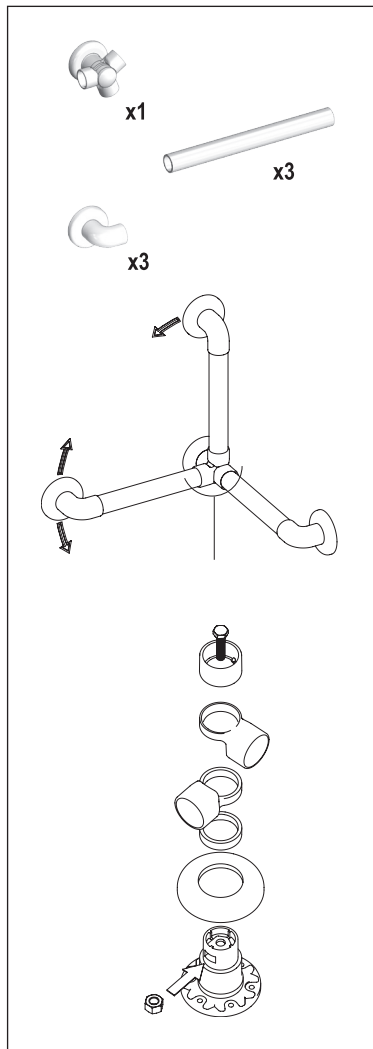
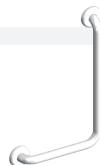
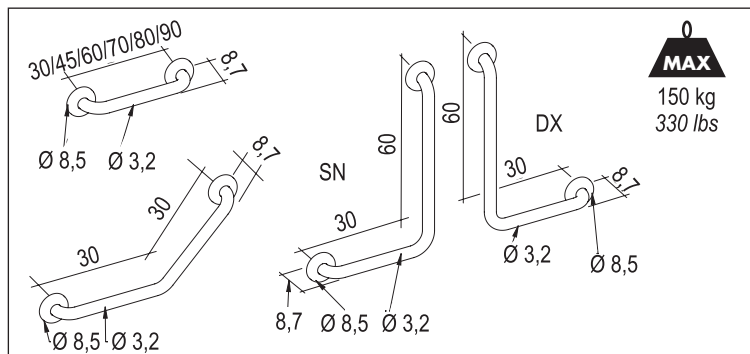
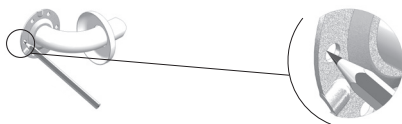
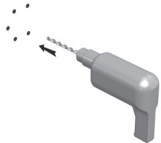
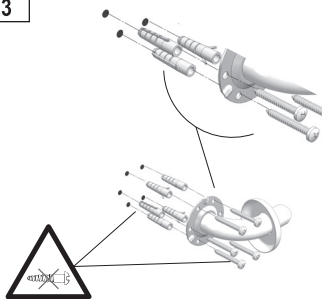
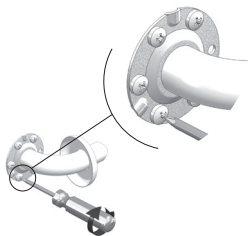
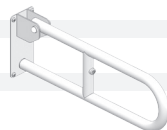
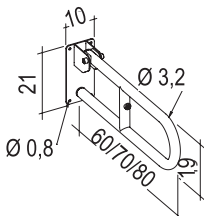
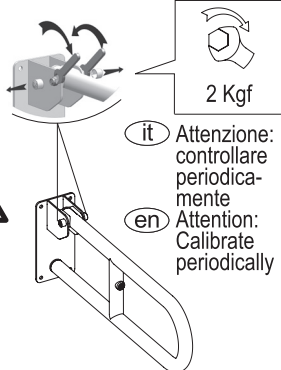
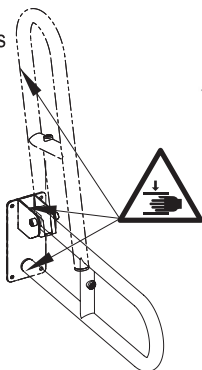
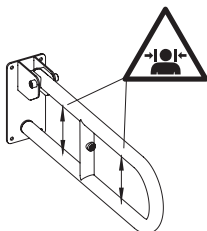
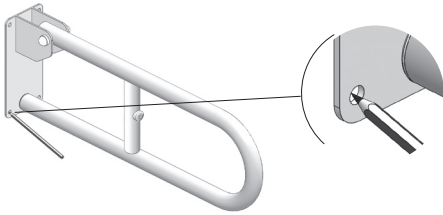
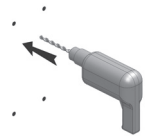
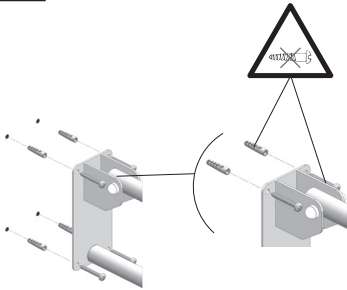
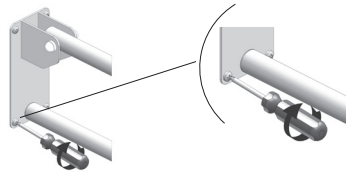
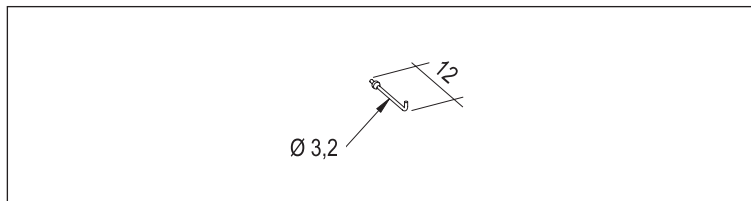
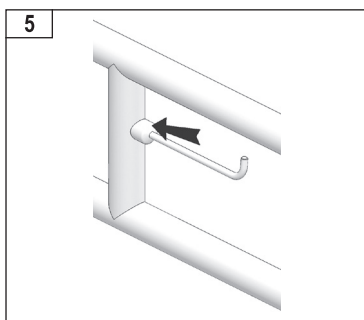
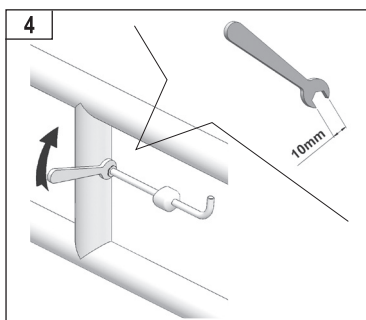
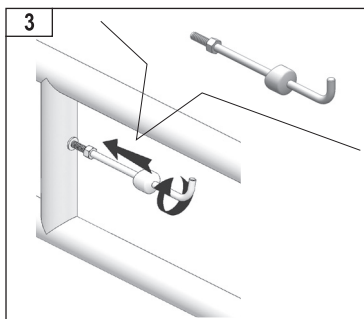
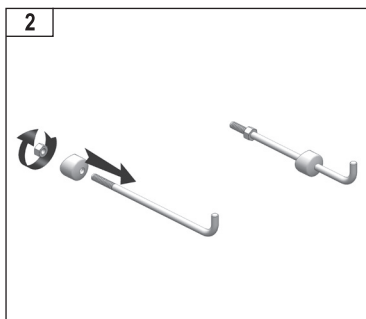
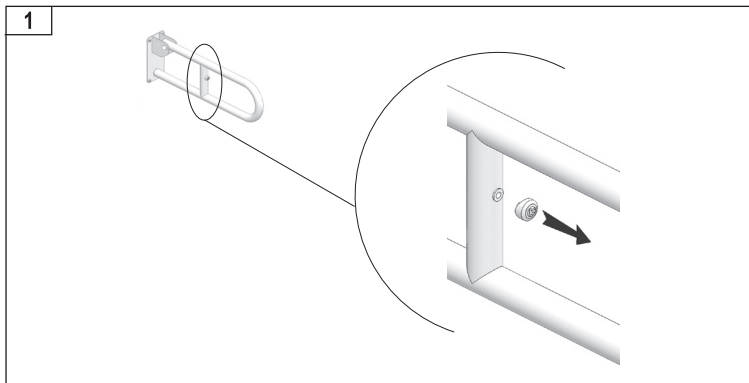
In accordance with Directive 99/44/EC of the European Parliament and of the Council, from the Code of Consumption, **K Design S.r.l.**, Galleria Goito n° 18, 19121 La Spezia, Italy, warrants its own products against manufacturing defects and against any conformity defects in accordance with the 'Instruction manual' for each single product. **The product is guaranteed for 24 (twenty four) months from the date of purchase.** To download the warranty and register your product, visit our website **www.kdesign-group.com**.

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(tel) +39 0187 1987200 - (fax) +39 0187 1987229

www.kdesign-group.com - info@kdesign-group.com

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K design S.r.l.