# GIMA

## GIMA BRUSH

MANUALE D'USO E MANUTENZIONE  
USE AND MAINTENANCE BOOK  
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN  
BETRIEBS UND WARTUNGS ANWEISUNGEN  
MANUAL DE USO Y MANTENIMIENTO  
MANUAL DE USO E MANUTENÇÃO  
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ  
دليل الإستعمال والرعاية

REF 29740

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## GIMABRUSH

Endocervical brush for cytological sampling.

The brush is made of nylon bristles, held by a metal winding. It is assembled on a plastic stem. It is used to collect cells from the uterine cervix through the endocervical channel.

The device must be used only by qualified trained personnel authorized to carry out the pap test.

Do not expose to sources of heat. Use only if the packing is not damaged. Extract the device from the package immediately before use. Before use, visually examine the device to check its integrity.

Intended use: GIMABRUSH must be used for taking uterine cervix endocervical cells.

Indications: it is indicated also when there is cervical canal stenosis.

Contra-indications: it is not suitable during pregnancy.

Directions for use: insert the speculum into the vagina to put in evidence the uterine cervix, open the package, insert the GIMABRUSH slowly into the endocervical canal, make a complete rotation, withdraw slowly, rub the removed material on the slide.

*The device IS NOT STERILE.*

*The device is SINGLE-USE.*

The product must be disposed according to the local provisions on dangerous medicines disposal.