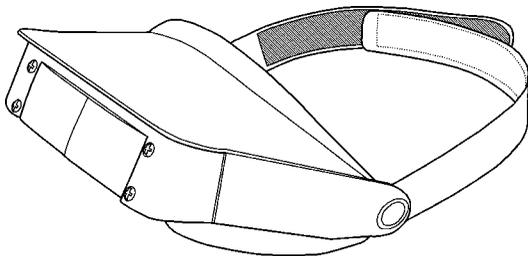
# GIMA

## HEAD LOUPE

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGS ANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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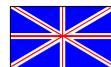
## FEATURES

Head Loupe is a special kind of device for precision magnification. Because of its features, it can be used also by people who normally wear lenses, placing it on them.

Head Loupe allows two kinds of magnification: 2,2x if you use the first system of loupes and 3,3x if you use the second system. The nominal value of the magnification is indicative and not absolute. It is also provided with a head belt in adjustable Velcro and with comfortable anti-reflection coating.

The product or its components cannot be used for purposes different from the one specified in the present manual.

## PRESCRIPTIONS

The product shall be used by qualified medical personnel only, in out-patients' department. It is not conceived for the employment in operation room.

*Do not use to watch directly the sun or other luminous sources!*

*Keep the focal distance of the loupes, in order not to provoke the blurring of the image, which can cause eye weariness.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## FUNCTIONING

At first, it is necessary to block the head loupe with the Velcro stripes of the head belt and then to adjust the inclination of the optical group. If necessary, it is possible to turn it 90° above the forehead (see picture).