carrello W-Line WSP

W-Line WSP trolley

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Dear Customer,

your choice of our appliance means, for our Company, your particular recognition of the care of our designers, in the achievement of an ergonomic, reliable and pleasant design product.

We would thank you to trust our Company and we are at your complete disposal for everything you need.

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You are kindly requested to pay particular attention to this manual, before using the appliance, as it contains important recommendations on the correct installation, use and maintenance of the product.

As a matter of fact it is the shortest way of communication between the Supplier and the User. Have a good work!

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Our trolleys are designed to support, to move and possibly power the medical devices, to be used in outpatient clinics, hospitals, Faculties of Medicine, etc.

That's why these trolleys don't have to be used for different uses from the indicated ones. Every other use has to be considered dangerous.

The trolleys in the Professional Basic and Professional Power line are considered accessories to medical devices and as such meet the requirements of Regulation 2017/745

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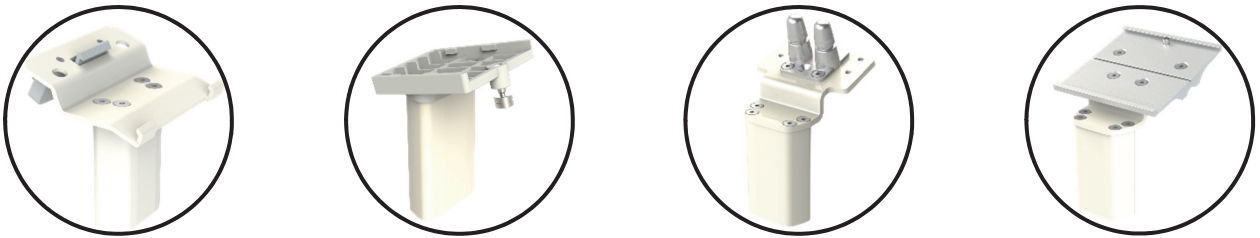
The W-LINE trolley is made up of an invariant structure consisting of the following elements:

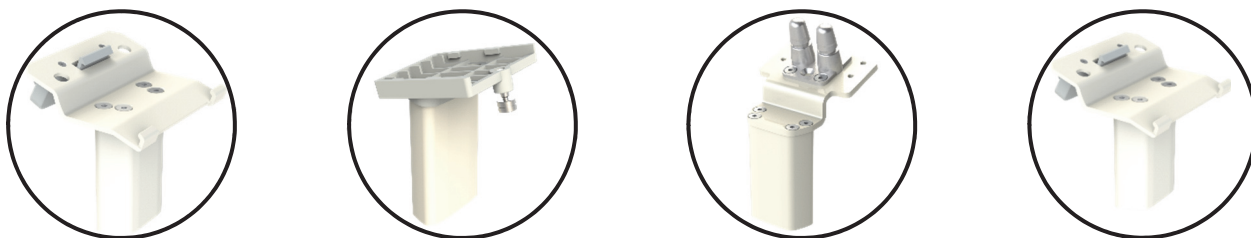
- **base :** *made of steel sheet and cover in plastic ABS*
- **columns:** *made of extruded aluminum and painted with epoxy powders. The maximum overall dimensions of the section are: 65x30mm (WSP). The height of the structural upright can be defined according to the needs of the end user.*

- **twin wheels:** The trolley is equipped with 5 twin pivoting ABS wheels; the standard configuration includes 3 wheels with locking system and two simple wheels, but on request it is also possible to have all 5 wheels with braking system. The diameter of the wheels is Ø65mm (WSP).

The following additional elements and / or accessories can be installed on the trolley:

- **adapters:** Different adapters are available according to the coupling system of the Medical Device.

The figure below shows some types currently available:

- **handle:** handle made of high-strength ABS for moving the trolley
- **basket:** accessories holder basket in welded steel rod and oven painted with epoxy powders
- **ABS basket:** In ABS plastic material

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After packaging removal, do not leave the different elements (carton boxes, plastic bags, polystyrene, etc.) in the environment and do not leave them in children hands, as they can be dangerous.

Anyway do not throw away the packaging before verifying the correct operation of the device, as in case of giving it back to the supplier, the device has to be put in its own original packaging. All the different elements of the packaging, have to be handled following the local Directives regarding the disposal of the waste.

If it is necessary to dispose the trolley, at the end of its useful life, or for other reasons, it is important to consider that the trolley has to be disposed following the directions of the Directives that regulate the electromechanical/electrical devices, for which the same Directives require to reutilization/recycle the parts of the device, to avoid damages to the environment and consequently for the people health.

The following warning is valid for cart powered with insulation transformers

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If the trolley is supplied disassembled, the assembly instructions are shown in the specific manual that you can find in the same box where all the trolley's parts are. Please follow the instructions.

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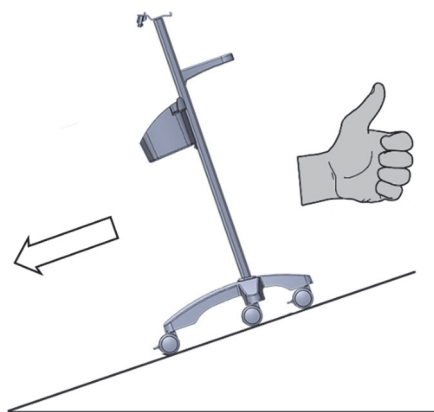
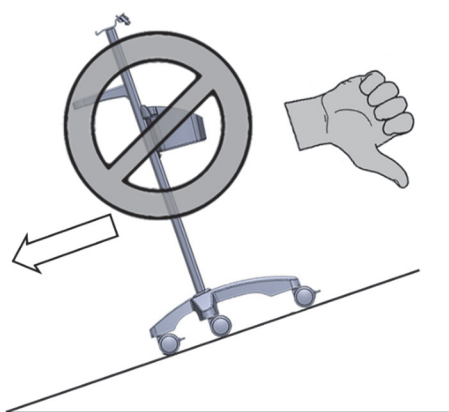
*The **normal use position** of the trolley requires that the braking mechanism on the wheels is engaged. Do not attempt to move the trolley when the braking mechanism is engaged. Before moving the trolley, release the braking mechanism on the wheels (generally the two front ones) and only then move the trolley.*

Engage the braking mechanism on the wheels again, when the new position of use or parking has been reached.

*The trolley has been designed to be stable on slopes up to a **maximum of 5°** with braked wheels by means of the locking device provided on the wheels.*

The trolley for slopes of more than 5° should not be left stationary unattended, but an operator must be present to avoid the risk of tipping over.

Do not load a device weighing more than 15kg onto the support at the top of the trolley. Do not load items totally heavier than 5kg inside the basket

It is forbidden to climb onto the base of the trolley or other projecting parts of it. This would lead to the risk of falls resulting in serious and, in extreme cases, disabling injuries.

It is forbidden to sit on the trolley shelf or other projecting parts of the trolley. This would lead to the risk of falls resulting in serious and, in extreme cases, disabling injuries.

8.a TRANSPORTATION POSITION

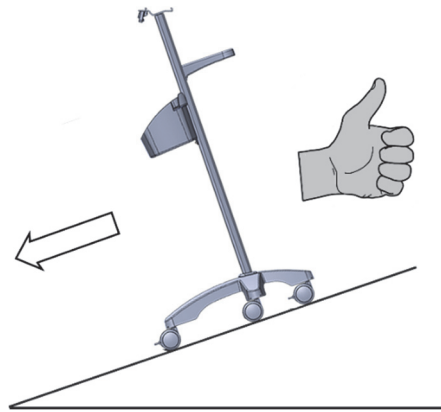
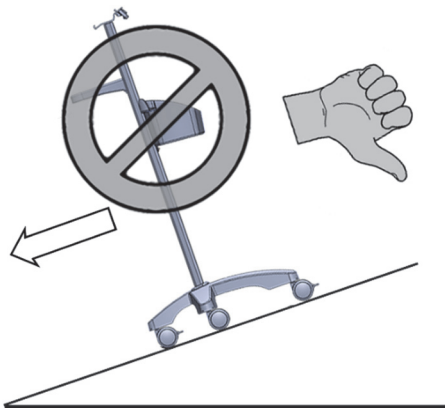
When it is necessary to move the trolley up or down on a slope, move it slowly exclusively as shown in the figure below, keeping it with the handle firmly to avoid the risk of instability and the risk of accidents.

Make sure that there are no obstacles, even small ones, when moving the trolley, in order not to damage the trolley and the equipment installed on it.

WARNING: DEVICES DISPOSAL

Pay particular attention to the arrangement of the equipment on the shelves, if present, taking care that the heavier devices shall be placed, when possible, on the lower shelves, in order to improve the stability of the trolley.

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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With regard to storage requirements, the following instructions must be strictly adhered to:

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The trolley - if correctly used for its "intended use" only and the instructions reported in this manual are strictly followed - does not need any maintenance being particularly robust as the materials used in its construction are aluminum and steel.

Nevertheless, we suggest the following:

- to prevent damage to the trolley resulting from its improper use, it is suggested to verify, every four months, the state of the reciprocal links between the various components. If necessary, tighten all screws / bolts tightly: experienced personnel using specific tools for each type of screws should do this operation.*

- in the event of damage to the trolley, for any accidental event (e.g. hit during movement), the trolley must be subjected to exceptional maintenance at the Manufacturer's premises where it is possible to detect the type of failure and then carry out the repair and the final testing.*

In case of repair made by the user or on his behalf by a specialized manufacturer of his trust, the latter must contact our Technical Department and ask for appropriate information, suggestions and permission to perform the intervention.

No liability can be charged to the Manufacturer for damage resulting from repairs made by the user without the prior authorization of the Manufacturer.

As regards the cleaning, make sure that the device installed on the trolley are disconnected from the mains. The cleaning must be done with a damp cloth and with non-corrosive products. Do not use any solvents.

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In case a return to Bieffe italia is necessary for maintenance purposes, it is mandatory to carry out a proper cleaning/disinfection procedure of the product before shipment, in order to avoid bacterial/viral contamination of the personnel taking charge of the device.

- CLEANING: Neutral or alcohol-based detergents must be used whose composition is such that they do not affect surfaces, are non-toxic, do not give off excessive fumes or in any way pose a risk to operators, and are easily rinsed and biodegradable.

- DISINFECTION: A large number of disinfectants are used in healthcare environments, including chlorine and chlorine compounds, glutaraldehyde, hydrogen peroxide, iodophores, etc. These disinfectants are not interchangeable with each other and it is the responsibility of hospital staff, depending on the level of risk of contamination and the environment in which the device was used, to identify the most appropriate disinfectant.

Damage to Bieffe italia's products that may result to painted or anodised surfaces due to improper use of products is therefore not attributable to the Company.

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The manufacturer guarantees the good quality and realization of its products by undertaking to repair or replace for free, during the below specified warranty period, as quickly as possible, those parts that – for the bad quality of the material or for manufacturing defects – are defective, provided that this does not depend on natural wear, failures caused by inexperience of the Customer, excessive load beyond the contractual limits, unauthorized interventions, tampering performed by the Customer or by others, by chance circumstances or for reasons of force majeure.

*The warranty is valid **12 months** from delivery even though the trolley is not, for whatever reason, placed in service.*

For components not manufactured by the manufacturer, the manufacturers' guarantees apply.

*The guarantee is valid **ONLY** if the trolley is used according to its intended use.*

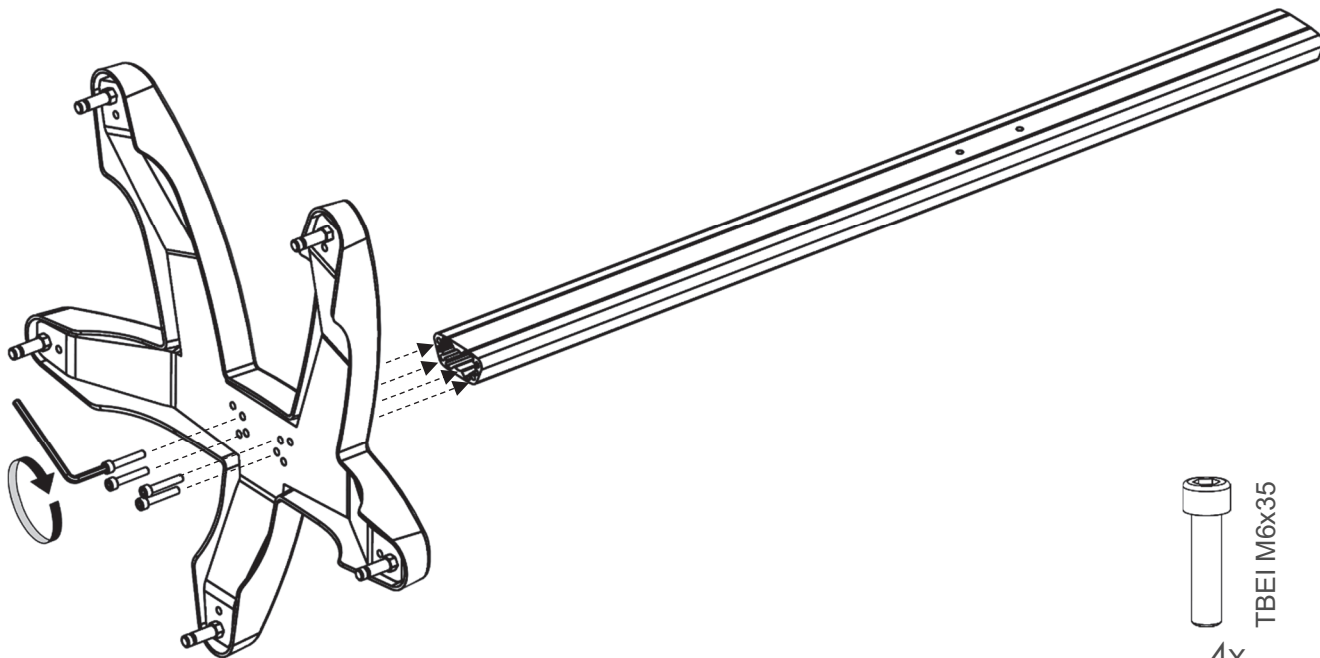
All transportations related to the operations performed under warranty take place at the Customer's expenses, risk and peril.

Under no circumstances shall the warranty period extend beyond the date of the original warranty period.

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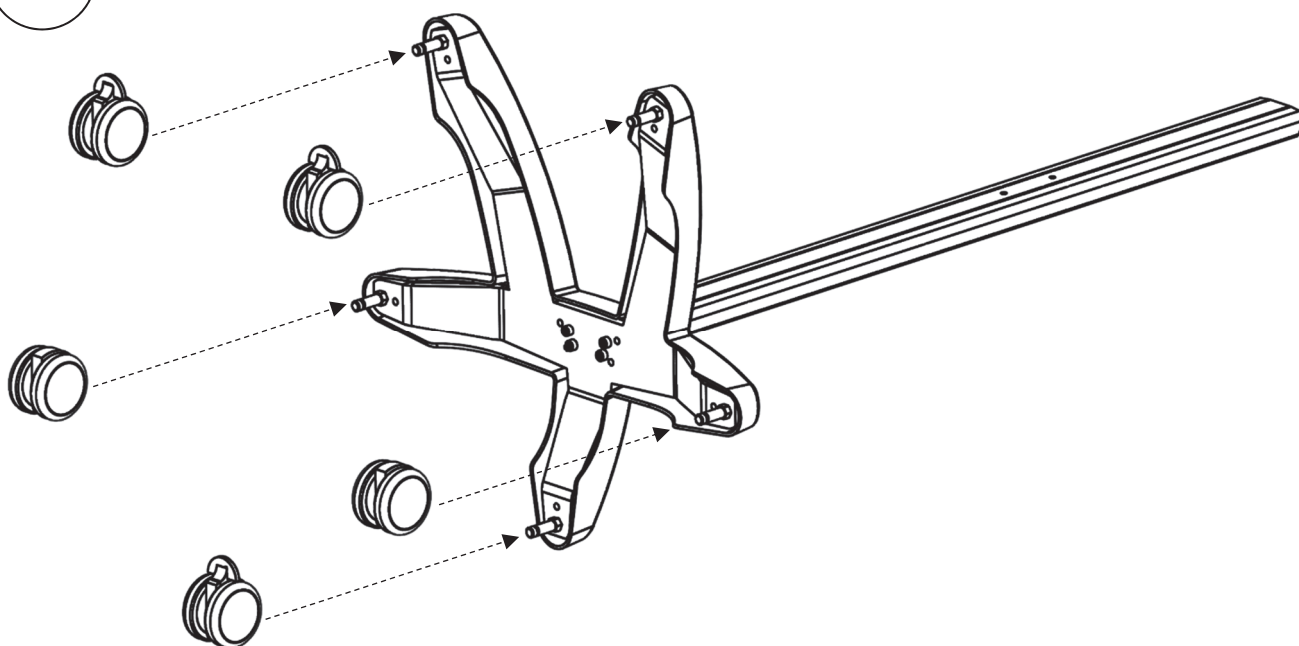
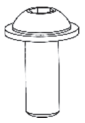
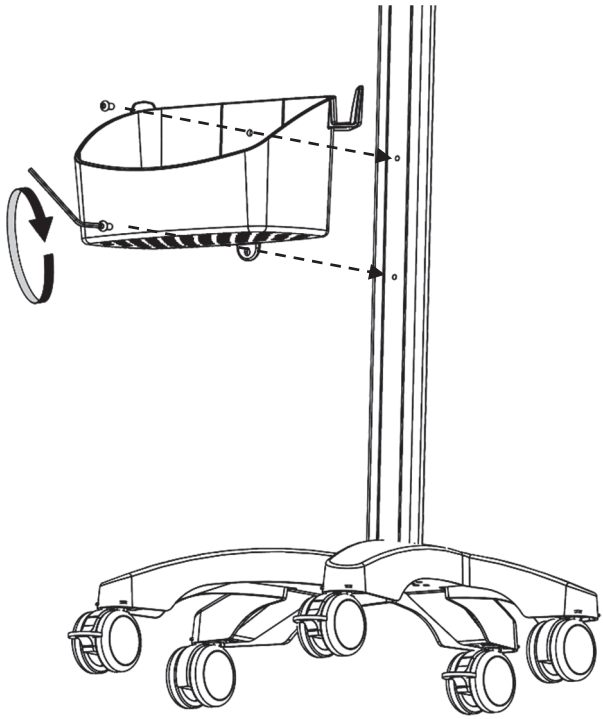
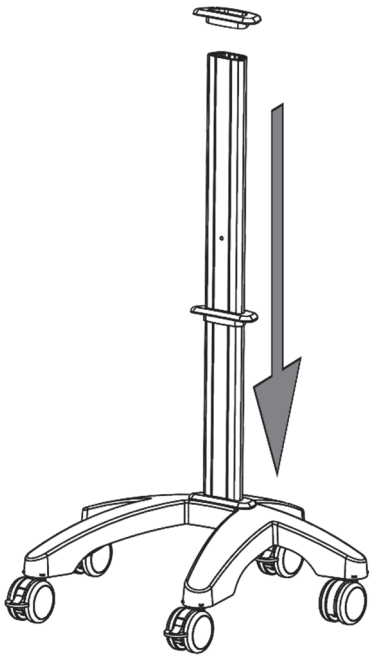
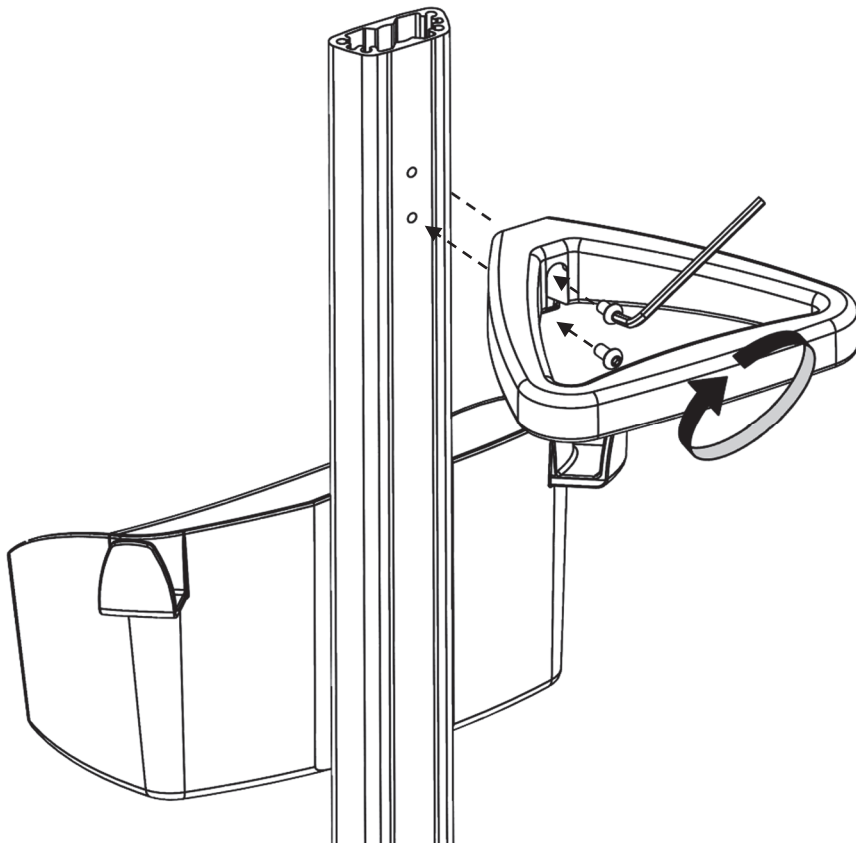
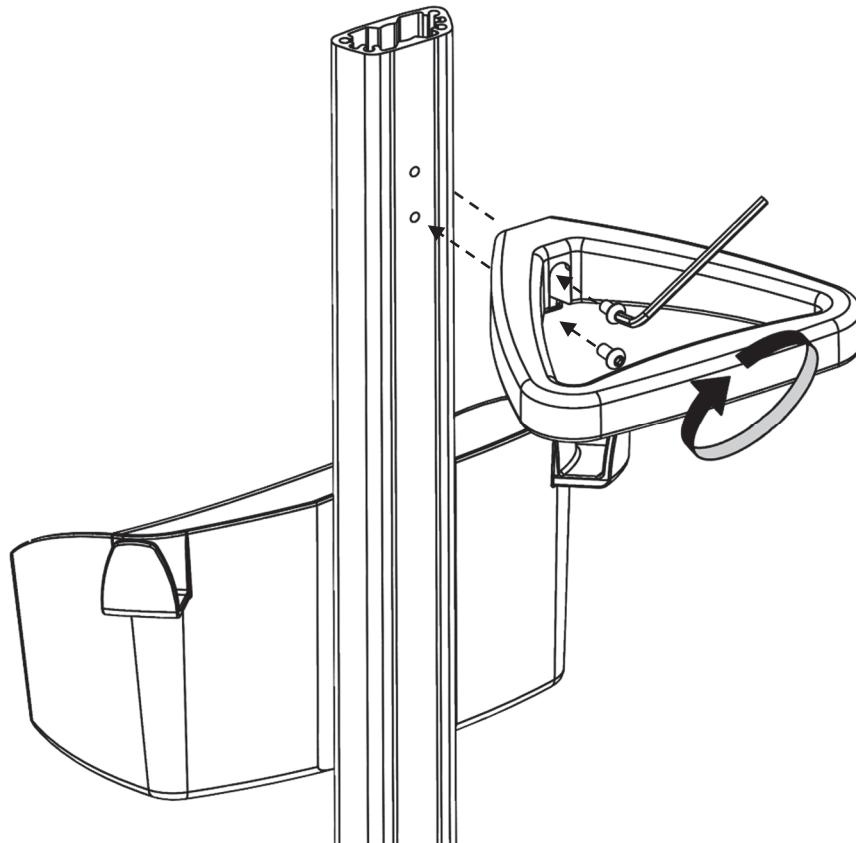
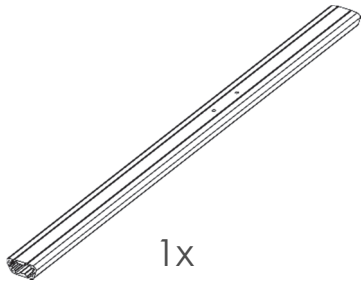
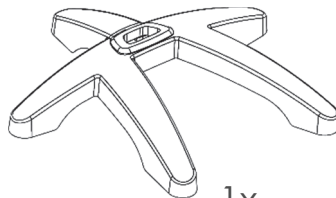
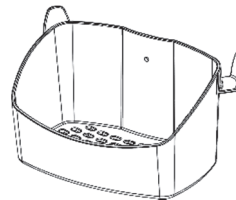
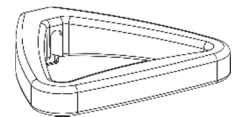
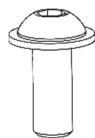
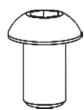
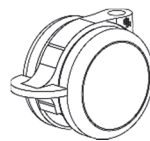
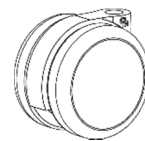
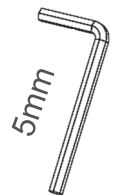
BIEFFE ITALIA s.r.l.

Actor ID/SRN: IT-MF-000035325

As manufacturer of the products underneath listed, declare under its own responsibility that the product/s:

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BASIC UDI-DI: 805373671WLSPP

is/are in conformity

with the essential requirements of the **Regulation (EU) 2017/745** as **Class I** medical devices according to Annex VIII, Chapter III, Rule 1 (one).

Intended use: The trolleys are intended for supporting, moving and possibly powering medical equipment.

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BIEFFE ITALIA S.r.l.

Dr. Flavio Ferrazzano

CEO

 **BIEFFE ITALIA S.r.l. weiko®**
Head Office: Viale Lincoln, 178 - 81100 Caserta (Italy)
Factory: Zona ASI Aversa Nord – c/o Consorzio Siné
81032 Carinaro (CE)
Tel: +39 081 8654662 Fax: +39 081 8654723
info@weiko.com - www.weiko.com