### ITALIANO

#### FORBICI PER MATERIALI MEDICI

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### ENGLISH

#### SCISSORS FOR MEDICAL MATERIALS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

#### Devices

Scissors for medical materials (Non Sterile, Stainless steel).

#### Product Specification

The product is made from high-grade stainless steel for the production of medical instruments according to ISO and ASTM standards.

#### Pre-cleaning

Remove gross deposits using a laparotomy sponge and sterile water during the procedure.

#### Manual cleaning

To minimize the risk to personnel undertaking manual cleaning, splashing and the creation of spray must be avoided at all times. Staff carrying out manual cleaning should wear PPE at all times.

#### Devices should be:

- 1) Cleaned using a non-linting cloth, impregnated with the appropriate detergent solution, followed by a clean, damp, non-linting cloth; and then
- 2) Dried using another clean, non-linting cloth. Alcohol-impregnated wipes may be used following a manual cleaning process.

**Detergents:** Use specific detergents: do not use common dish detergent.

#### Disinfection

GIMA powder / paper coated instruments are not autoclavable but they can be cleaned/disinfected with disinfectants like Endo Star (high level instruments disinfectant) or with any other better disinfectant/sterilizing solution.

#### Point of use handling

May only be used for the purpose of which they are designed, by adequately qualified personal only.

#### Storage & maintenance

Products should be stored away from direct sunlight and water, in a safe, dry and cool environment.

#### Warning

Don't use the rusty instrument. Wash the hands with anti-bacterial soap or use approved hand sanitizer before use. For professional use only - disinfect before use.

#### Conclusion

This Medical Devices classified as class-I as per Medical Device Regulation (MDR) 2017/745.

#### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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**Gima S.p.A.**