GIMA

PROFESSIONAL MEDICAL PRODUCTS

CATETERE RETTALE SMALTIBILE
DISPOSABLE RECTAL CATHETER
CATHÉTER RECTAL JETABLE
CATÉTER RECTAL DESECHABLE
CATETER RETAL DESCARTÁVEL
REKTALKATHETER FÜR DEN EINMALGEBRAUCH
ΚΑΘΗΤΗΡΑΣ ΑΕΡΙΩΝ ΜΙΑΣ ΧΡΗΣΗΣ
JEDNORAZOWY CEWNIK DOODBYTNICZY
REKTÁLNÍ KATÉTR NA JEDNO POUŽITÍ
REKTALKATETER FÖR ENGÅNGSBRUK
KERTAKÄYTTÖINEN REKTAALIKATETRI
REKTALNI KATETER ZA ENKRATNO UPORABO
REKTÁLNY KATÉTER NA JEDNO POUŽITIE
CATETER RECTAL DE UNICĂ FOLOSINȚĂ
RECTALE KATHETER VOOR EENMALIG GEBRUIK
JEDNOKRATNI REKTALNI KATETER
EGYSZER HASZNÁLATOS VÉGBÉL KATÉTER
ENGANGS REKTALKATETER
PEKTAΛEH KATETŪP ZA EΔHOKPATHA YΠOTPEBA
VIENKARTINIS TIESIOSIOS ŽARNOS KATETERIS
VIENREIZĒJĀS LIETOŠANAS TAISNĀS ZARNAS KATETRS
ÜHEKORDNE REKTAALNE KATEETER

REF F5301.22 (Gima 22169) - F5301.24 (Gima 22170) - F5301.26 (Gima 22171)
F5301.28 (Gima 22172) - F5301.30 (Gima 22173)

Jiangsu Suyun Medical Materials Co., Ltd. No.18
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
Made in China

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße, 80, 20537, Hamburg, Germany

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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ENGLISH**Operation Instructions****Product usage**

Insert it into rectum to clean it, derive the feces, or inject medicine.

Usage Method

Prepare necessary instruments such as rectal catheter and irrigation syringes to the patient's bedside, let the patient stay in supine or left lying position. Take off the peelable package, and pick up the rectal catheter and connect it to other instrument like the irrigation syringe firmly, lubricate the front end of the rectal catheter, and insert it into the rectum for normal use. After use, the products shall be collected, transported, stored and disposed to ensure that the products are harmless to human health and the environment. Or the product can be disposed in accordance with the local medical regulations for waste disposal.

Caution

Single use only. Do not re-sterilize. Do not use if package is opened or damaged.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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