Jelco IntuitIV Safety IV Catheter™ (Straight)

EN INSTRUCTIONS FOR USE	nl GEBRUIKSAANWIJZING
fr MODE D'EMPLOI	sv BRUKSANVISNING
de GEBAUICHSANWEISUNG	da BRUGSANVISNING
it ISTRUZIONI PER L'USO	no BRUKSANVISNING
es INSTRUCCIONES DE USO	fi KÄYTTÖOHJEET
pt INSTRUÇÕES DE UTILIZAÇÃO	el ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

Manufacturer:
Smiths Medical International Ltd.
St. Crispin Way, Haslingden,
Rossendale, Lancashire, BB4 4PW, UK.

Tel: +44 (0)1706 233800
Distributed by:
Smiths Medical ASD, Inc.

201 West Queen St,
Southington, CT 06489, USA.
Tel: +1 860 621 9111

Australian Representative:
Smiths Medical Australia Pty. Ltd.
61 Brandt Street, Eight Mile Plains,
Brisbane, QLD 4113, Australia.
Tel: +61 (0)7 3340 1300

www.smiths-medical.com
smiths medical

CN 004/008/015 REV.001 06/12

en English	fr Français	de Deutsch	it Italiano	es Español	pt Português	nl Nederlands	sv Svenska	da Dansk	no Norsk	fi Suomi	el Ελληνικά
Caution	Précaution	Achtung	Cautela	Aviso	Aviso	Let op	Försiktighetstidård	Forsigtig	Forsiktig	Varoitus	Προσοχή
Do not re-use	Ne pas réutiliser	Nicht zur Wiederverwendung	Non riutilizzare	No reutilizar	Não reutilizar	Niet opnieuw gebruiken	Får ikke återanvändas	Må ikke genanvendes	Til engangsbruk	Ei saa käyttää uudelleen	Μην επανοχησούτετε
Do not re-sterilize	Ne pas restériliser	Nicht resterilisieren	Non risterilizzare	No reesterilizar	Não reesterilizar	Niet opnieuw steriliseren	Får ej omstertiliseras	Må ikke resteriliseres	Skal ikke resteriliseres	Älă sterilă uudeleen	Μην επαναστερίζετε
REF	Catalogue Number	Numéro de catalogue	Bestellnummer	Número de catálogo	Número de catálogo	Catalogusnummer	Katalognummer	Katalognummer	Katalognummer	Katalognumero	Αριθμός καταλόγου
LOT	Batch code	Numéro de lot	Chargenbezeichnung	Codice del lotto	Código de lote	Batchcode	Batchkod	Batchkode	Kode for parti	Eräkoodi	Κωδικός παρτίδας
Date of manufacture	Date of fabrication	Herstellungsdatum	Data di fabbricazione	Fecha de fabricación	Data de fabrico	Fabricagedatum	Tillverkningsdatum	Fremstillingdato	Produksjonsdato	Valmistuspäivä	Αριθμός παρτίδας
Use by	Utiliser avant le	Verwendbar bis	Utilizzare entro	Fecha de caducidad	Utilizar até	Te gebruiken voor	Använd före	Anvendes inden	Brukes innen	Käytettävä ennen	Ημερομηνία λήξης
Manufacturer	Fabricant	Hersteller	Fabbricante	Fabricante	Fabrikant	Tillverkarkare	Fabrikant	Produsent	Valmistaja	Κατασκευατής	
Does not contain Natural Rubber Latex	Ne contient pas de latex de caoutchouc naturel	Enthält keinen Latex aus Naturkautschuk	Non contiene lattice di gomma naturale	No contiene latex de caucho natural	Não contém latex de borracha natural	Bevat geen natuurlijke rubberlatex	Innehåller inte naturligt latexgummi	Inneholder ikke naturlig gummitax	Ei sisällä luonnonkumi lateksia	Περιέχει φυσικό ελαστικό λάτεξ ή μηχανικά φυσικού ελαστικού λάτεξ	
Quantity	Quantité	Menge	Quantità	Cantidad	Quantidade	Aantal	Kvantitet	Antal	Määrä	Ποσότητα	
STERILE	STERILE	STERILE	STERILE	STERILE	STERILE	STERILE	STERILISERAD	STERILISERAD	STERILISERAD	STERILITET	Αποτελεσματικό με χρήση οξειδίου του αιθανένιου
Fluid path components non-pyrogenic	Eléments de trajet du liquide non pyrogéniques	Komponenten des Flüssigkeitssystems nicht pyrogen	Componentes del percorso del fluido arioprogenici	Componentes de la vía de pasaje del fluido arioprogenicos	Onderdelen vloeistofpad niet-pyrogen	Vätskeledarkomponenter är ej pyrogen	Väskedebankomponenter ikke-pyrogen	Väskedebankomponenter ikke-pyrogen	Nestapolun osat ei-pyrogenisia	Εξαιρετικά δέλτας υγρού μη πυρογένειας	
Temperature Limitation	Limite de température	Temperaturbereich	Limite di temperatura	Limite de temperatura	Temperaturlimiet	Temperaturgränsn	Temperaturbegrensning	Temperaturgrense	Lämpötilarajitus	Περιορισμός θερμοκρασίας	
This way up	Haut	Oben	Alto	Este lado hacia arriba	Este lado para cima	Deze kant boven	Denna sida upp	Denne vej op	Denne siden opp	Tämä puoli ylös	Επάνω πλευρά
Fragile, handle with care	Fragile : manipuler avec soin	Vorsicht, zerbrechlich	Fragile, maneggiare con cura	Fragíl, manipulear con cuidado	Breekbaar, voorzichtig behandelen	Bräcklig, hanteras varsamt	Forsiktig	Sikker, håndteres forsiktig	Helytösiä särkyvä, käsittelyä varovasti	Εύφρων, ο γεράς πρέπει να γίνεται με προσοχή	
Do not use if package is damaged	Ne pas utiliser si l'emballage est endommagé	Inhalt bei beschädigter Verpackung nicht verwenden	Non utilizzare se la confezione è danneggiata	No utilizar si el envase está dañado	Niet gebruiken indien de verpakking beschadigd is	Får ikke användas om förpackningen är skadad	Må ikke brukes hvis emballagen er beskadiget	Ei saa käyttää, jos pakkaus on vahingotiltunut	Μην το χρησιμοποιείται εάν η συσκευασία έχει υποστεί ζημιά		
Keep dry	Conserver au sec	Trocken aufzuhören	Tener seco	Manter seco	Droog houden	Forvaras torrt	Oppbevares tort	Pidettävä kuivana	Να διατηρεται στη γηρά	Suojaavata kuivana	
Keep away from sunlight	Conserver à l'abri de la lumière du soleil	Vor Sonnenlicht schützen	Tener al riparo dalla luce solare	Manter afastado da luz solar	Niet blootstellen aan zonlicht	Skyddas för sollys	Oppbevares utenfor direkte sollys	Suojaavata aurinkovalta	Διατηρεται μακρι απο τη ηλιακη φωτα		

English

Jelco IntuitIV Safety IV Catheter™
(Straight)


INSTRUCTIONS FOR USE

These instructions apply to the following Jelco® products:
7130 to 7138 & 7188 Jelco IntuitIV Safety IV Catheter™ FEP
7260 to 7269 & 7288 Jelco IntuitIV Safety IV Catheter™ PUR

These instructions contain important information for safe use of the product. Read the entire contents of these Instructions For Use, including Warnings and Cautions, before using this product. Failure to properly follow warnings, cautions and instructions could result in death or serious injury to the patient and/or clinician.

NOTE: DISTRIBUTE THIS INSERT SHEET TO ALL PRODUCT LOCATIONS.

1. DESCRIPTION:
Each Jelco IntuitIV Safety IV Catheter™ consists of an introducer needle and an integral needle tip-protector. Key parts are: (1) needle bevel, (2) needle, (3) catheter, (4) catheter hub, (5) tip-protector, (6) fluid chamber, (7) flash plug assembly, (8) sheath, (9) needle hub housing.

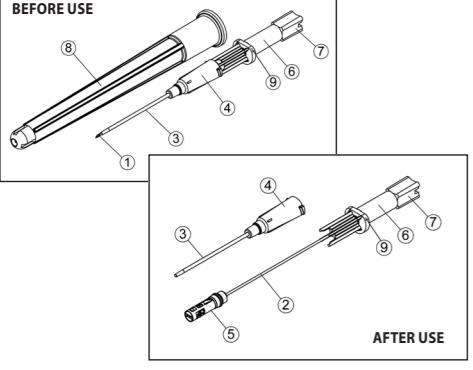CATHETER GAUGE AND REFERENCE CHART

Gauge	14G	16G	18G	20G	22G	24G
Colour	Orange	Grey	Green	Pink	Blue	Yellow

This device is intended for single use only and is provided sterile and non-pyrogenic. The materials used to manufacture this radiopaque IV catheter do not contain latex and are PVC and DEHP-free.

BEFORE USE
1. DESCRIPTION:
Each Jelco IntuitIV Safety IV Catheter™ consists of an introducer needle and an integral needle tip-protector. Key parts are: (1) needle bevel, (2) needle, (3) catheter, (4) catheter hub, (5) tip-protector, (6) fluid chamber, (7) flash plug assembly, (8) sheath, (9) needle hub housing.

INSTRUCTIONS FOR USE
These instructions contain important information for safe use of the product. Read the entire contents of these Instructions For Use, including Warnings and Cautions, before using this product. Failure to properly follow warnings, cautions and instructions could result in death or serious injury to the patient and/or clinician.

NOTE: DISTRIBUTE THIS INSERT SHEET TO ALL PRODUCT LOCATIONS.

2. INDICATIONS:
A properly placed Jelco IntuitIV Safety IV Catheter™ provides access to a vein or artery to sample blood, monitor blood pressure, or administer fluids.

2.1 The tip-protector that locks over the needle as the needle is withdrawn helps reduce the risk of accidental needlesticks.

2.2 These catheters may be used for any patient population with consideration given to patient size, appropriateness for the solution being infused and duration of the therapy.

2.4 To 14 to 24 gauge catheters may be used with power injectors up to 300 psi.

2.5 Ensure stabilization of catheter to the patient. Improper stabilization may lead to loss of vascular access.

2.6 Follow current institutional policies and procedures for catheter insertion, maintenance and removal.

2.7 The catheter is shorter than the introducer needle. Therefore, blood flashback may occur before the catheter tip is fully in the blood vessel. If necessary, slightly advance the catheter and needle together to assure full catheter entry into the blood vessel lumen. To avoid inadvertently puncturing the posterior wall of the vessel, lower the needle until it is parallel to the skin.

2.8 Ensure fluid administration/hub connection is secure in order to prevent leaking.

2.9 Ensure stabilization of catheter to the patient. Improper stabilization may lead to loss of vascular access.

2.10 Connect IV administration line and dress site per institutional policy.

2.11 Immediately discard unit into a puncture resistant, leak-proof, disposable sharps container.

2.12 Do not manipulate the tip-protector assembly before or after use.

2.13 Select and prepare site per institutional policy. Apply tourniquet.

2.14 Remove sheath in straight outward motion and inspect device. Ensure that the colour-coded catheter hub is fully seated, and verify the needle bevel is oriented in the "up" position.

2.15 Anchor the device with gentle skin traction and insert the needle into the skin and blood vessel at an appropriate angle.

2.16 Blood flashback into the flash chamber confirms entry into the blood vessel.

2.17 Decrease angle and insert device slightly to assure catheter entry into the blood vessel.

2.18 Remove tourniquet.

2.19 Before removing needle, apply digital pressure to the blood vessel distal to the catheter tip and secure the catheter hub. Remove the needle by pulling straight back. The tip-protector will automatically engage over the needle tip.

2.20 Note: A slight resistance will occur as the protected needle is removed from the hub.

2.21 Never reinsert the introducer needle inside the catheter as it may sever its tip.

2.22 To avoid potential of an embolus:

• DO NOT CUT THE CATHETER or use sharp instruments near the catheter.

• DO NOT BEND THE NEEDLE during insertion, threading, or removal of the catheter assembly.

• If venipuncture is not successful, engage the safety mechanism and discard both the needle and the catheter.

• In the unlikely event that the device does not lock out; dispose of the sharp immediately into an approved sharps container.

• Needles are not intended to be used with a guidewire - use will prevent activation of safety tip-protector.

• Do not manipulate the tip-protector assembly before or after use.

• Do not use if package is damaged.

• Do not use if catheter is bent.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not parallel to the skin.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

• Do not use if catheter hub is not seated correctly.

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