**JELCO**

# Jelco IntuitIV Safety IV Catheter™ (Side Port)

<b>en</b> INSTRUCTIONS FOR USE	<b>nl</b> GEBRUIKSAANWIJZING
<b>fr</b> MODE D'EMPLOI	<b>sv</b> BRUKSANVISNING
<b>de</b> GEBRAUCHSANWEISUNG	<b>da</b> BRUGSANVISNING
<b>it</b> ISTRUZIONI PER L'USO	<b>no</b> BRUKSANVISNING
<b>es</b> INSTRUCCIONES DE USO	<b>fi</b> KÄYTTÖOHJEET
<b>pt</b> INSTRUÇÕES DE UTILIZAÇÃO	<b>el</b> ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

**CE 0086**

Manufacturer: **Smiths Medical International Ltd.**, St. Crispin Way, Haslingden, Rossendale, Lancashire, BB8 4PW, UK. Tel: +44 (0)1706 233800

Distributed by: **Smiths Medical ASD, Inc.**, 201 West Queen St, Southington, CT 06489, USA. Tel: +1 860 621 9111

Australian Representative: **Smiths Medical Australasia Pty. Ltd.**, 61 Brand Street, Eight Mile Plains, Brisbane, QLD 4113, Australia. Tel: +61 (0)7 3340 1300

www.smiths-medical.com **smiths medical**

CCN 004/008/014 - REV.001 06/12

**English**

## Jelco IntuitIV Safety IV Catheter™ (Side Port)


**INSTRUCTIONS FOR USE**

These instructions apply to use of the following Jelco™ products: 7114 to 7122 Jelco IntuitIV Safety IV Catheter™ FEP 14G to 22G 7214 to 7222 Jelco IntuitIV Safety IV Catheter™ PUR 14G to 22G

These instructions contain important information for safe use of the product. Read the entire contents of these Instructions For Use, including Warnings and Cautions, before using this product. Failure to properly follow warnings, cautions and instructions could result in death or serious injury to the patient and/or clinician.

**NOTE: DISTRIBUTE THIS INSERT SHEET TO ALL PRODUCT LOCATIONS.**

**1. DESCRIPTION:** Each Jelco IntuitIV Safety IV Catheter™ consists of an introducer needle with an integral tip-protector. Key parts are (1) needle bevel, (2) needle, (3) catheter, (4) catheter hub, (5) tip-protector, (6) flush chamber, (7) flush plug assembly, (8) sheath, (9) snap cap, (10) thumb tab, and (11) Luer lock cap.

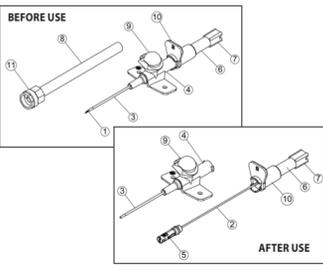CATHETER GAUGE AND REFERENCE CHART	14G	16G	18G	20G	22G
Colour	Orange	Grey	Green	Pink	Blue

This device is intended for single use only and is provided sterile and non-pyrogenic. The materials used to manufacture this catheter do not contain latex and are PVC and DEHP free.

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