GIMA

PROFESSIONAL MEDICAL PRODUCTS

GONIOMETRO DIGIT 1 CON DESIGN PER LE DITA DIGIT 1 FINGER DESIGN GONIOMETER GONIOMÈTRE NUMÉRIQUE POUR DOIGT DIGIT 1 GONIÓMETRO DE DEDOS DIGIT 1

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GB - INSTRUCTIONS OF USE

A finger goniometer measures range-of-motion (ROM) of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes. Unique design allows measurements to be taken by the therapist using only one hand; this frees up the therapist's other hand to stabilize the patient during measurement or to record the measurement; the standard finger goniometer requires two hands to make a finger ROM measurement. 6" Stainless steel finger goniometer has head with opposing measures 110° flexion through 40° hyper-extension in 5° increments. Linear scale on arm measures in inches and centimeters and has 10cm / 4" scale.

INDICATION OF USE & PRECAUTIONS

Those devices are designed for professional use by qualified operators with necessary skills.

The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion.

Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content.

This product is intended to provide easy-to-read indicative measurements.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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