# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**TUBI SILICONE**  
**SILICONE TUBE**  
**TUYAU EN SILICONE**  
**TUBO DE SILICONA**  
**TUBO DE SILICONE**  
**ΣΩΜΑΤΑΣ ΣΙΑΙΚΟΝΗΣ**  
**RURY SILIKONOWE**  
**FURTUNURILE DE SILICON**  
**SILIKÓNOVÉ HADICE**  
**SILIKONSKE CIJEVI**  
**SILIKONA CAURULES**  
**SILIKONIPUTKET**

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Silicone tubes for external medical use are intended to be used for the collection of substances that are not intended to be reinfused into the body. Its typical employment is the liquids or gas conduction, but only and exclusively for EXTERNAL use.

### Technical features

Raw material: HCR,  
 Colour: Transparent  
 Hardness: 60° ±5% Shore  
 Density: 1.17 g/ccm  
 Extension: 490/530%  
 Tensile strength: 34-35 N/mm

- In addition to what is indicated in the technical features, check compatibility with the liquids used and any deterioration over time.
- Check that cutting the tube does not generate sharp edges or flaps that could cause injuries.
- Check the tightness of the tube
- The dimensions of the tube (internal diameter, external diameter and length) must be compatible with the connections and/or the hole into which the tube is inserted.

- Do not use improperly
- Do not place it in an inappropriate way that could generate danger or hindrance
- Do not tie a knot to close one end
- The product is sold NOT sterile.

	Keep in a cool, dry place	<b>REF</b>	Product code
	Keep away from sunlight	<b>LOT</b>	Lot number
	Consult instructions for use	<b>CE</b>	Medical Device compliant with Regulation (EU) 2017/74
	Manufacturer		Caution: read instructions (warnings) carefully
<b>MD</b>	Medical Device		Date of manufacture
	Do not reuse	<b>UDI</b>	Unique device identifier

### GIMA WARRANTY TERMS

The standard GIMA B2B 12-month warranty applies.

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