WATCH FOR NURSES
OROLOGI DA INFERMIERE
MONTRES INFIRMIERES
RELOJ DE ENFERMERÍA
RELÓGIOS PARA ENFERMEIROS
KRANKENSCHWESTERUHREN
ΡΟΛΟΙ ΓΙΑ ΝΟΣΟΚΟΜΟΥΣ

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How to use the watch for nurses

We thank you for choosing our silicone watch for nurses and we hope that its use will give you full satisfaction.

Before enabling the watch, please pay attention to any movement of one of the needles. This is due to the fact that the movement drive has not yet been activated. Once the battery protection tab is removed, the needle will move in only one direction.

To enable the clock, pull it out of its silicone body and remove the protect tab on the charging mechanism. Pull the charging mechanism, rotate it to set the time and bring the mechanism into its housing to lock it. Replace the body of the watch on the opposite side of the securing pin.

Cleaning

Remove the watch from its silicone holder and clean the silicone with a cloth dampened with water and soap or alcohol at 70°. For the body of the watch, use a damp cloth as the watch is watertight.

WEIGHT: 28 grams

BATTERY: SR626SW button battery (when the battery is low, throw it into the collection container; please refer to the procedure for the collection of batteries).

BATTERY LIFE: approximately 2 years

BATTERY REPLACEMENT: on the back of the watch, there is a notch to open the housing and get to the battery. Lift the rear with a sharp and thin instrument. Remove the white plastic cover and the battery by sliding it on the side. Insert the new battery complying with the correct polarity with the "+" facing up. Replace the white cover and close the case.

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