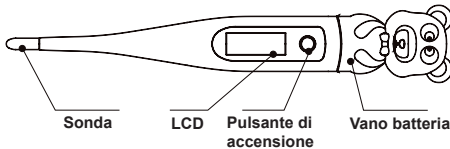
TERMOMETRO DIGITALE

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REF 25555 / DT-11G

Fabbricante
Manufacturer

Hangzhou Hua'an Medical & Health Instruments. Co., Ltd.
Building 2, Baimiao Ind. Park, Economic Develop. Zone Wuchang
310023 Hangzhou, Zhejiang - China
Made in P.R.C.

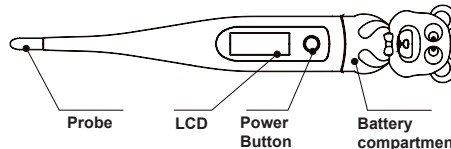
EC REP

Shanghai International Holding Corp. GmbH
Eiffestrasse 80, 20537 Hamburg - Germany

DIGITAL THERMOMETER

Carefully read the following instructions before use. The digital thermometer is a device for body temperature measurement suitable both for hospital and home use. Compared to the traditional glass and mercury models, it offers the advantage of simple reading, quick and accurate measurement, memory and alarms, etc. In particular it is safer for people and environment because of the absence of mercury. The thermometer has been calibrated by the manufacturer and further calibrations are not necessary. For any doubt on the device accuracy, please get in contact with your retailer.

Technical features

Measuring range: 32~42 °C / 89,6~107,6 °F
Accuracy: ± 0.1 °C / 0.2 °F (35.5~42.0 °C / 95.9~107.6 °F)
± 0.2 °C / 0.4 °F (32~35.5 °C / 89.6~95.9 °F)
Battery life: ca. 100 hours of uninterrupted use

Use

- Before use disinfect the probe or position a probe cover on the tip.
- Press the power button, you will hear a beep.
- After the initial control test the display will visualise the last measurement made for 2 seconds.
- When the word "Lo" appears on the display and the letter "C" or "F" starts to flash, measuring can begin.
- During measuring remain in the same place/position until the end.
- The temperature measured will appear on the display.
- Measuring is completed when a constant value is reached and the thermometer releases four beeps.
- It is possible that, continuing measuring after the beep, the value measured still increases slightly.
- The thermometer turns off automatically after approximately 10 minutes, in any case to extend the duration of the battery it is recommended to turn it off after use, pressing the button.
- To change the measuring scale (between °C and °F) turn on the thermometer and keep the ignition key pressed for 2 seconds.

Methods of measuring

- For rectum measurements, insert the probe delicately in the rectum. Do not move at the end of measuring. The measuring time ranges from approximately 40 to 60 seconds.
- For measuring under the armpits, before starting keep your arm resting on your thorax so that the armpit remains closed, for approximately 5 minutes. Position the probe in the middle of the armpit in a stable position, do not move the arm until measuring has been completed. Armpit measuring may be less than 0,5 °C - 1,5° C compared with rectal measurement, and the measuring time ranges from approximately 80 to 120 seconds.
- For under-tongue measuring, before starting do not open your mouth for approximately 5 minutes. Position the probe under your tongue in a stable position, do not open your mouth until the end of measuring. Do not eat or drink before measuring. In general oral temperature is 0,3 °C to 0,8 °C less than the rectal temperature. The measuring time ranges from approximately 50 to 70 seconds.

Disinfect the probe only with alcohol or similar, DO NOT boil it! DO NOT sterilize in autoclave!
Do not dip it in water or liquids!
Avoid exposing the unit to direct sunlight or atmospheric agents.
Do not measure except body temperature!

Battery replacement

- The battery must be replaced when the symbol  or  blinks on the display .
- Delicately withdraw the back cover and remove the old battery by means of a toothpick or a needle tool.
- Insert the new 1,5 V battery (LR/SR-41).
- Replace the back cover

Do not disassemble the unit except for battery replacement!
Do not leave the batteries or the back cover within children's reach!
The product or battery disposal must be in compliance with the standards in force in your country.

Comply with the laws in the country of use to properly dispose of the product.

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