GIMA

PROFESSIONAL MEDICAL PRODUCTS

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GB - USER MANUAL FOR REUSABLE NON-SURGICAL INSTRUMENTS

These instruments are intended to be used by medical practitioners who are specially trained on how to use and care of them. The incorrect use, poor or inappropriate maintenance can rapidly lead to deterioration of the instruments. The first time, and after every use it is recommended to clean, dry the instruments. The instruments must always be cleaned prior to sterilisation. For automated cleaning use only washing equipment with approved and certified detergents. For manual cleaning use approved and certified detergents, brush and running water. Always follow instructions on how to use the detergent; clean the instruments both when open and closed; rinse for 3 minutes and check that water also enters and exits the blind holes several times. Use completely demineralised water in the final rinse phase. Instruments that are not dried could suffer damage by corrosion. Always dry the instruments. Also make sure that the product does not come into contact with acids or other aggressive disinfectants that could corrode it. Instruments that show signs of corrosion must be discarded immediately. Always perform a visual inspection for damage or signs of wear: sharp edges must be free of dents and with continuous edges; there should be no distortion of instruments with long parts; the instruments that are part of a larger assembly, must be checked together with other assembly components; always check the rotating movement of hinges, which must not have excessive play; always check that the locking systems are working. Whilst the instruments are under warranty, repairs and/or replacement of all defective parts, for reasons that are accepted by the man-

ufacturer, are free, with the exception of costs for labour, travel, transport, packing, etc. Damage caused by improper use of the product is not covered by the warranty

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

WARNINGS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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Gima S.p.A.