# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**BILANCIA DIGITALE IN VETRO**  
**GLASS DIGITAL SCALE**  
**PÈSE-PERSONNES EN VERRE À AFFICHAGE NUMÉRIQUE**  
**GLASPERSONENWAAGE**  
**BÁSCULA DIGITAL DE CRISTAL**  
**BALANÇA DIGITAL DE VIDRO**  
**ΨΗΦΙΑΚΗ ΖΥΓΑΡΙΑ ΑΠΟ ΓΥΑΛΙ**  
**SZKLANA WAGA CYFROWA**  
**چاگز ن م یلات یج ی د نازی م**  
**DIGITAALINEN LASIVAAKA**  
**STAKLENA DIGITALNA VAGA**  
**ÜVEGBŐL KÉSZÜLT DIGITÁLIS MÉRLEG**  
**CÂNTAR DIGITAL DIN STICLĂ**  
**DIGITAL GLASVÁG**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGS ANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**PODRĘCZNIK EKSPLOATACJI I KONSERWACJI**  
**KÄYTTÖ- JA HUOLTO-OPAS**  
**PRIRUČNIK ZA UPOTREBU I ODRŽAVANJE**  
**HASZNÁLATI ÉS KARBANTARTÁSI ÚTMUTATÓ**  
**MANUAL DE UTILIZARE ȘI ÎNTREȚINERE**  
**ANVÄNDNINGSG- OCH UNDERHÅLLSMANUAL**

**REF** GBS-810D (GIMA 27248 - 27249)

Fabbricato da/Made by/Faite par/Hecho por/Feito por/Hergestellt von/Zrobione przez/Ἐτιασμένο από/  
**GUANGDONG TRANSTEK MEDICAL ELECTRONICS.CO.LTD.**  
Jin'an Road , Minzhong, Zhongshan 528441, China  
Made in China

Importato e distribuito da/Imported and distributed by  
**GIMA S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
www.gimaitaly.com

## CARATTERISTICHE & SPECIFICHE

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## FEATURES & SPECIFICATION

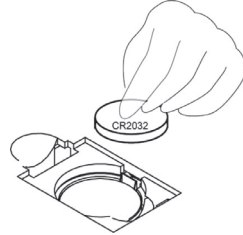
- Auto ON
- Auto OFF
- Weight Units:kg,lb,st:lb
- Capacity;150kg/330lb/23st:8lb
- Graduation: 0.1kg/0.2lb/0.2lb(1lb,≥20st)
- Overload indication: “ — — — — ”
- Low battery indication “Lo”
- Power supply: 1 x CR2032 battery

LCD Full Display


## TO CHANGE BATTERY

- Ensure +/- terminals are correct way around.
  - Use proper tool to help release the battery from the compartment if necessary.
- Please remove the insulating strip before use.

## FUNCTION KEYS
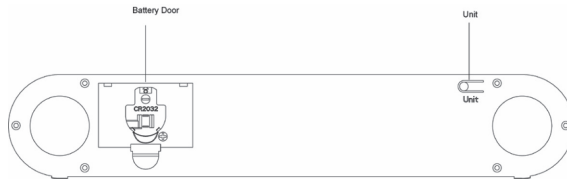
## OPERATION INSTRUCTIONS

1. Press UNIT key to switch on and select unit.
2. Place the scale on the solid and flat ground ,let the scale turn off automatically.
3. User stand on the platform of scale directly, it will flicker and lock on the LCD after the weight steady, then automatically turn off after 10 seconds.
4. If without any operation , it'll turn off automatically
5. When you don't use it ,please balanced lay it, then do not put any item on the scale.

### **KINDLY REMINDER:**

*Don't use scale on the carpet*

*In order to ensure weighing accurate, when you move the scale, please first turn on (or tap on) and wait for turn off, then, you can stand on the platform again and measure your weight. Otherwise,it will add scale own weight.*

### **DISPOSAL:**

*The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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