GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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FEATURES

The GIMA FAMILY scale is a device used to weight babies. The scale is designed for home use and includes a removable tray. Manual zero setting.

Technical specifications

Maximum load: 20 Kg / 44lb

Accuracy: 50 g (0-10 Kg), 100g (10-20 Kg)

Weight of machine: 3.0 Kg

PRESCRIPTIONS

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

After unpacking the scale, assemble the tray by fixing the two fastening screws with a screwdriver. There is no need to tare the scale. It has to be set on a flat surface (table or floor) for correct operation.

OPERATION

Before weighing the baby check the position of the needle, if the needle is not on the zero bring it to zero by rotating the adjusting knob. After adjusting the scale, slightly press down on the tray several times (1-2 Kg pressure). If after pressing the needle goes back to zero, the scale is ready to be used.

If the child can stand, you can use the scale without the tray. In this case, let the child stand directly on the base of the scale.

When the tray is removed, adjust the scale again. The needle has to be set to zero!

MAINTENANCE

The product does not require any special maintenance. The technical life of the scale depends on its use. It is resistant against most chemical products except for M.E.K. ketones, diluting substances and polish removers.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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