LETTINI VISITA
EXAMINATION COUCHES
LITS D'EXAMEN
CAMILLAS DE VISITA
MARQUESAS PARA VISITA
UNTERSUCHUNGSBETTEN
BESÖKSSÄNGEN

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
INTRODUCTION

In accord with Regulation (EU) 2017/745, this product is part of the CLASS I MEDICAL DEVICES. You should carefully read this manual before using this medical device.

DESCRIPTION

The examination couch is small in size, particularly suitable for medical practices / surgeries. The mattress, washable and fireproof, is composed of two parts, a first part fixed, a second part reclining in different positions. Depending on the model, the structure can be painted or chromed. On request: sheet holder.

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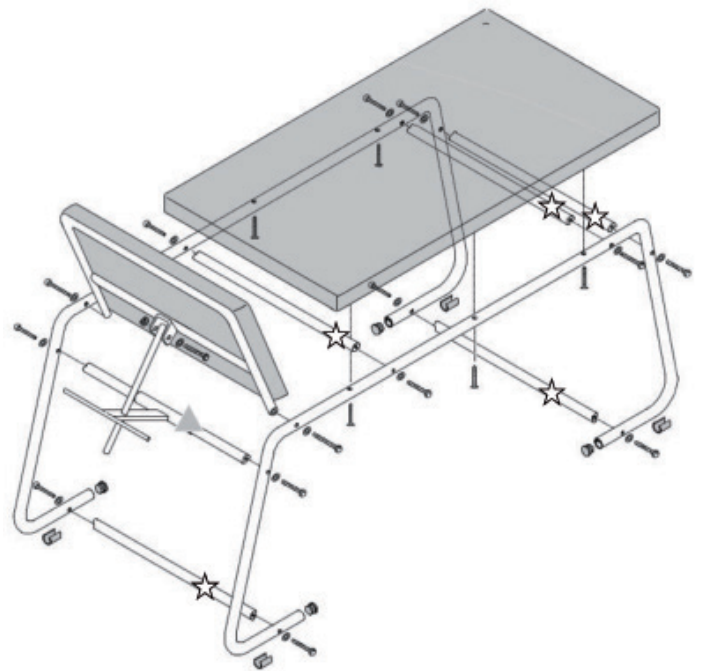The couch is completely disassembled.

Follow the assembly procedures shown on the drawing.

1. Take the legs of the couch and assemble them with the 5 crossbeams using the screws and washers.
2. Fix the crosspiece to the structure with the head support rod already mounted to the padding of the backrest using an hexagonal screw and his nut.
3. Finally fix the pads to the frame using screws.

ASSEMBLY INSTRUCTIONS

TECHNICAL DATA

Measurement	cm.193 x 62 x h73 * box 173x16x74h
Weight	26,4 kg (29,5 kg with box)
Max capacity	120 kg

* The paddings are not assembled by manufacturer but must be positioned during assembly by purchaser/user. Consequently, the overall dimensions measurements may differ slightly from those given in the data sheet.

WARNINGS!

- Check the locking nuts are tightened.
- Observe the maximum flow rates indicated in the data sheets.
- Avoid direct contact with open wounds.
- Not for use in open environments.
- When adjusting the height of the couch and the back, make sure that no one inserts hands, legs or anything into the movement area: CRUSHING RISK!
- Move the bed without lifting it from the floor.

- Before moving the bed, unlock the wheels.

MAINTENANCE AND CLEANING

- Use only original spare parts.
- Periodically check all fixing points.
- To preserve at best, it is recommended to avoid prolonged exposure to sunlight, to contact and deposit in saline and very humid environments.
- Wash the stainless steel surfaces with soapy water and use appropriate disinfectants.
- Rinse with a damp cloth and dry thoroughly.
- Do not spill on the surface cleaners containing chlorine or ammonia, fatty and oily substances.
- Do not use cleaning utensils containing iron. If necessary disinfection only use suitable non- corrosive products.

Cleaning rules for coating:

Clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches, synthetic detergents, polishes or aerosol spray. **Attention:** Normally synthetic leather light colours should not come in contact with no fixed cloured clothes (such as jeans and similar) not to cause stains and rings that cannot be cleaned up.

	Keep away from sunlight		Medical Device compliant with Regulation (EU) 2017/745		Keep in a cool, dry place
	Product code		Consult instructions for use		Manufacturer
	Date of manufacture		Lot number		Medical Device
	Caution: read instructions (warnings) carefully		Unique device identifier		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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