TITANOX s.r.l. - via Canove 2/A - Loc. Canove dé
Biazzì - 26038 Torre dé Pícenardi (CR) - Italy -
www.titanox.it info@titanox.it MADE IN ITALY

Manuale d'uso "T5000"

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Instruction manual "T5000"

Structure made of epoxy painted steel tube. Backrest adjustable by gas pump. Leg rest manually adjustable. Metal footrest manually extractable. Armrest with articulations, manually adjustable in height. Padding made of high-density polyurethane foam, fireproof class 1 IM, covering made of washable, fireproof, plastic material (leatherette) – fireproof class 1 IM. Back push handle. Mattress overall dimensions: 170x55cm. Height from floor to seat: 56 cm. Four wheels diam 100 mm, 2 of which are directional and 2 with brake. Supplied with chart holder support and blanket tray complete with fixing belts to fix oxygen bottle.

Carrying capacity: 150Kg

All models are produced eliminating sources of potential risk to the user, should unintended collisions occur, such as cutting edges or sharp corners.

These instruments are intended to be used by medical practitioners who are specially trained on how to use and care of them. Use for purposes other than what it is intended for is not allowed. The incorrect use, poor or inappropriate maintenance can rapidly lead to deterioration of the instruments.

TITANOX s.r.l. - via Canove 2/A - Loc. Canove dé
Biazzì - 26038 Torre dé Picenardi (CR) - Italy -
www.titanox.it info@titanox.it MADE IN ITALY

- Check the right tightening torque of all mobile parts before and during use.
- For cleaning, do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants, and acetone: the manufacturer company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

TITANOX WARRANTY TERMS

Titanox 12-month warranty applies from final invoice date.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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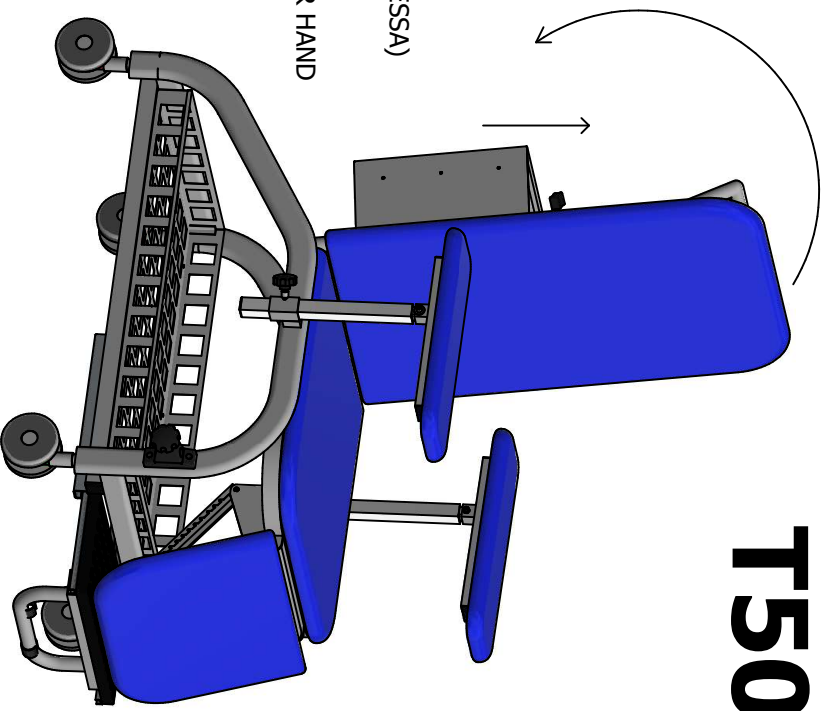
HANDLE: ALWAYS CHECK THE HANDLE NOT BE LOOSE, IN CASE IT IS LOOSEN, TIGHTEN IT

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PUSH THE LEVER UPWARD, TO LOWER THE BACKREST
(N.B. HELP THE POSITIONNING OF THE BACKREST, BY PUSHING SOFTLY WITH YOUR HAND
IF A PERSON IS LYING ON THE COUCH)

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PUSH THE LEVER TO THE LEFT TO LIFT THE BACKREST
(N.B. HELP THE POSITIONNING OF THE BACKREST, BY PUSHING SOFTLY WITH YOUR HAND
IF A PERSON IS LYING ON THE COUCH)

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