Ref						
255618-255811-255813-255820-255817-255824-255825-255823-255828-255829-255830	72-94 cm	Ø 20mm	Ø 17mm	110kg	0.38 kg	
255613-255611-255619-255617-255623-255819-255624-255620-255625-255827	86-96 cm					
250817-250813-250811	74.5-97.5 cm					
250617-250611-250613	87-97 cm					
243625-243626	74-94 cm	Ø 22mm	Ø 19mm	120kg	0.36 kg	
243645-243646-243655-243656-243635-243636	70-94 cm	Ø 20mm	Ø 17mm	120kg	0.4 kg	
250280-250281	82-92 cm	Ø 20mm	Ø 17mm	100kg	0.4 kg	
255826	90 cm	Ø 22mm	-	110 kg	0.51 kg	
250722	73-96 cm		Ø 18mm	90kg	0.52 kg	
250723	74-96,5 cm				0.58 kg	

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EN- Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (clips, handle, rubber tip..).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..)!!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your stick.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking.

These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support.

Indications

- Loss of autonomy due to walking disorders
- Balance and/or posture disorders
- Walking problems
- Rehabilitation programme after an injury or an operation

Contraindication

Don't use in case of:

- Severe cognitive disorder
- Severe loss of balance
- Severe motor or perception disorders
- Weakness in the upper limbs

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Références : 250811-13-17, 255811-13-17-19-20-23-24-25-28-29-30, 255618

Height Adjustment : For height adjustment, loose locking ring, grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position and tighten locking ring. The sticks have 1 adjusting hole on the outside pipe and 10 holes on the inside pipe, what means 10 adjusting positions. The stick can be adjusted every 2 cm.

Références : 250611-13-17, 255611-13-17-19-20-23-24-25, 255819-27, 250280, 250281

For height adjustment, loose locking ring, grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position and tighten locking ring. Prior to use always ensure that the locking pin fully protrudes from the selected hole position and by applying downward pressure on the handle of the walking stick test that the walking stick has been assembled correctly.

The stick folds in 4 elements. Prior to use always ensure that the locking ring is tightly secured in place and that, following height adjustment of the stick, the metal locking pin is always fully engaged in the selected hole position.

To unfold, release one tube at a time, guide them onto the handle tube until all tubes are connected together. Check that all tubes are seated in place.

To fold, pull the bottom two tubes apart and fold onto each other. Repeat this operation with the remaining tubes. Place into the bag when folded.

References : 250722-250723

Both sticks are equipped with an integrated lamp (no batteries delivered with the product) and extra large tips for a better stability.

References : 24362*, 24363*, 24364*, 2435* (anatomical sticks)

For the height adjustment, please see the ref 250611 10 times adjustable.

Keep the sticks close to your body while going to avoid accidents.

3. Cleaning

The stick has to be cleaned regularly with a mild cleaning agent and a soft cloth. NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

4. Materials

Tube in anodised aluminium and polypropylen handle.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Check the table at the top of the user's manual.

This product has a 24 months warranty, from the date of purchase, against

any manufacturing defect. Standards: DIN-EN-12182: 1999, EN-ISO 11334-1, DIN-EN-ISO 10993-1. Biocompatible. Duration of use: 2 years.

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