Spray nasale salino per **BAMBINI**

# SINUS **C**

## 20 ml Spray nasale salino per **BAMBINI**

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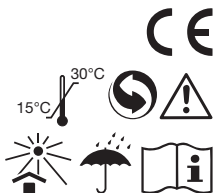
**Properties:** Sinus C is an isotonic saline solution of physiological concentration and pH. Sinus C also contains a constituent which moistens the nasal mucosa. Sinus C helps you to clean your child's nose in depth in a gentle and soothing manner.

### Why should you clean your Children's nose?

The nature method of cleaning the nose is blowing it. A clean nose helps eliminate nasal problems. Children and infants do not know how to blow their noses. For this reason, your children's nose must be regularly cleaned. The best way of doing so is to wash the nostrils with isotonic saline solution which dilutes the nasal secretions while maintaining the humidity of mucous membranes.

**Usage & Application:** Sinus C is recommended for children, and infants. When you use the Spray, apply it once or twice in each nostril, several times a day, depending on the case.

Wash applicator with soap and water after use.

**Storage:** Store between 15-30°C.

Use within 12 months of opening.

**Packaging:** Nasal Spray Bottle 20 ml.

In case of accidental eye exposure, immediately and thoroughly flush with fresh water, also under eyelids, for at least 15 minutes.

Seek for medical attention if irritation persists.

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