EFFECT 100 MATERASSO AD ACQUA

EFFECT 100 WATER MATTRESS

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INTENDED USE

- Static mattress overlay.
- Made with heavy duty PVC for durability and comfort.
- Constructed with 3 independent baffles for better body contour fit and pressure relief.
- Outer air frame provides additional support and security.

BEFORE USE



1. Unpack the content from packaging.
2. Check all contents:
 - A. Base frame x 1
 - B. Water baffles x 3
 - C. Hand pump x 1
 - D. Water hose x 1
 - E. User manual leaflet x 1
3. Visually inspect any damage such as puncture.
4. Be sure the appliance (i.e. bed frame) or surface this mattress intended to place on is strong and sturdy enough to hold the mattress water-filled weight and user's weight combined.
5. Read this instruction carefully before use.

INSTALLATION


1. Place the main base air frame on an even flat surface.
2. Use the hand pump provided to inflate the air frame by connecting air valves.

Be careful NOT to overinflate. Air frame should only be sturdy enough to securely hold 3 water baffles.
3. Fill 3 independent water baffles with normal household tap water. Be careful NOT to overflow baffles with excessive water. It should taken user's weight and height into account and consideration. An ideal volume is enough to support user's weight while not bottoming out and mattress remains steady on the bed frame. Please apply common sense and judgment.

MATTRESS COMPONENTS AND PARTS ILLUSTRATED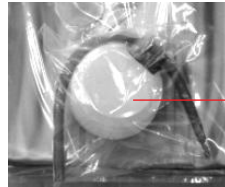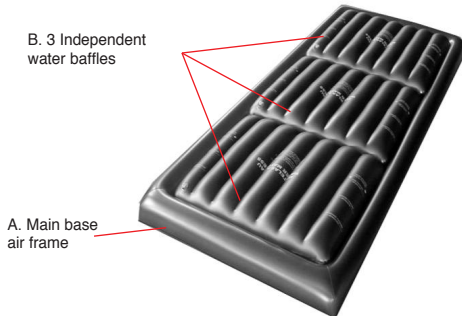

C. Hand pump

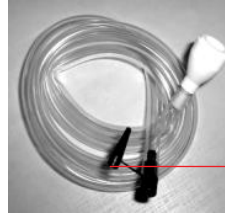D. Water hose

CAUTION

1. DO NOT place this mattress near any sharp edges or objects such as knives.
2. DO NOT place this mattress near open/exposed electrical wires or appliances.
3. DO NOT jump or bounce on the mattress or allowing children to use it as a toy.
4. DO NOT move or relocate this mattress while baffles are filled with water.

Please empty the water from the baffles before moving.
5. THIS IS NOT A FLOATATION DEVICE OR CRAFT CANNOT BE USED AS SUCH.

CLEANING GUIDELINES

- Use damp cloth with mild detergent or disinfectant solution and wipe thoroughly.
- Do not use abrasive or corrosive type of chemical solution.
- Do not machine wash.
- Do not use dryer to dry. To prevent mildew build-up, use soft cotton cloth to dry or place it in an open air environment allows it to be aerated until it is fully dried.

TECHNICAL SPECIFICATIONS

Effect 100 Water Mattress
Material: PVC 0.40mm + K75
Overall Size: 200 (L) x 93 (W) x 11.5 cm (H)
Water Baffle Size: 69 (L) x 60 (W) x 11.5 cm (H)
Weight: 3.5kg

	Caution: read instructions (warnings) carefully	REF	Product code
EC REF	Authorized representative in the European community	LOT	Lot number
	Keep away from sunlight		Manufacturer
CE	Medical Device compliant with Regulation (EU) 2017/745	MD	Medical Device
	Consult instructions for use		Keep in a cool, dry place
	Date of manufacture		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF	Effect 100 (28502)	MD	CE
	Airflo (Xiamen) Medical Co., Ltd 1F, 3F, 4F, No. 6, East Haijing Road, Haicang Xiamen, Fujian, China Made in China		
EC REF	Emergo Europe B.V. Prinsessegracht 20 2514 AP The Hague The Netherlands		

Importato da / Imported by:
Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com