# BACINELLA SHAMPOO SHAMPOO BASIN

## Manuale d'uso - User manual

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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## INTENDED USE

1. Designed for bedridden or immobilized persons.

## CONTENTS

1. Shampoo basin with drainage hose attached.
2. Water pouch with shower head and clip.
3. A plastic pipe and string for better water pouch handling and hanging.
4. A repair kit.

1. Shampoo basin

2. Water pouch kit

3. A plastic pipe and string

Drainage plug

5. Place shampoo basin under patient's neck area, adjust for comfort when necessary.

Application illustration

## FEATURES

1. A 20cm deep shampoo basin with dual tubing for sturdiness and comfort.
2. It comes with a drainage plug with hose for ease of discharging soiled water during or after use.
3. A separate water pouch for easy water transport where water source may not be conveniently accessible. It also comes with a shower head and an adjustable clip for ease of water flow control.

## SETUP INSTRUCTION

1. Inflate the shampoo basin either by mouth or with air pump (not included).  
**NOTE:** the air valve has a one-way valve inside to help to prevent air from leaking out. To inflate, squeeze air valve by its neck to help to open the one-way valve inside.
2. Place the shampoo basin on a flat and semi-rigid surface for better use.
3. Place the drainage hose to a water bucket (not included).  
**NOTE:** water bucket must be placed lower than the shampoo basin to ensure soiled water flows downwards.
4. Unplug the drainage plug for water drainage while in use.

	Caution: read instructions (warnings) carefully	<b>REF</b>	Product code		Consult instructions for use
<b>EC REP</b>	Authorized representative in the European community	<b>LOT</b>	Lot number		Date of manufacture
	Keep away from sunlight		Manufacturer		Keep in a cool, dry place
<b>CE</b>	Medical Device compliant with Regulation (EU) 2017/745	<b>MD</b>	Medical Device		

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

**REF C00303 (28507)**

Airflo (Xiamen) Medical Co., Ltd  
1F, 3F, 4F, No. 6, East Haijing Road,  
Haicang Xiamen, Fujian, China  
Made in China

Emergo Europe B.V.  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Importato da / Imported by:  
**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)