GIMA

PROFESSIONAL MEDICAL PRODUCTS

MANIGLIONE A VENTOSA
SUCTION GRAB BAR
BARRE D'APPUI À VENTOUSE
SAUGNAPFHALTEGRIF
TIRADOR DE VENTOSA
MANILHA A VENTOSA
ΛΑΒΗ ΜΕ ΒΕΝΤΟΥΖΑ
مقبض محجمي أبير

Manuale d'uso - User manual - Notice d'utilisation - Gebrauchsanweisung
Manual de uso - Guia para utilização - Οδηγίες χρήσης - دليل الإستعمال والرعاية

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- The suction cup grab bar is not designed to hold or support your full body weight.
- This grab bar is designed to assist your natural motion, as when stepping into or out of a shower. Do not stress by pulling or with other movements.
- Always check the attachment and holding condition of suction cup through windows every time before each use.
- Do not attach to moving surfaces.
- Maximum User Weight : 60 kg (vertical); 45 kg (lateral)
- Only for product code 28974 - Maximum User Weight: 85 kg (vertical); 60 kg (lateral)

The manufacturer will not be held responsible for damages to people or things if the product is used for non compliant use or for incorrect or unsafe use. It is designed for use only on smooth and non-porous surfaces; tile, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces.

FITTING

1. Thoroughly clean and the surface with alcohol wipe and allow to dry and check the condition of the suction cups. Do not use if damaged, cracked or split.
2. Raise both levers, place the suction grab bar so that both suction cups lie firmly and completely on a smooth surface.
3. Hold one end and press on one lever with your palm, You should feel resistance as the suction cup tightens.

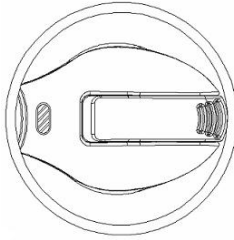
REMOVAL


1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Always to keep the levers flip up until next use.

PATENTED SUCTION INDICATOR

“GREEN” means the correct attaching, otherwise, it will turn into the “RED” request to check the mounting surface and re-attach again. Do not place over joints or grout lines.

“GREEN” = OK

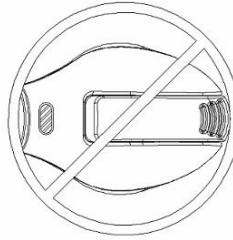“RED” = REMOUNT

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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