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ENGLISH

INTENDED USAGE:

The products of the KING Folding Bar line allow the user to increase independence and stability during the daily use of the bathroom or the toilet bowl more specifically. They work as a robust and safe handle, on which the user can hold on to relieve part of the body weight from the legs, reducing muscle fatigue due to standing position as well as slip and injuries risks. After use, the folding bar can be easily put back in vertical position and locked by its safety hook, preventing accidental falls in horizontal position. The folding bar is available in different materials and length, and thanks to the folding and locking mechanisms, it is suitable also for compact bathrooms. The folding bar can be provided with roll holder, keeping it in a more accessible position and avoiding spine twisting movements to reach it.

INDICATIONS AND CONTRAINDICATIONS:

The KING Folding Bar is aimed towards elderly people, wheelchair users and to all those who need a further support during the use of the bathroom. The user is required to have sufficient ability to grip firmly the folding bar as well as sufficient autonomy.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the folding bar is firmly fixed to the wall.
- During assembly, strictly adhere to what is shown in the User Manual.
- Do not modify the folding bar in any way.
- Do not use the folding bar if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- Do not touch the folding bar with greasy hands. Slippery surface may reduce the grip and could cause a fall.
- Do not use the folding bar for a different purpose from the intended one: it must not be used to replace a stair.