GIMA

PROFESSIONAL MEDICAL PRODUCTS

OTOSCOPIO F.O. XENON VISIO 2000 - 3,5V **VISIO 2000 F.O.XENON OTOSCOPE - 3.5V** **OTOSCOPE F.O. XÉNON VISIO 2000 - 3,5V** **OTOSCOPIO F.O. XENÓN VISIO 2000 - 3,5 V**

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OTOSCOPE

The Otoscope is designed to examine the external ear and for general non-invasive examination.

Features

- Made from Impact Resistant re-enforced plastic and highly reliable corrosion resistant stainless steel.
- Easy replacement of lamp.
- Distal fibre optic illumination with super bright 3.5V Xenon/Halogen.
- Insufflation port pneumatic testing of tympanic mobility.
- Swivel type window with 3x Magnification lense for sharp images and minimal Reflection.

The Otoscope should only be used to examine the ear when a tip is fitted. Pressure should be applied with great care when doing pneumatic testing.

OPERATING INSTRUCTION

Attach the otoscope head to the handle by bayonet locking system, it is necessary to hold the battery handle and head in each hand and match the bayonet grooves on both the head and handle, push the head down and turn the head clock wise as in Fig.1.

1. Check that the connection between the head and handle is perfect and that the rheostat works properly.
2. Push the button (F) on the handle and twist the black knob (G) towards the left hand side.
The unit is now in operating position. The intensity of the light can be adjusted by turning the knob clock wise or counter clock wise.
3. Switch off the unit after use to preserve batteries. Twist the black knob towards right until an audible click sound is heard once the button comes to its off position.

Attaching a tip

The Otoscope may only be inserted into the auditory canal when a disposable (grey) or reusable (black) tip is fitted. Push the tip onto the instrument so that the internal projection engages in the slit (A) as in Fig.1. Twist to the right to lock. Do not reuse the disposable speculum, because may cause patient-to-patient contamination.

Changing the lamp

With the Otoscope, first unlock the head from the handle and pull the lamp (D) downwards as in Fig.2. Push in the new lamp.

Magnifying lens

The lens (B) has 3 x magnifications and can be swiveled aside for instrumentation.

Insufflations port

An insufflations bulb can be fitted to the port (C). A separately Insufflations bulb is used for pneumatic testing of tympanic mobility.

General illumination

The Otoscope can be used with or without a tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

CLEANING AND STERILIZATION

Otoscope head

Clean the outside with a soft cloth, inside with a cotton wool bud, if necessary soaked in disinfectant or water.

Do not use rough cloth for cleaning the lens to avoid scratches.

Reusable Ear tips

These can be cleaned, disinfected or sterilized by any conventional method. Autoclaving is allowed. (121°C)

OPERATING THE HANDLE

Push button (A) in Fig.3 inward and rotate the knob clock wise which will turn the power on and the lamp will light up. This position is indicated by a RED indicator comes into view.

To switch off the instrument rotate the knob counter clock wise till an audible sound is heard.

Changing the batteries

Unscrew the end-cap (E) and shake out the old batteries.

Insert new batteries as Shown in Fig.3. Take care that the poles point in the right direction.

CLEANING

The handle can be cleaned with a cloth soaked with disinfectant however it cannot be dipped into liquids.

Caution:

- Do not use the equipment in case it is damaged. Apply to your Retailer.
- Periodically check the battery conditions, making sure that no sign of corrosion or oxidation is present. In case of necessity replace them with new ones.
- Carefully handle the batteries as the liquids can irritate skin and eyes.
- Before being used, thoroughly check the product. The same Operation shall be carried out after cleaning.
- Check that the connection between the head and the handle is perfect and that the On/Off Button works correctly.
- If the light is intermittent or in case it does not turn on, check the bulb, the batteries and the electrical contacts.

	Caution: read instructions (warnings) carefully		Follow instructions for use
	Keep in a cool, dry place		Keep away from sunlight
	Manufacturer		Date of manufacture
	Product code		Lot number
	Medical Device compliant with Regulation (EU) 2017/745		Type B applied part
	WEEE disposal		Medical Device

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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