GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

STETOSCOPI GIMA - GIMA STETHOSCOPES
STÉTHOSCOPES BIAURICULAIRES GIMA
ESTETOSCOPIOS GIMA - ESTETOSCÓPIO GIMA
GIMA STETHOSKOPE - GIMA SZTETOSZKÓP
STETOSKOPY GIMA - GIMA STETOSKOP
ΣΤΗΘΟΣΚΟΠΙΑ GIMA - GIMA السماعة الطبية

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso
Guia para utilização - Gebrauchs- und instandhaltungsanleitung
Felhasználói és karbantartási leírás - Podręcznik obsługi i konserwacji
Användar- och underhållsmanual - Οδηγίες χρήσης
دليل الإستعمال والرعاية

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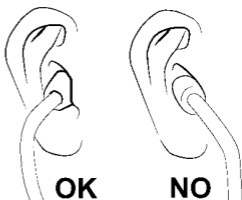
STETHOSCOPES GIMA

Congratulations for choosing a stethoscope of the series GIMA, as a guarantee of quality which has always characterized our products.

The product, according to the model and relevant features, is suitable to be used by various professional figures: cardiologists, general doctors, paediatricians, nurses and paramedics.

Bow adjustment

Position the bow so that its inclination is about 15° and the eartips are in line with the nose bridge (see picture). This way the sound will be sharp and highpitched.

Adjustment of the bow tension

To reduce the tension of the bow, firmly hold the bow with your hands in the middle of the “Y” (tubes branching) and your thumbs on the sides. Bend the bow towards the outside until the desired tension is reached.

To increase the tension, seize the eartips and cross the tubes of the bow until the desired tension is reached.

Attention: an excessive tension could weaken the bow.

Replacement of the diaphragm

Although the diaphragm is solid and planned to last long, it could be necessary to replace it.

In this case proceed as follows:

- Remove the backing ring, paying attention not to damage it (for screw models, counter clockwise rotate the ring), then pull the diaphragm out.
- Fit the new diaphragm and reposition the backing ring making sure that it is well fixed (for screw models clockwise rotate the ring).

Selection of frequency / cushion for double-head models only

180° rotate the head to change the listen modes:
select diaphragm to listen to high frequency sounds or open
bell to listen to low frequency sounds.

To know which side is active, position the eartips on your ears
and exert a light pressure on the diaphragm or on the bell
hole.

With regular and accurate maintenance your stethoscope will
give you excellent long-term performance:

- Avoid the contact with heat, cold sources, solvents
and oils.
- Avoid exposing the stethoscope to direct sunlight
for a long time and do not heat sterilize (autoclave).
- Do not dip the product in liquids. Clean it with
alcohol or suitable detergent and a soft cloth.
- Avoid damaging the diaphragm and the heavy wall
tubing with sharpened objects.
- Periodically unscrew the eartips to clean them.
Always check that they are correctly screwed down.
Do not use the stethoscope without eartips
and diaphragm.

WARRANTY

Thank you for having acquired our product.

This product meets the most stringent requirements regarding
the selection of quality manufacturing materials and also the
final control.

The product has a 12-month warranty, valid from the date it is
delivered by GIMA.

During the guarantee period free repair and/or replacement
of any defective parts due to faulty manufacture will be given,
labour, postal, transport and packaging charges etc. are not
included. The guarantee therefore excludes components
subject to wear and tear such as parts in rubber or PVC, etc.
Faulty appliances must be returned only to the retailer where
the item was bought. Any item dispatched directly to us will
be refused.

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