GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

STETOSCOPI JOTARAP - JOTARAP STETHOSCOPES STÉTHOSCOPES BIAURICULAIRES JOTARAP ESTETOSCOPIOS JOTARAP - ESTETOSCÓPIO JOTARAP JOTARAP STETHOSKOPE - JOTARAP SZTETOSZKÓP STETOSKOPY JOTARAP - STETOSKOP JOTARAP ΣΤΗΘΟΣΚΟΠΙΑ JOTARAP - JOTARAP السماعة الطبية

Manuale d'uso - User manual - Manuel de l'utilisateur - Guia de uso - Guia para utilização
Gebrauchs- und instandhaltungsanleitung - Felhasználói és karbantartási leírás
Podręcznik obsługi i konserwacji - Användar- och underhållsmanual
Οδηγίες χρήσης - دليل الإستعمال والرعاية

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SPECIFICATIONS

5 - WAYS CONVERTIBLE

Converting in 5 different ways, this instrument takes the places of all other stethoscopes and actually functions as 5 stethoscopes for 5 specific purposes in one unit.

Designed for all purposes, 3 different sizes of bell type chestpieces are furnished in addition to the original 2 diaphragms, large and small, making up one complete set. Therefore each diaphragm or bell, suits any specific purpose.

1 LARGE DIAPHRAGM

Generally, this chestpiece is used to detect heart sounds of low frequency, diastolic and third heart sounds, as well as first and second heart sounds. The high pitched murmurs emitting from the heart is also detectable.

2 SMALL DIAPHRAGM

This is designed to hear infant heart sounds of high pitched nature.

3 ADULT SIZE BELL

To pinpoint low and medium pitched sounds and murmurs, this bell is very useful.

4 MEDIUM SIZE BELL

Concentrates on narrow spots such as between ribs or any other small places to detect low and medium pitched sounds or murmurs.

5 INFANT SIZE BELL

Best suits the examination of infants and small children, low and medium pitched sound and murmurs emitting from infant heart are best sensed by this bell.

You will find all these combinations are astonishingly well designed and their high sensitivities will enable you to clearly discern low and high frequencies and to hear the faintest murmurs with ease.

Also spare eartips of 3 different sizes are furnished enabling the user to choose a pair that best fits him for comfortable use. They are packed in a compact small case put with 3 bell type chestpieces, and 2 spare transparent plastic diaphragms, large and small.

HOW TO USE

Please, see the included documentation before use.

JOTARAP must be used only by specialized medical or paramedical staff.

The JOTARAP is dual headed and readily furnished with large and small size diaphragms fitted on revolvable twin head drum which contains leakage free stem valve made in hair thin precision.

- A** First, put the binaural tips into ears.
- B** A soft tapping of the diaphragm by your finger will let the sounds come through and you can tell if it is in working position.
- C** If you don't hear the tapping sound, rotate the head by 180° where it clicks settle in the ready position, facing opposite side.
- D** Then, tap again and you will hear the tapping sounds which means the instrument is correctly set for use.
- E** Now you can diagnose the examinee with the stethoscope.
- F** Go through the same procedures when you convert from one chestpiece to another.

JOTARAP GIMA Stethoscope comes as a complete set with the following components:

1. Chestpiece with large and small diaphragm	1pc
2. Binaurals, chrome plated with eartips	1 pair
3. Heavy wall tubing with metal clip	1 pair
4. Interchangeable bell type chestpiece	1 pc
5. Interchangeable bell type	1 pc
6. Interchangeable bell type	1 pc
7. Spare eartips, large and small size	2 pairs
8. Spare transparent plastic large and small	2 pcs
9. plastic container for accessories	1 pc

WARRANTY

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control.

The product has a 12-month warranty, valid from the date it is delivered by GIMA. During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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