GIMA

PROFESSIONAL MEDICAL PRODUCTS

BRACCIALE PER INFUSIONE
INFUSION CUFF
BRASSARD DE PERFUSION A PRESSION
INFUSIONSMANSCHETTE
BRAZAL PARA INFUSIÓN
MANGUEIRA DE INFUSÃO
ΕΓΧΥΜΑΤΙΣΜΕΝΟ ΜΑΝΣΑΚΙ

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Infusion Cuff Set

This device is applicable to be used as the ancillary of infusion in clinic, and it is recommended to be used together with the infusion bag. Comparing with the traditional infusion method with pressure provided by gravity, this device can avoid the contamination of the ambient air as no ambient air will contact with the liquid medicine. In the meantime, it makes the paramedic more convenient as the paramedic don't need to hold the infusion set at given level accompanying the patient, especially when the patients go for eating, outdoor activity or toilet.

The device consists of infusion cuff, inflation bulb, manometer and knot. Please connect these parts before use.

The instruction of using this instrument

This device can provide necessary pressure for the infusion bag so as to ensure the liquid medicine flows from the infusion bag to the patients' body.

- Put the infusion bag into the infusion cuff, and the infusion bag will be just below the transparent net. Put the infusion cuff in safe area to avoid extrusion or falling down, fix the infusion bag by the pressure of infusion cuff;
- Pump the inflation bulb to inflate air into the inflation cuff. Observing the manometer carefully to make sure that the pressure reaches the average of the patient's systolic pressure and diastolic pressure;
- Connect the disposable infusion set with the infusion bag, and the liquid medicine will flow into the whole disposable infusion set. Make sure that there is not bubble in it, and close the infusion switch. Then it is ready for other ordinary infusion operation;
- During the infusion, the pressure will decrease gradually. Please pump the inflation bulb to increase the pressure. If the infusion speed is too low or too high, please modify the pressure in the infusion cuff by pumping the bulb to inflate or releasing deflation valve to deflate so as to get expected infusion speed;

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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